EURUS S6 INSTRUCTIONS FOR USE

(Belmont

Dear Customers Thank you for purchasing our product.

This booklet explains how to use EURUS. Before using EURUS, carefully read the operating instructions and make sure to use the product correctly. Using the product without reading these instructions may lead to an accident.

For easy access to the information contained herein, keep this booklet at hand and review it as needed.

Ask your local authorized Belmont dealer to install this product by following the installation instructions contained in the product.

If you have any questions regarding the operating instructions or this product, contact your local authorized Belmont dealer. If you find any dirt on or damage to the booklet and need a new booklet, report the catalog number indicated below to your local authorized Belmont dealer to order a new one.

This document describes the full version of the system. It may therefore cover components that are not included in the system you purchased.

| 2021-11 (; | | 3rd version) |
|------------|--|--------------|
| REF | | 1E089SCE |

1

| Dear Customers | | | | |
|---------------------|------------------------------|----|--|--|
| General Information | | | | |
| 1-1 | Intended Use of the Product | 8 | | |
| 1-2 | Compliance with Directives | 8 | | |
| 1-3 | Declaration of Conformity | 8 | | |
| 1-4 | How to Dispose of the Device | 9 | | |
| 1-5 | Disposal of Residues | 9 | | |
| 1-6 | Compatible Handpieces | 9 | | |
| 1-7 | Compatible Dental Lights | 9 | | |
| 1-8 | Symbols | 10 | | |
| 1-9 | Technical Description | 14 | | |

2 Safety Consideration

| 2-1 | Risk Level Interpretation | 15 |
|-----|---|----|
| 2-2 | Safety Precautions | 17 |
| | Points to remember when operating the product | 27 |
| 2-3 | Locations Where Caution Labels Are Attached | 29 |
| 2-4 | EMC Information | 32 |
| 2-5 | Compatible Handpieces | 36 |
| 2-6 | Compatible Dental Lights | 36 |

3 Precautions for Use

| 3-1 | Operating Precautions | . 37 |
|-----|--|------|
| 3-2 | Precautions for Using Synthetic Leathers | . 39 |

4 Product Specifications

| 4-1 | Technical | Data | 40 |
|-----------------------|-----------|----------------------------------|----|
| | 4-1-1 | Specification Variations | 40 |
| | 4-1-2 | S6 | 41 |
| 4-2 | System Ov | verview | 49 |
| 4-3 Name of Each Part | | Each Part | 52 |
| | 4-3-1 | Chair | 52 |
| | 4-3-2 | Headrest | 53 |
| | 4-3-3 | Foot Controller (wired/wireless) | 53 |

4 Product Specifications

| 4-3-4 | Doctor Unit | . 54 |
|-------|----------------|------|
| 4-3-5 | Cuspidor Unit | . 56 |
| 4-3-6 | Vac Pac Unit | . 58 |
| 4-3-7 | Assistant Unit | . 58 |
| 4-3-8 | Junction Box | . 59 |

5 Operation

| 5-1 | Preparation Before Use60 |
|------|--|
| 5-2 | After use |
| 5-3 | Operating the main switch for chair during treatment (when the main switch for chair is available) 65 |
| 5-4 | Raising/lowering the chair and raising/reclining the backrest(manual operation) 66 |
| 5-5 | Moving the chair to the preset position(automatic operation)67 |
| 5-6 | Operating the headrest (manual type)68 |
| 5-7 | Operating the headrest (electrohydraulic type)69 |
| 5-8 | Turning the armrest70 |
| 5-9 | Relationship among the angles of the backrest, legrest and footrest71 |
| 5-10 | Operating the legrest |
| 5-11 | Operating the doctor table |
| 5-12 | Control of the assistant tray74 |
| 5-13 | Handpiece |
| 5-14 | Doctor operation panel |
| 5-15 | Assistant operation panel |
| 5-16 | Cancellation function |
| 5-17 | Lock function |
| | 5-17-1 Chair lock function |
| | 5-17-2 Handpiece lock function |
| 5-18 | Micromotor |
| | Setting the micromotor slide mode (NBX/NLX plus/NLX nano/MX2/MCX) 108 |
| | Setting the maximum rotation (NBX/NLX plus/NLX nano/MX2/MCX) 109 |
| | Memory setting for starting condition after powering on (NBX/NLX plus/NLX nano/MX2/MCX) \dots 110 |
| | Gear conversion display function (NBX/NLX plus/NLX nano/MX2/MCX) 111 |
| | Overheating prevention function (NBX/NLX plus/MX2/MCX)112 |
| | NLX plus |
| | MX2 |

5 Operation

| 5-19 | Ultrasonic scaler |
|------|--|
| | Perio/Endo/Scaling switch setting122 |
| | Setting of the maximum output in the slide mode Va |
| | Setting of the maximum output in the slide mode Fix |
| | Memory setting for starting condition after powering on |
| 5-20 | Air turbine/motor |
| | Setting of the slide mode125 |
| | Setting of the maximum output126 |
| | Memory setting for starting condition after powering on |
| 5-21 | Syringes |
| 5-22 | Foot controller (wired/wireless) |
| | 5-22-1 Confirmation of the pairing of the wireless foot controller |
| | 5-22-2 How to replace batteries of the wireless foot controller |
| | 5-22-3 Cable connection of a wireless foot controller |
| | 5-22-4 Operating precautions for the wireless foot controller |
| 5-23 | Maintenance panel |
| 5-24 | Handpiece (assistant holder)135 |
| 5-25 | Assistant hose clamp |
| 5-26 | Filling a Cup |
| 5-27 | LED indicator |
| 5-28 | Water bottle |
| 5-29 | USB port |
| 5-30 | EURUS Light |
| 5-31 | 900 Dental Light |
| 5-32 | Monitor bracket |
| 5-33 | Panoramic viewer |
| 5-34 | Dental viewer |

6 Setting/Adjustment

| 6-1 | Setting for | automatic operation143 |
|-----|-------------|---|
| | 6-1-1 | Setting the treatment and entry/exit positions143 |
| | 6-1-2 | Setting the mouth rinsing position143 |
| | 6-1-3 | Freely setting the legrest angle144 |
| 6-2 | Function s | witch |
| | 6-2-1 | Setting items with the function switch146 |
| | 6-2-2 | Selecting the flushing method and carrying out flushing |

6 Setting/Adjustment

| 6-2-3 | Selecting the vacuum line washing method and conducting washing $\ldots 156$ |
|--------------|--|
| 6-2-4 | Displaying the drive air pressure |
| 6-2-5 | Setting the spray mode |
| 6-2-6 | Setting the light activation timing |
| 6-2-7 | Setting the micromotor brightness 176 |
| 6-2-8 | Setting the cupfiller water level |
| 6-2-9 | Setting the cup-filling/bowl flush interlock |
| 6-2-10 | Setting the bowl flush time |
| 6-2-11 | Setting the language |
| 6-2-12 | Setting the timer alarm tone |
| 6-2-13 | Setting the key touch volume |
| 6-2-14 | Setting the sleep mode time |
| 6-2-15 | Setting emergency cancellation of the chair lock |
| Adjusting tl | he quantity of water/air supplied from the doctor unit |
| Adjusting tl | he quantity of water/air supplied to the cuspidor unit |

7 Maintenance and Cleaning

6-3 6-4

| 7-1 | Exterior | |
|-----|-------------|--|
| | 7-1-1 | Cleaning and disinfecting the surfaces |
| 7-2 | Chair | |
| | 7-2-1 | Leather part |
| 7-3 | Doctor uni | it |
| | 7-3-1 | Touch panel |
| | 7-3-2 | Silicone mat |
| | 7-3-3 | Handle cover |
| | 7-3-4 | Waste container holder193 |
| | 7-3-5 | Instrument holder |
| | 7-3-6 | Cleaning of 77-type 3WAY syringe |
| | 7-3-7 | Handpiece hose |
| | 7-3-8 | Oil mist separator |
| 7-4 | Cuspidor un | it |
| | 7-4-1 | Assistant holder 199 |
| | 7-4-2 | Assistant hose clamp |
| | 7-4-3 | Assistant silicone mat |
| | 7-4-4 | Washing and sterilizing the vacuum handpiece and saliva ejector handpiece202 |

7 Maintenance and Cleaning

| | 7-4-5 Vacuum line / Saliva ejector line | |
|-----|---|-----|
| | 7-4-6 Cleaning of 77-type 3WAY syringe | |
| | 7-4-7 Sensor window for cupfiller | |
| | 7-4-8 Cuspidor bowl | |
| | 7-4-9 Solid collector | |
| | 7-4-10 Spittoon valve | |
| | 7-4-11 Air filter | 210 |
| | 7-4-12 Air compressor | 210 |
| | 7-4-13 Vacuum pump | 210 |
| 7-5 | How to detach battery of the doctor unit | 211 |
| 7-6 | Maintenance and inspection | |
| | Notes on daily maintenance and inspection (by the user) | |
| | Notes for periodic inspection | 214 |
| 7-7 | Detachable parts | |
| 7-8 | Storage method | 215 |

8 Maintenance by Service Engineers

| 8-1 | After-sales service | 216 |
|-----|---------------------------|-----|
| 8-2 | Service life | 216 |
| 8-3 | Period of Parts Retention | 216 |

9 Troubleshooting

| 9-1 | Troubleshooting | 217 |
|-----|---|-----|
| 9-2 | Message on the touch panel and measures to be taken | 218 |

10 Accessories and Consumables

| 10-1 | Accessories | 21 |
|------|-------------|--------|
| 10-2 | Consumables | 21 |

1-1 Intended Use of the Product

This product is an active therapeutic device intended for the exclusive use for diagnoses, treatments and relative procedures of dentistry. The product must be operated or handled by the qualified dentists or by dental staffs under the supervision of the dentist. Such dentists or dental staffs should instruct and/or assist the patients to approach to and leave from the product. Patients should not be allowed to operate or handle the product unless he/she is so instructed.

The product is supplied together with the handpieces like electric micromotor, air turbine and/or motor, scaler and so on.

1-2 Compliance with Directives

This product complies with Medical Device Directive 93/42/EEC, RoHS Directive 2011/65/EU, and RE Directive 2014/53/EU.

1-3 Declaration of Conformity

We hereby declare that the product listed below complies with the essential requirements of the Medical Device Directive: 93/42/EEC and RoHS Directive: 2011/65/EU based on category 8 of Annex I.

Product : DENTAL UNIT AND CHAIR (CLASS II a) Model : EURUS S6

" EURUS " has been defined by the rule 9 of MDD Annex IX. The product has been designed and manufactured in accordance with the European standards as the listed in the Declaration of Conformity.

Our full quality assurance system has been approved to MDD ANNEX II, excluding section 4 (Registration No. HD 60143723 0001) by Notified Body, TUV Rheinland LGA Products GmbH (NO.0197).

1-4 How to Dispose of the Device

When disposing of this product and parts replaced, carefully take infection control measures and handle them properly in accordance with the relevant laws and regulations (applicable regulations and local regulations).

In the EU area, EU Directive 2012/19/EU (Directive on Waste Electrical and Electronic Equipment [WEEE Directive]) applies to this product. Environment-conscious recycling/disposal is mandatory under this Directive.

Remove all batteries when disposing of the product.

Batteries are located inside the wireless foot controller and doctor unit.

[Reference] How to detach battery of the doctor unit [Page 211] [Reference] How to replace batteries of the wireless foot controller [Page 131]

1-5 Disposal of Residues

Ask a professional to dispose of amalgam residues.

1-6 Compatible Handpieces

Select a handpiece from the list of compatible handpieces. (For reference, see page 36.)

The connectors of our handpiece hose are designed and manufactured in accordance with ISO9168:2009 (DENTISTRY-HOSE CON-NECTORS FOR AIR DRIVEN DENTAL HANDPIECES). However, there still could be a case that the connectors of turbines or air motors may not fit into some handpieces due to the manufacturing tolerances. Have your local authorized Belmont dealer to check the connectability before purchasing the handpiece.

Except for our recommended handpieces, we shall not be liable for any problems deriving from bad connectability or their performance.

1-7 Compatible Dental Lights

Select a dental light from the list of compatible dental lights. (For reference, see page 36.)

1-8 Symbols

The symbols listed below are used on this product, on labeling, and in this booklet. Check the meaning of each symbol.

| | Switch (ON) | \bigcirc | Switch (OFF) |
|----------------|--|-----------------|---|
| ↑ | Switch for raising the chair/headrest | ₹ ↓ | Switch for lowering the chair/headrest |
| X | Switch for reclining the backrest/headrest | N. | Switch for raising the backrest/headrest |
| | Switch for changing to the chair/headrest | P | Switch for returning to the last position |
| 0 | Automatic return switch | 1 | Preset switch 1 |
| 2 | Preset switch 2 | | Dental light switch ON/OFF |
| t الل | Cup filler switch/flow control | رک بغ | Bowl flush switch/flow control |
| # | Extra switch | ⊭ * 7 | Stick switch for headrest (electrohydraulic) |
| ⊮ ` <u>→</u> ⊼ | Stick switch for manual operation | 1 🕍 2 0 | Stick switch for automatic operation |
| \\/ ት ና | Service coupler for water use | ₩ 75 | Water flow control of service coupler for water use |
| \// 7 F | Service coupler for air use | w 🕭 A | Water/air flow control of syringe spray |
| 1/2 | Water flow control of handpiece spray | Û | Draining water from air filter |
| | Protective earthing | <u> </u> | Functional earthing |
| | Caution *The base color is yellow. | | General warning sign *The base color is yellow. |

| \bigcirc | Generally prohibited activity | | Disassembly, repair or modification prohibited |
|-----------------------|--|-----------------------|---|
| 0 | Instructions for mandatory actions in general | E | Follow instructions for use *The color of black part is blue in the actual label. |
| Ŕ | Type B Applied Parts contacting the patient's body surface only | \sim | Alternating current |
| 135℃ ∫∫∫∫ | An autoclave symbol that indicates parts can be sterilized in an autoclave at temperatures up to 135°C | Ŷ | USB terminal |
| ~ | Year of manufacture | SN | Serial number |
| Ŷ | Switch for selecting the patient's memory (Adult Mode) | Ť | Switch for selecting the patient's memory (Child Mode) |
| | Switch for selecting the patient's memory (Elderly Mode) | A | Switch the user memory |
| | Dental light switch ON | | Dental light switch OFF |
| 4 | Timer switch | Ш | Cupfiller switch ON |
| | Cupfiller switch OFF | Ҁ | Bowl flush switch ON |
| с, й | Bowl flush switch OFF | | Function switch |
| Switch | Light change switch | Manual Manual Sensor | Sensor/ switch mode |
| Manual Manual Manual | Switch mode | ON : Keep light ON | ON mode |
| OFF Keep light OFF | OFF mode | Ĩ | Lock switch (unlocked) |

| | Lock switch (locked) | Heater Hon | Water heater switch ON |
|-----------------|--|---------------|--|
| U Heater | Water heater switch OFF | | Handpiece No. picked up |
| High | Handpiece heater switch High | Low | Handpiece heater switch Low |
| SS OFF | Handpiece heater switch OFF | ÷20 | Wireless foot controller indicator (paired) |
| × | Wireless foot controller indicator (not paired) | 05:00 | Flushing status for handpiece (flushing in progress) |
| | Page feed | _05:00 唢 | Flushing status for cupfiller and bowl (flushing in progress) |
| | Start | | Pause |
| × | Cancel | | Spray change switch, Air ON |
| K A | Spray change switch, Air OFF | W | Spray change switch, Water ON |
| | Spray change switch, Water OFF | | Handpiece light switch (ON) |
| * | Handpiece light switch (OFF) | ٨ | Bell switch |
| | Select switch (e.g. brightness of handpiece) | | Show the menu |
| Ø | Micromotor rotation switch for normal rotation (clockwise) | | Micromotor rotation switch for reverse rotation (counterclockwise) |
| Reciprocal mode | Reciprocal mode switch for micromotor | Ø | Micromotor rotation switch for reciprocal mode |

| + | +/- switch | \ | Store switch |
|-------------------------------------|--|----------|--|
| AUTO 5 Bev 3.0 _{Ncm} | Torque control switch | J Va | Slide mode change switch Slide mode Va |
| Eix Fix | Slide mode change switch Slide mode Fix | AUTO | No automatic reverse setting |
| AUTOS | Automatic reverse setting | AUTOD | Automatic forward setting |
| 🕸 1:1 | Gear ratio change switch | | Home switch |
| స స | Return switch | Skip | Skip switch |
| | Name registration switch | 123 | Alphanumeric switch |
| | Backspace switch | | Chair preset copy |
| | Handpiece setting copy | | Alarm volume |
| | Touch bar graph | | Alarm sound switch (active) |
| K | Alarm sound switch (muted) | ♪3 | Alarm sound type |
| H | Overheat prevention activate | e | Chair lock indicator (chair is locked!) |
| ? | Explanation page | ตุ้ | Flushing, Washing the vacuum line |
| | Vacuum/saliva ejector | 6 | Indication of cleaner |

| | Separate collection of electrical and electronic equipment | for battery | Separate collection of all batteries |
|-------------------|---|-------------|---------------------------------------|
| REF | Catalog number | EC REP | European Authorized Representative |
| CE 0197 | Third-party certification stipulated in Medical Device Directive 93/42/EEC | | Name and address of the manufacturer |

1-9 Technical Description

The following are explained in the documents listed below:

| Item | Document |
|-----------------------------|--|
| How to install this product | Installation instructions Pre-installation instructions |
| Wiring/plumbing | Installation instructions |

2-1 Risk Level Interpretation

Precautions before use

Make sure to carefully read the Safety Precautions and Operating Precautions and use the product correctly.

These precautions are intended to ensure the safe use of the product and prevent harm or damage to users or other people.

According to the magnitude of harm and damage and the degree of urgency, an incident that may be caused by misuse of the product is classified into one of the following categories: CONTRAIN-DICATION, WARNING, and CAUTION.

All of these categories are important for safety. Always follow the instructions provided.

We assume no responsibility for any accident due to failure to follow the Safety Precautions or Operating Precautions even in the event of harm or damage to users or other persons.

In such case, users or other persons who use the product without observing the Safety Precautions and Operating Precautions are responsible for any harm or damage incurred.

The graphical symbols are explained in detail below.

Once you have fully understood this explanation, read the text.

Classification by degree of harm or damage and urgency

CONTRAINDICATION

Use of the product without regard to this indication will create a hazardous condition that may result in death or serious injury.

🕂 WARNING

Improper handling of the product without regard to this indication will create a hazardous condition that may result in death or serious injury.

Improper handling of the product without regard to this indication will create a potentially hazardous condition that may result in moderate or slight injury or property damage.

The following graphical symbols are used to explain your responsibilities for using the product safely:

Graphical symbols for prohibited activity



Generally prohibited activity



Disassembly, repair or modification prohibited

Graphical symbol for mandatory instructions



Instructions for mandatory actions in general

Installing or transferring the product



Use and maintenance of the product



2-2 Safety Precautions CONTRAINDICATION

Precautions regarding installation Do not install the product near electromagnetic sources such as communication facilities or elevators. Malfunction of this product may occur in the presence of

electromagnetic interference waves.

Do not use the equipment in an explosive atmosphere (e.g., in the presence of inflammable gases). Improper use in such an atmosphere may cause injury or fire.

Use with caution in the presence of electromagnetic waves. Do not use equipment generating electromagnetic waves, such as mobile phones, around this product. Malfunction of this product may occur.

Be sure to turn off the main switch of the product when HF surgical equipment is in use.

Be sure to turn off the main switch when HF surgical equipment is in use because the noise generated from HF surgical equipment may cause incorrect operation of this product.

Never disassemble, repair or modify the product.

Individuals other than your local authorized Belmont dealer should not disassemble or repair this product.

This could lead to an accident, failure, electric shock, or fire. Never modify the product as it is extremely dangerous.



Installing or transferring the product





Precautions for installation Ask your local authorized Belmont dealer to install the product.

Make sure to place the product on a firm and flat floor. Placing the equipment on a non-flat floor may cause it to fall.

Be sure to ground the product securely. (Ask a professional to ground the product.)

Failure or electric leak may result in electric shock.

To avoid the risk of electric shock, this equipment must only be connected to a supply mains with protective earth.

Use and maintenance of the product



Keep moving parts of the product away from patients' and users' hands, fingers, and bodies. Hands, fingers or bodies may become caught in the product, causing injury.

The patient must not lie on his/her stomach, kneel in the formal seiza position (with his/her buttocks on his/her heels), or sit on the product with a child in his/her arms. The chair may move unexpectedly, causing injury.

The patient must not sit on the edge of the seat. The chair may fall or move unexpectedly, causing injury or damage to peripheral devices.

The patient must not sit in an area other than the designated location.

Excessive load must not be applied to the product.

The patient must not sit on the headrest, armrest, backrest, or legrest. This may cause injury due to falling, or damage to peripheral devices.

Excessive load must not be applied to the doctor arm or assistant arm.

The patient must not sit on the doctor arm or assistant arm. This may cause injury due to falling, or damage to peripheral devices.

A load exceeding the weight limit (3 kg) must not be applied to the doctor table.

This may cause damage or injury.

A load exceeding the weight limit (1.5 kg) must not be applied to the assistant tray.

This may cause damage or injury.





Do not wash the product with water. This may cause failure or electric shock.

Points to remember when a patient sits on the chair

As shown in the figure on the left, instruct the patient to sit on the chair and make sure his/her body fits into the chair. Ensure that the patient maintains a good posture before moving the chair.

For a young child, confirm that the buttocks are placed on the seat before raising the backrest.

The child might slide down while the chair is moving, resulting in injury.

Pay attention to patients and children.

Keep your eyes on the patient when this product is in use. Patients (especially children) may touch the control switch or system inadvertently, leading to an accident due to incorrect operation of the product.

Keep children away from this product except when giving treatment.





Precautions for getting on and off EURUS

When the patient gets on or off EURUS, the chair should always be at the lowest position and each operation must be locked by the lock switch.

Before moving a patient from a wheelchair to EURUS or vice versa, ensure that no obstacles (e.g., foot controllers) are situated around the wheelchair or the caregiver.

Contact of the feet or the wheelchair castors with obstacles (such as foot controllers) may cause injury or damage to peripheraldevices.

When the patient is moved from a wheelchair to EURUS or vice versa, he/she must be carefully monitored to prevent falling.

If the patient has to get off EURUS at a position other than the designated position for getting on and off EURUS due to power failure or in the event of an emergency, he/she must be carefully monitored to prevent falling.

Remove the patient's glasses and any accessories (such as a necklace or bracelets) before treatment. Failure to follow this instruction may hamper treatment and lead to an accident.

Precautions when raising/lowering the backrest

Before lowering the backrest, ensure that the patient's arm or hand is not sandwiched between the backrest and the seat.

Before raising the backrest, ensure that the patient's arm or hand is not sandwiched between the backrest and the armrest.

If the backrest is lowered during automatic operation, the legrest will be raised. Operate the backrest carefully to prevent the patient's toes from contacting the peripheral devices.

If the backrest is raised during automatic operation, the legrest will be lowered. Operate the backrest carefully after ensuring that the space beneath the legrest is clear of people and objects.

Precautions for operating the chair and automatic operation Before and during the chair operation, pay close attention to the operation range (seat raising/lowering direction, backrest lowering/raising direction, and legrest traveling direction), and ensure that the chair does not contact any body parts, hands or feet, or obstacles. Body parts, hands or feet, or objects may become caught in the chair, causing injury or damage to peripheral devices.

Before operating the chair, ensure that the patient is sitting in the correct position, and keep your eyes on him/her when the chair is in use.





Precautions when using the headrest (electrohydraulic)

Before operating the headrest, ensure that the patient is sitting in a normal position and keep your eyes on him/her when the chair is in use. [Reference] Points to remember when a patient sits on the chair [page 19]

Do not use the headrest at an angle at which the patient feels pain.

Before operating the headrest, ensure that no obstacles are placed around the headrest and make sure that there are no body parts or objects sandwiched between the headrest and the backrest.

Precautions when using the headrest (manual)

During the headrest operation, ensure that your fingers do not become caught in the control lever and are not sandwiched between the headrest and the backrest.

Pay attention to inadvertent use of wireless foot controllers.

If there is more than one EURUS on the same floor, do not operate a wireless foot controller connected to another product.

Precautions when using a micromotor (NBX, NLXplus, NLXnano) with an anti-heat function

When the anti-heat function is activated, do not turn on the main switch for the power cycle and continue operation. The micromotor may generate heat and cause the burns or motor's damages.

Development of subcutaneous emphysema

Cooling air (alone or spray) is released from the chip of the handpiece (micromotor and air turbine/motor). This may cause the patient to develop subcutaneous emphysema, so use cooling air with caution.

Pay close attention to a patient who has a cardiac pacemaker or defibrillator implanted.

If any abnormality occurs, immediately turn off the main switch and discontinue use of the product.

EURUS may affect the function of the pacemaker or defibrillator, leading to an accident.





Prohibition of using this equipment adjacent to or stacked with other electronic equipment

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Prohibition of placing portable RF communications equipment adjacent to this product

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm(12 inches) to any part of EURUS, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Immediately wipe off any water spillage or leakage on the floor. Decreased strength of the floor may cause the product to fall, leading to injury or damage to peripheral devices.

Clean the product thoroughly.

Poor cleaning may cause bacteria to grow, posing a health risk. See cleaning procedures on pages 189 to 210.

Be sure to conduct maintenance.

Use of the product without maintenance may cause injury or damage to peripheral devices. See maintenance information on pages 212 to 214.

Discontinue use of the product if it is broken.

In the case of a broken product, immediately discontinue use and turn off the main switch. Then, ask your local authorized Belmont dealer to repair the product.

Turn off the main switch during cleaning.

Failure to follow this instruction may cause electric shock or ignition. The product may also move unexpectedly, causing injury.

Action taken for power failure

To prevent unexpected operation of the product after recovery from power failure, turn off the main switch and put the handpiece/syringe in the instrument holder/assistant holder.

Be sure to turn off the breaker for devices when the product is not used for a long period of time.

Make sure to turn off the breaker for devices in a clinic when the product is not used for a long time for reasons such as closing time and non-consultation day.

If the breaker is not turned off, a fire may be caused by a leakage of electricity due to insulation deterioration.





Do not allow a person who exceeds the weight limit to sit on the chair. Falling or operational failure may occur. [Reference] Weight limit 4 Product Specifications, 4-1 Technical Data [Pages 43]

Do not hit or rub the product. This may cause damage to the cover or operational failure.

Use a battery specified by Belmont only for the wireless foot controller.

Do not combine a new battery with an old one and do not use different types of batteries (e.g., an alkaline battery and a nickel-hydrogen battery). Improper use of batteries may lead to battery leakage or rupture.

Do not touch leakage from a battery.

Contact of battery fluids with the eyes may result in blindness. Rinse with clean water immediately without rubbing, and consult a physician promptly.

Contact of battery fluids with the skin may cause skin disorders, so immediately rinse with clean water.

Ensure that each part operates normally, with no abnormalities detected, before operation.

Always inspect the product for abnormal findings such as loose components, backlash, tilting, vibration, sound, abnormal temperature, or bad odors.

If you feel something is wrong, immediately discontinue use of the product and turn off the main switch. Then, contact your local authorized Belmont dealer.

Read the accompanying documents for each device.

Before use, make sure to carefully read the instructions for use supplied with each device and use the device correctly.

Be sure to operate switches manually.

Make sure to operate the switches manually, except for the foot controller operated by the foot and stick switches. Failure to operate the switches by hand may cause damage or malfunction.

Carefully operate the switches to prevent a mistake.

The operator should pay close attention to people around the product and say something before operation to prevent any errors.





Peel off the film attached to the touch panel before initial use Using the touch panel with the film attached may lead to incorrect operation, resulting in injury.

Precaution when cleaning the touch panel

To clean and disinfect the touch panel, use ethanol for disinfection. Using other disinfectant may lead to incorrect operation , resulting in injury.

Be certain to wipe off all water and detergent. Otherwise, it may cause operational failure, malfunction, resulting in injury.

Precautions for moving the doctor unit

Carefully look around while moving the doctor unit. Contact with the chip of the handpiece may cause injury.

Precautions when using the cart

When using the cart within the operation range of the chair, pay attention not to collide with person or chair. Contact with the tip of the handpiece may cause injury.

Do not sit or lean on the cart. Do not step on the cart base or cart hose. Otherwise, such actions may damage the hose or cause the cart to fall resulting in injury.

Precautions when using a handpiece

In the event of heat or smoky smell, stop using the handpiece, turn off the main switch, and contact your local authorized Belmont dealer. This may cause burns or have an adverse effect on the dental pulp.

Precautions when using an ultrasonic scaler

After use, attach a dedicated cover (if any) to the scaler tip and put it in an instrument holder. Contact with the tip of the scaler may cause injury.

Precautions when using a dedicated function for NLXplus/ MX2

If a reverse-screw-shaped file is attached, make sure to select "No auto reverse function," and change the rotation direction by micromotor rotation switch. If the NLXplus/MX2 is used with "Auto reverse function/Auto forward function" selected, the file will be damaged, causing injury.





Precautions for armrest rotation

Ensure that no obstacles are placed around the armrest and fingers do not become caught in the armrest.

Do not operate the chair when the armrest is rotated. Before operating the chair, return the armrest to its original position.

Precautions when using a monitor bracket

Do not apply excessive load to or make an impact on a monitor or monitor bracket. Do not mount a monitor if it does not meet the conditions provided below to prevent damage or injury. Size, 21 inches; weight, 6 kg

Precautions when handling batteries

Remove the battery from the wireless foot controller when it is not used for more than a week.

Turn off the main switch during battery replacement. If not, and a switch or pedal may be pressed mistakenly, it will lead to incorrect operation of the product and may cause any damage to the person or property.

Put on the cleaning gloves when detaching cuspidor bowl. Cuspidor bowl is made of ceramic that might cause injury when broken.

Immediately wipe off any drug solutions or water adhered to EURUS.

Adherence of drug solutions or water to the control unit may cause operational failure or electrical leak. If drug solutions or water are adhered, immediately turn off the main switch and wipe them off with a dry, soft cloth.

Close the main water valve at the end of day.

To prevent the water leakage, make sure to close the main water valve at the end of day.

Turn off the main switch at the end of day or during a recess. Malfunction due to contact with EURUS will cause damage or injury. Safety precautions regarding water quality





Safety precautions regarding Conduct flushing before treatment.

Eluching water circuits of handpicees

If this product is not used for a long time, water retained in ducts and in the water heater will be susceptible to bacterial growth. To provide safe treatment and operate the handpiece without any trouble, never forget to perform flushing (discharge) of the water lines before starting treatment.

To inhibit the growth of various bacteria, you are recommended to carry out flushing with fresh water at the end of treatment in the morning and evening.

Time required for standard flushing of the unit water lines

| Flushing water circuits of handpieces | 40 seconds |
|--|------------|
| Air turbine Micromotor Air motor Ultrasonic scaler Syringe | |
| Flushing the water circuits of handpieces Flushing the water circuits of cupfiller and cuspidor bowl | 10 minutes |
| Air turbine Micromotor | 5 minutes |

10 coondo

5 minutes

| Ultrasonic scaler |
|-------------------|
| Syringe |
| + |
| Cupfiller |
| Bowl flush |
| |
| |

Micromotor Air motor


Time required for standard washing for the vacuum line

| Rinsing the vacuum line | 4 minutes |
|---|-----------|
| | |
| Washing the vacuum line with cleaner | 4 minutes |
| | |
| Rinsing the vacuum line+washing the vacuum line | |
| with cleaner | 8 minutes |
| | |


If the cuspidor bowl is not equipped, the functions to flush the water circuits of cupfiller and cuspidor bowl (5 minutes) are not available.

[Reference] Selection and implementation of flushing method [pages 147 to 155]

2 Safety Consideration

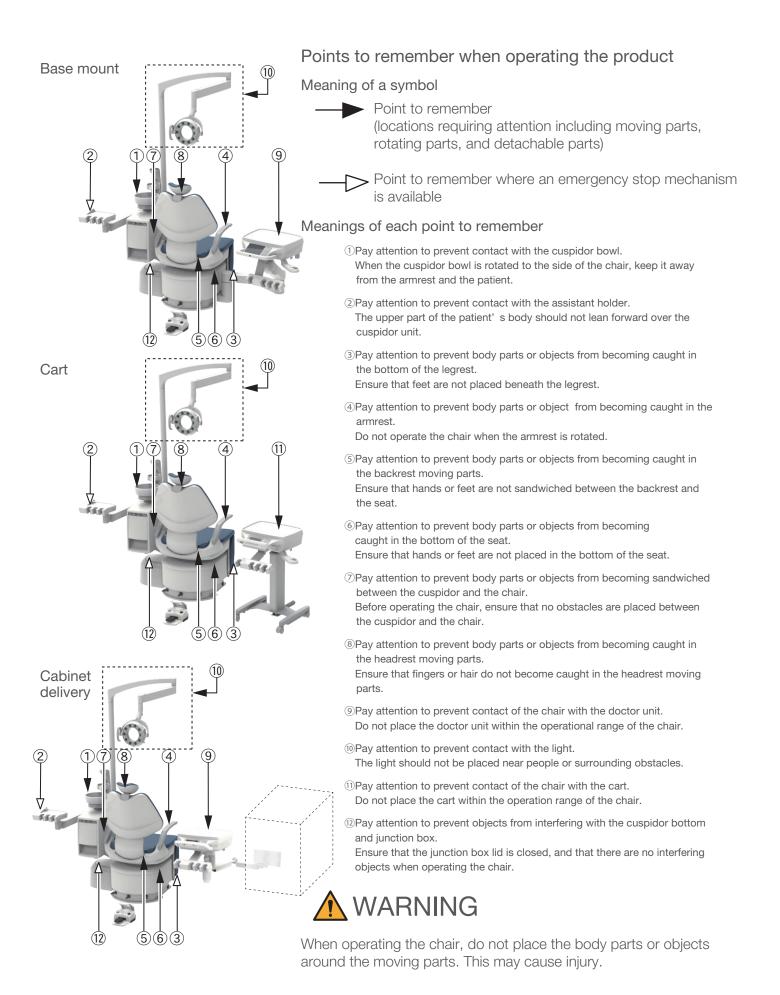


2 Safety Consideration

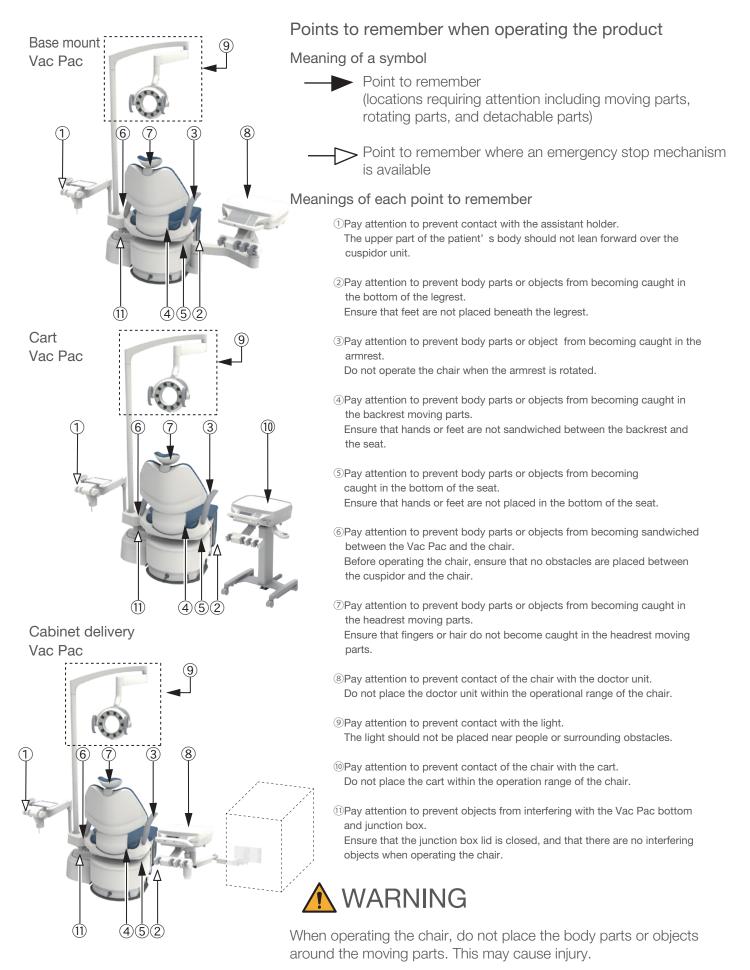


2-3 Locations Where Caution Labels Are Attached



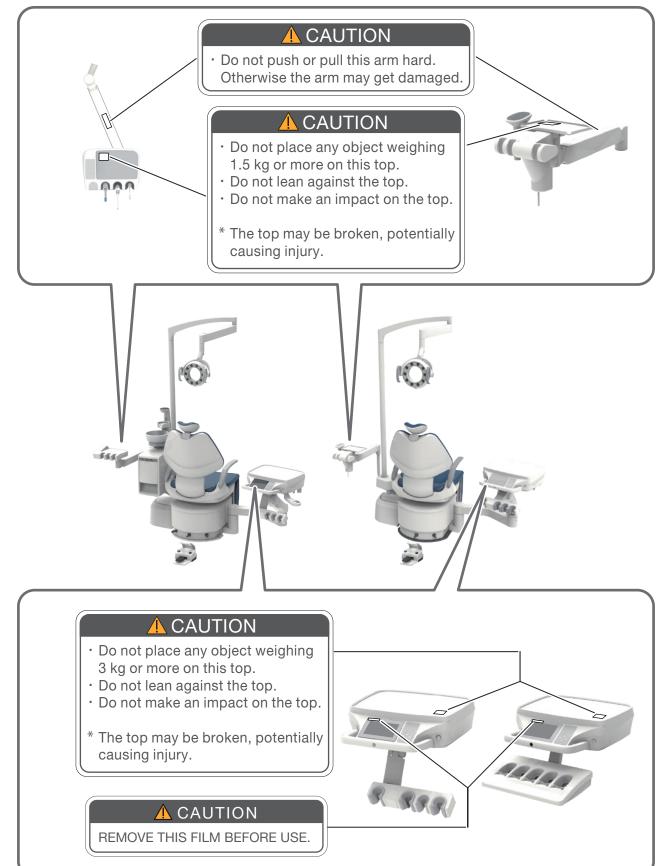


2 Safety Consideration



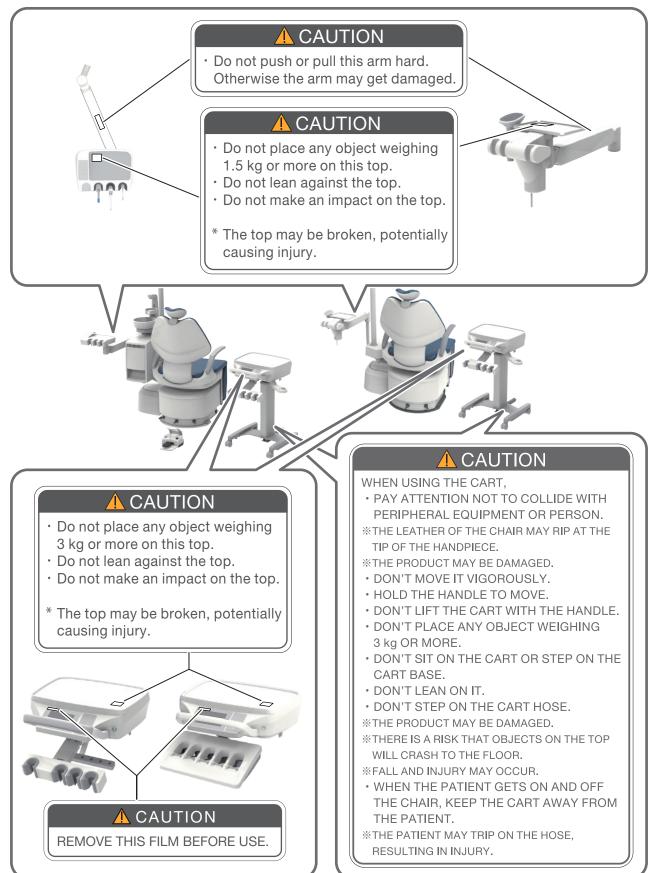

Cabinet delivery

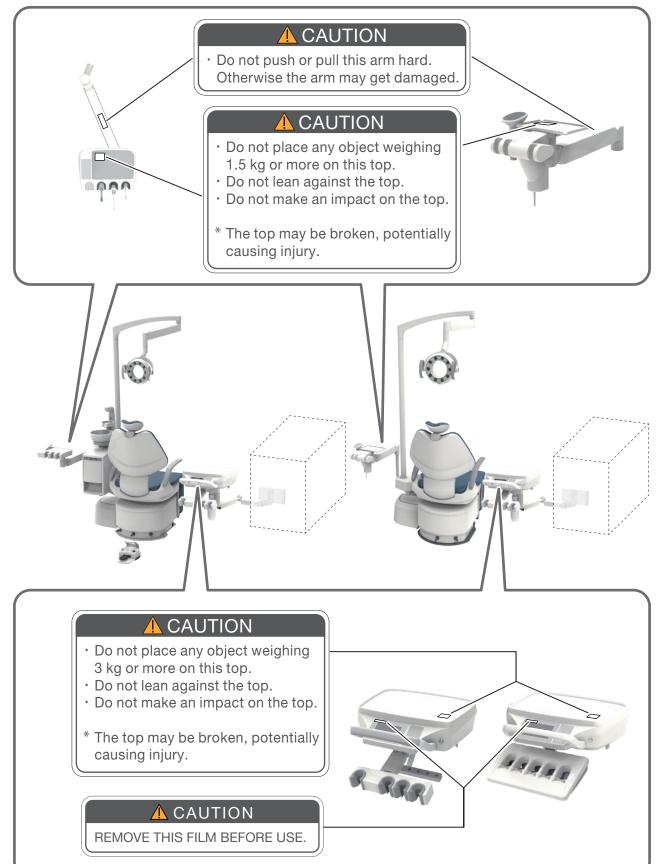

2-4 EMC Information

This product complies with EMC Standard EN60601-1-2:2015.

1. Precautions regarding EMC and compliance with accompanying documents

Medical electrical equipment requires special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in this booklet.

2. Effects of RF communication devices

Portable and mobile RF communication devices can affect medical electrical equipment.

3. Installation exclusion environment

Hospitals except for near active HF SURGICAL EQUIPMENT and the RF shielded room of an ME SYSTEM for magnetic resonance imaging, where the intensity of EM DISTURBANCES is high.

4. Electromagnetic emission declaration

| Guidance and manufacturer's declaration-electromagnetic emissions | | | |
|---|------------------------|----------|---|
| | | | ctromagnetic environment specified below. The ensure that it is used in such an environment. |
| Emissions test | Compliance Japan CE | | Electromagnetic environment—guidance |
| RF emissions CISPR 11 | Group 1 | | EURUS only uses RF energy for its internal functions.Therefore, its RF emissions are very low and are not likely to cause any interference with nearby electronic equipment. |
| RF emissions CISPR 11 | Class B | | EURUS is suitable for use in all establishments, |
| Harmonic emissions IEC 61000-3-2 | Not applicable | Class A | including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings |
| Voltage fluctuations/ Flicker emissions IEC 61000-3-3 | Not applicable | Complies | used for domestic purposes. |



Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

5. Electromagnetic immunity declaration 1

| Guidance and m | nanufacturer's de | eclaration—elect | romagnetic immunity | |
|--|--|---|--|--|
| | | | nvironment specified ire that it is used in such | |
| Immunity test | IEC 60601 test level | Compliance level | Electromagnetic environment —guidance | |
| Electrostatic discharge (ESD) IEC 61000-4-2 | ±8 kV contact ±15 kV air | ±8 kV contact ±15 kV air | Flooring should be wood, concrete, or ceramic tiles. If the floor is covered with synthetic material, the relative humidity should be at least 30%. | |
| Electrical fast transient/burst IEC 61000-4-4 | ±2 kV for power supply lines ±1 kV for input/output lines | ±2 kV for power supply lines ±1 kV for input/output lines | The mains power quality should be that of a typical commercial or hospital environment. | |
| Surge IEC 61000-4-5 | ±1kV differential mode ±2kV common mode | ±1kV differential mode ±2kV common mode | The mains power quality should be that of a typical commercial or hospital environment. | |
| Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11 | 0% UT ; 0.5 cycles ; 0.5 cycles ; 0.5 cycles ; 180°, 225°, 270° and 315° ; 1cycle and 70% UT ; 1cycle and 70% UT ; 25/30 cycles at 0°, single phase 0%UT ; 250/300 cycles | 0% UT ; 0.5 cycles 0°,45°,90°,135°, 180°, 225°, 270° and 315° 0% UT ; 1cycle and 70% UT ; 25/30 cycles at 0°, single phase 0%UT ; 250/300 cycles | The mains power quality should be that of a typical commercial or hospital environment. If the user of EURUS requires contin- ued operation during mains power interruptions, it is recommended that EURUS be powered from an uninterruptible power supply or a battery. | |
| Power frequency (50/60 Hz) magnetic field IEC 61000-4-8 | 30A/m | 30A/m | Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environ- ment. | |
| Note UT is the AC mains voltage prior to the application of the test level. | | | | |

| | | 0 | nvironment specified |
|---|--|--|--|
| an environment. | ier or user of EUF | RUS Should ensu | re that it is used in such |
| Immunity test | IEC 60601 test level | Compliance level | Electromagnetic environment —guidance |
| Conducted RF IEC 61000-4-6 | 3V 0.15MHz-80MHz 6V 0.15MHz-80MHz in ISM bands and amateur radio bands | 3V 0.15MHz-80MHz 6V 0.15MHz-80MHz in ISM bands and amateur radio bands | Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of EURUS,including cables specified by the manufacturer. Otherwise, degradation of the |
| Radiated RF IEC 61000-4-3 | 3V/m 80MHz-2.7GHz 80% AM (1 kHz) | 3V/m 80MHz-2.7GHz 80% AM (1 kHz) | performance of this equipment could result. |
| Near electromagnetic field caused by RF wireless communication devices IEC61000-4-3 | See the table on the next page (page 35) | See the table on the next page (page 35) | |

6. Electromagnetic immunity declaration 2

Guidance and manufacturer's declaration-electromagnetic immunity

7. Essential performance

Unless operated by the chair control switch, the chair will not make any movements. Unless operated by the foot controller, the handpiece will not move except for sounding a buzzer and switching the indicator on/off.

Loss or decline of essential performance may cause the chair or handpiece to move unexpectedly, causing harm or damage to the patient, operator or people or objects around the patient or operator.



Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm(12 inches) to any part of EURUS, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Near electromagnetic field caused by RF wireless communication devices

| (| | | |
|-------------------------|--|-------------------------|-------------------------------|
| Test frequency (MHz) | Modulation | IEC 60601 test level | IEC 60601 compliance level |
| 385 | Pulse modulation ^{a)} 18Hz | 27V/m | 27V/m |
| 450 | Frequency modulation ±5kHz shift 1kHz sine wave | 28V/m | 28V/m |
| 710 745 780 | Pulse modulation ^{a)} 217Hz | 9V/m | 9V/m |
| 810 870 930 | Pulse modulation ^{a)} 18Hz | 28V/m | 28V/m |
| 1720 1845 1970 | Pulse modulationª) 217Hz | 28V/m | 28V/m |
| 2450 | Pulse modulationª) 217Hz | 28V/m | 28V/m |
| 5240 5500 5785 | Pulse modulation ^{a)} 217Hz | 9V/m | 9V/m |
| Note a) | The carrier is m with a 50% dut | | square wave |

2-5 Compatible Handpieces

| Syringe | BT14 | | |
|------------------|---|--|--|
| | 77-type 3WAY | | |
| | DCI | | |
| | LUZZANI 3WAY | | |
| | LUZZANI 6WAY | | |
| | BIEN AIR BORA S36L/UNIFIX with LIGHT | | |
| Air turbine | NSK Ti-Max X | | |
| | NSK MACH-LITE XT | | |
| | NSK PANA-MAX | | |
| Air motor | BIEN AIR Aquilon 830/UNIFIX with LIGHT /PM1132 | | |
| | NSK EX-203/EX-6 | | |
| | BIEN AIR MX2 (DMX3) | | |
| | BIEN AIR MCX | | |
| Micromotor | NSK NLX plus | | |
| | NSK NLX nano | | |
| | NSK NBX | | |
| | NSK VARIOS VS170 SCALER | | |
| | NSK VARIOS VS170 LUX SCALER | | |
| Scaler | DENTSPLY CAVITRON SCALER(TYPE G139) | | |
| Scaler | SATELEC SP4055 NEWTRON | | |
| | SATELEC Xinetic | | |
| | EMS NO PAIN | | |
| Curing light | SATELEC MINI LED STD OEM | | |
| | SOPRO ACTEON SOPRO 617 | | |
| Intraoral camera | SOPRO ACTEON SOPRO CARE | | |
| | I.C.LERCHER L-CAM HD SHOT | | |
| P | - | | |

The following handpieces are compatible with this product:

2-6 Compatible Dental Lights

The following dental lights are compatible with this product:

| Dontal light | EURUS LIGHT |
|--------------|------------------|
| Dental light | 900 DENTAL LIGHT |

3-1 Operating Precautions

Immediately wipe off any drug solutions adhered to EURUS. Otherwise, they may cause deterioration or discoloration.

Do not apply heat to the product.

This may cause deterioration or discoloration.

Discoloration of resin

Resin materials are used in external components of the product. Carefully selected materials are used; however, discoloration may occur for reasons such as natural deterioration or the adherence of drug solutions.

To ensure use of EURUS for as long as possible, immediately wipe off any drug solutions adhered and avoid sunlight.

Do not place objects other than a human being on the seat.

This may cause deformation, tears, or corrosion of the leather.

Check the operation of the doctor table

Moving the doctor table upward/downward without releasing the brake may cause damage to the product.

Be sure to release the brake first by pressing the brake release switch. Then, move the doctor table upward/downward.

Precautions when using the cart

Pay attention not to collide with peripheral equipment. The leather of the chair may rip at the tip of the handpiece or the product may be damaged.

If you do not follow the instructions below, the product may be damaged or there is a risk that objects on the top will crash to the floor.

(Do not move it in rough and vigorous way. /Hold the handle to move it. /Do not lift the cart with the handle. /Do not place any object weighing 3 kg or more. /Do not sit on the cart or step on the cart base.)

Carefully handle handpieces/syringes.

Handle handpieces/syringes with care to prevent dropping. Dropping them may cause damage or deformation.

Illumination of LED indicators

Since illumination of the LED indicators acts as a guide for chair/ handpiece locking, the occurrence of mode switching, ensure that the control switches function normally as displayed before the start of work every day.

Check the operation of the compressor.

This product will not work unless air is supplied. Switch on the compressor before operating this product.

Pay attention to prevent the use of water other than tap water.

This product is intended for use with tap water, purified water, distilled water, or pure water; use of water other than those mentioned above may lead to failure.

If the product breaks down due to the use of unspecified water, it will not be covered by the guarantee.

Action to take in the case of a water leak

In the event of a water leak, close the main water valve, turn off the main switch and breaker for devices used in the clinic, and contact your local authorized Belmont dealer.

Use this product only for dental treatment.

This product is a dental unit and patient chair used for dental treatment.

Only a dentist and dental staff are allowed to use this product.

Chair's weight limit

The chair may rise slowly if a person weighs around the maximum lifting capability. [Reference] Weight limit 4 Product Specification, 4-1 Technical Data [Pages 43]

3-2 Precautions for Using Synthetic Leathers

Adherence of clothing dye

To clean and disinfect the synthetic leathers, wipe the surface with a soft cloth or paper towel moistened with FD360 manufactured by Dürr, and then wipe it with a dry cloth.

Dye may migrate from clothes or belts to the synthetic leathers of the product. If dye is adhered, wipe gently with a soft cloth impregnated with a neutral detergent diluted to about one-tenth with water as soon as possible to prevent penetration. Next, wipe with water and then thoroughly with a dry cloth.

Pay attention to prevent deterioration and dye migration due to contact.

Contact with other resin materials, coated products, solvents, and adhesive tapes may cause changes in the surface gloss, cracks, deformation or abrasion.

Dye may migrate from newspapers or printed matters to the product.

Dye may migrate from printed clothes such as T-shirts or denim clothes to the leathers of this product.

Adherence of benzene, nail polish remover, alcohol, or oil may cause discoloration, surface dissolution, changes in gloss, hardening, softening or abrasion.

Use of a bleaching agent or a sheet cleaned by bleach may cause changes in gloss or discoloration.

Placing the product near a heat source such as a heater may cause deformation or discoloration.

Shield the product from direct sunlight by using a curtain. Direct sunlight may cause surface change, shrinkage, discoloration or fading.

Placing a heavy object on the product for a long time may leave a mark or wrinkle.

4-1 Technical Data

4-1-1 Specification Variations

S6

| | | S6H | S6P | S6CH | S6CP | S6CDH | S6CDP |
|---------------|------------------|------------|------------|------------|------------|------------|------------|
| Chair | Contour | | | | | | |
| | Folding legrest | \bigcirc | \bigcirc | \bigcirc | \bigcirc | \bigcirc | \bigcirc |
| Cuspidor unit | Chair mount | \bigcirc | \bigcirc | \bigcirc | \bigcirc | 0 | 0 |
| Vac Pac unit | Chair mount | \bigcirc | \bigcirc | \bigcirc | \bigcirc | 0 | 0 |
| Doctor unit | Base mount | \bigcirc | 0 | | | | |
| delivery | Cart | | | 0 | 0 | | |
| | Cabinet delivery | | | | | \bigcirc | 0 |
| Instrument | Holder | 0 | | 0 | | 0 | |
| delivery | Place holder | | 0 | | 0 | | 0 |

| 4-1-2 S6 | |
|---|--|
| Model | Doctor unit AU-ER6-BM (Base mount) AU-ER-CT (Cart) AU-ER-CD (Cabinet delivery) Cuspidor unit AU-ER6-CM* Vac Pac unit AU-ER6-VP* Chair AC-ER6-BP* (Base mount Power Headrest) AC-ER6-BD* (Base mount Manual Headrest) AC-ER6-CP* (Cart Power Headrest) AC-ER6-CD* (Cart Manual Headrest) (* represents single or multiple strings or numbers.) |
| Classification for protection against electric shock | Class I Equipment |
| Classification according to the degree of protection against electric shock | Type B Applied Parts (handpiece/syringe/chair upholstered parts/armrest) |
| Classification according to the degree of protection against ingress of water or particulate matter | Foot controller IPX1 |
| Rated voltage | AC230 V |
| Power frequency | 50/60 Hz |
| Power input | 5.5 A |

Fuse

Fuse holder Primary circuit: 8A / 250V (Interrupting capacity: 80A / 250VAC) Operating speed: Time lag Size: 5.2 × 20 mm

Secondary circuit for 24V / 6.5A line 10A / 250V (Interrupting capacity: 1500A / 250VAC) Operating speed: Time lag Size: 5.2 × 20 mm

Secondary circuit for 24V / 5A line 6.3A / 250V (Interrupting capacity: 1500A / 250VAC) Operating speed: Time lag Size: 5.2 × 20 mm

Secondary circuit for 14V / 5A line 6.3A / 250V (Interrupting capacity: 1500A / 250VAC) Operating speed: Time lag Size: 5.2 × 20 mm

Secondary circuit for handpiece water heater 6.3A / 250V (Interrupting capacity: 1500A / 250VAC) Operating speed: Time lag Size: 5.2 × 20 mm

Secondary circuit for 6-way syringe/cup water supply and water heater 6.3A / 250V (Interrupting capacity: 1500A / 250VAC) Operating speed: Time lag Size: 5.2 × 20 mm

Power board for NLX Plus/NLX Nano/MX2/MCX 5A / 250V (Interrupting capacity: 100A / 250VAC) Operating speed: Time lag Size: φ8.35 mm

Foot controller control board 0.315A / 32V (Interrupting capacity: 1.2A / 32VAC) Operating speed: Normal operation Size: 1.6 × 0.8 mm

Chair control board 1.25A / 250V (Interrupting capacity: 1500A / 250VAC) Operating speed: Interrupting Size: 5.2 × 20 mm


AC/DC power circuit board 2.0A/250V (Interrupting capacity: 100A/300VAC) Operating speed: Time lag Size: 8.5x8.5x4 mm

| Operation mode | Non-continuous operation (motor) Maximum operating time, 3 minutes DUTY 1:15 | | |
|---------------------------------------|--|--|--|
| Weight | Cuspidor unit40 kgDoctor unitBase mount32 kgCart25 kgCabinet delivery35 kgVac Pac20 kgChair160 kg | | |
| Main air pressure | 0.5 MPa | | |
| Main water pressure | 0.2 MPa | | |
| Chair Initial height/stroke | 505 mm/265 mm | | |
| Chair raising and lowering mechanism | Electrohydraulic | | |
| Headrest | Electrohydraulic Manual | | |
| Backrest | Electrohydraulic | | |
| Seat | Backrest-linked tilt mechanism | | |
| Armrest | Left fixed/Right rotated Angle of rotation 180° | | |
| Upholstered parts | Synthetic leather | | |
| Control switch | Stick switch | | |
| Usage environment | Temperature 0°C to 40°C Humidity 10% to 95% (No condensation) Atmospheric pressure 700 to 1060 hPa | | |
| Transportation/storage environment | Temperature –20°C to 70°C Humidity 10% to 95% (No condensation) Atmospheric pressure 700 to 1060 hPa | | |
| Weight limit | Doctor table 3 kg Assistant tray 1.5 kg Chair 165 kg | | |
| Adaptation to high-oxygen environment | The product is not suitable for use in a high-oxygen environment. | | |

Refer to the rating plate for the capacity of power supply.


The specification is subject to change without notice.

Base mount

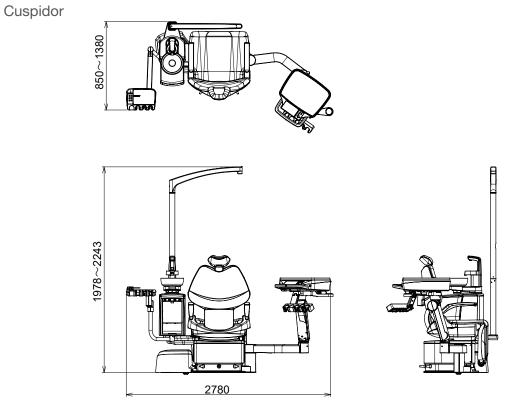


Dimensional drawing (standard values are provided)

Base mount Vac Pac

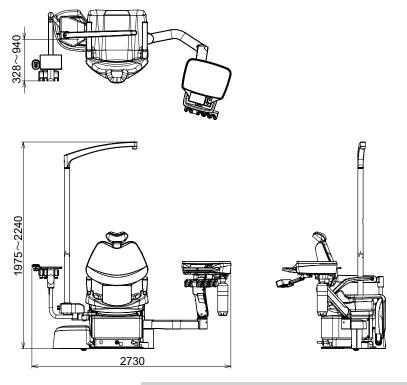

Unit: mm Tolerance in dimensions: ±10%


The specification is subject to change without notice.

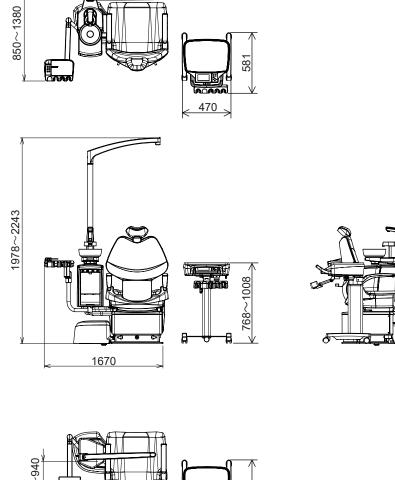

Dimensional drawing (standard values are provided)


Cart Cuspidor

Cart Vac Pac

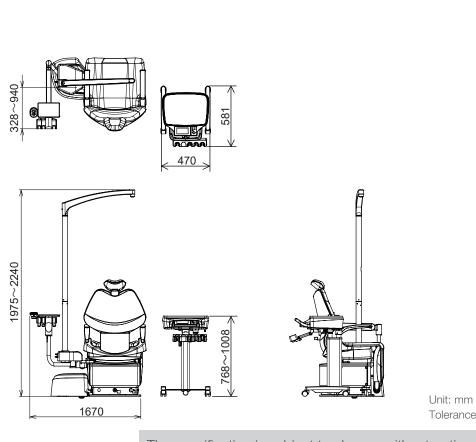


Unit: mm Tolerance in dimensions: ±10%

The specification is subject to change without notice.

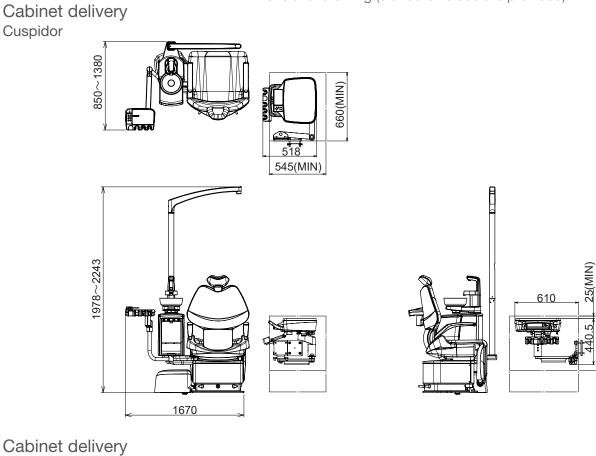

Vac Pac

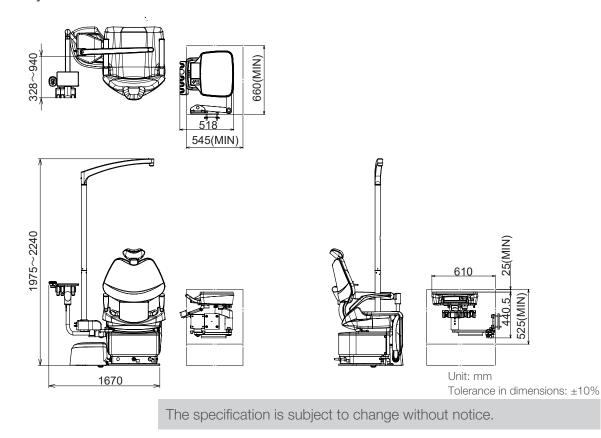


Dimensional drawing (standard values are provided)

525(MIN)

Chair

Dimensional drawing (standard values are provided)


Headrest: Electrohydraulic Base mount

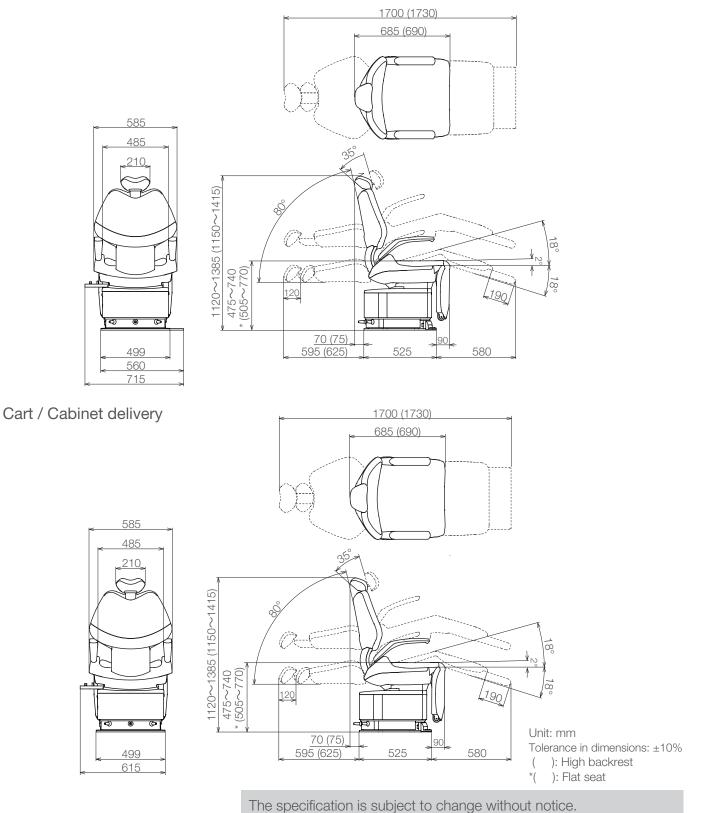


Headrest: Manual


Base mount


Dimensional drawing (standard values are provided)

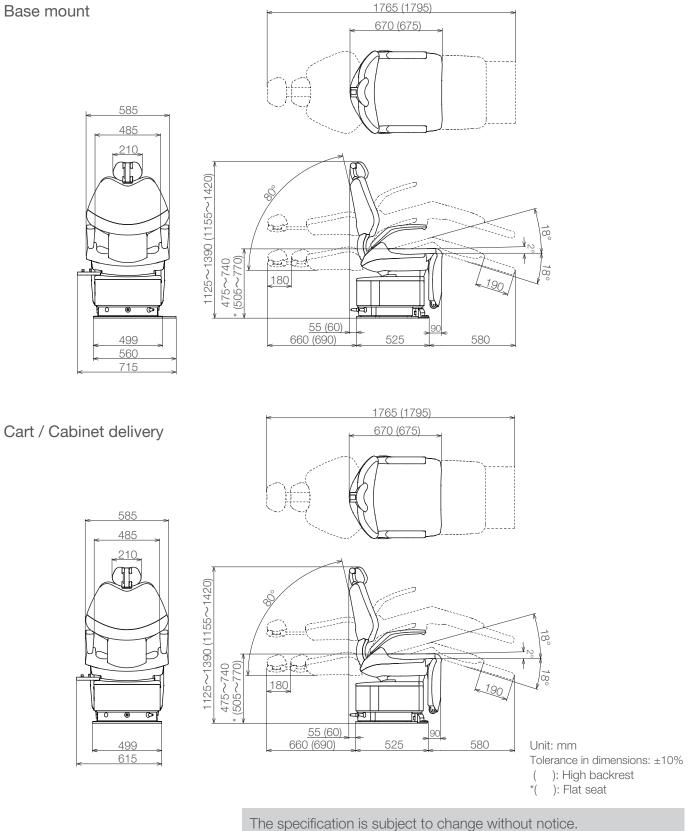





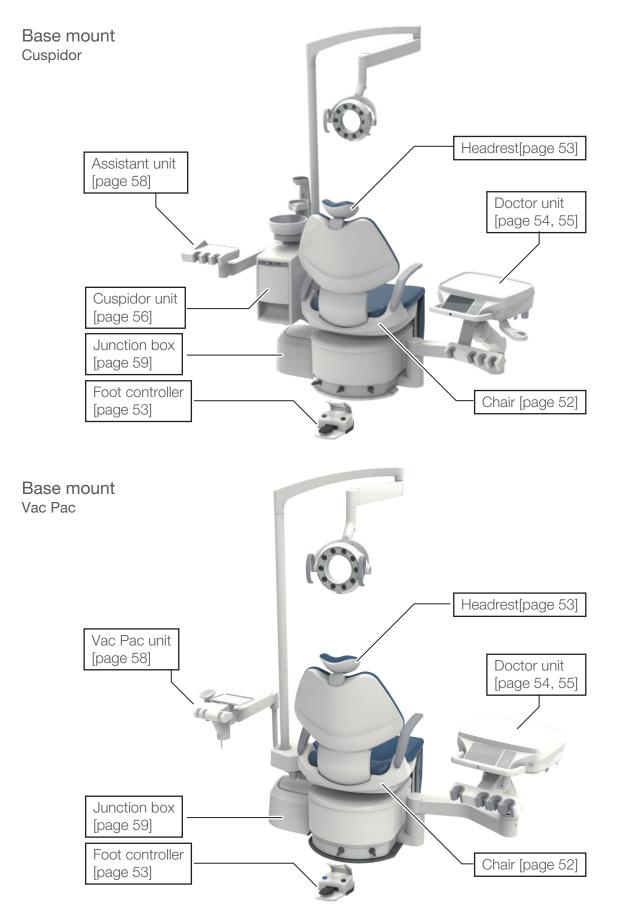

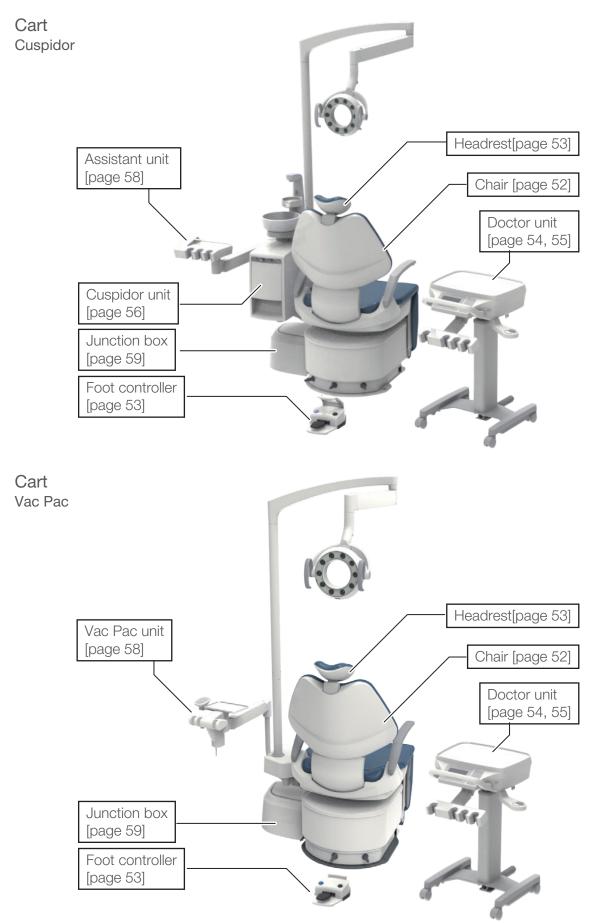

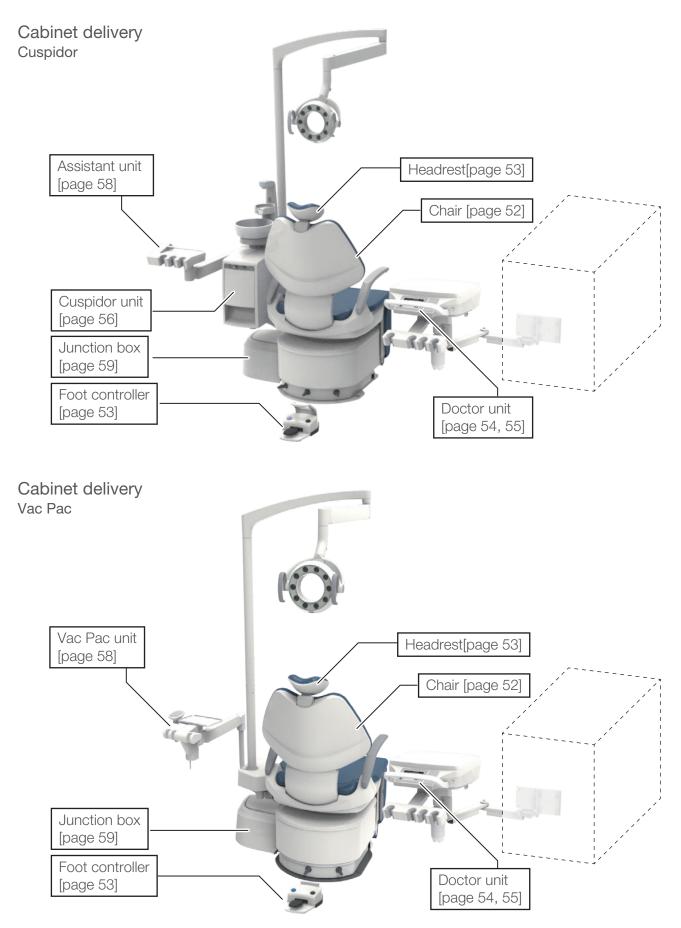

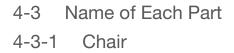

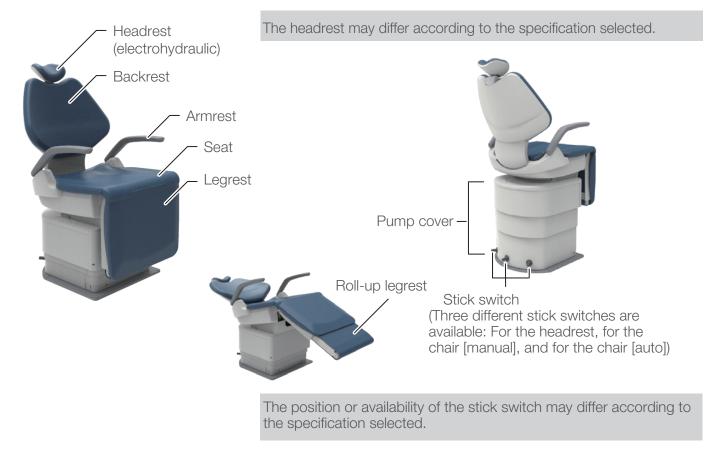

4-3-2 Headrest


Headrest (electrohydraulic)




Headrest (manual)



4-3-3 Foot Controller (wired/wireless)

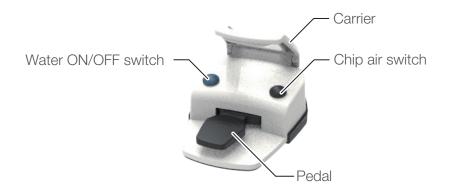

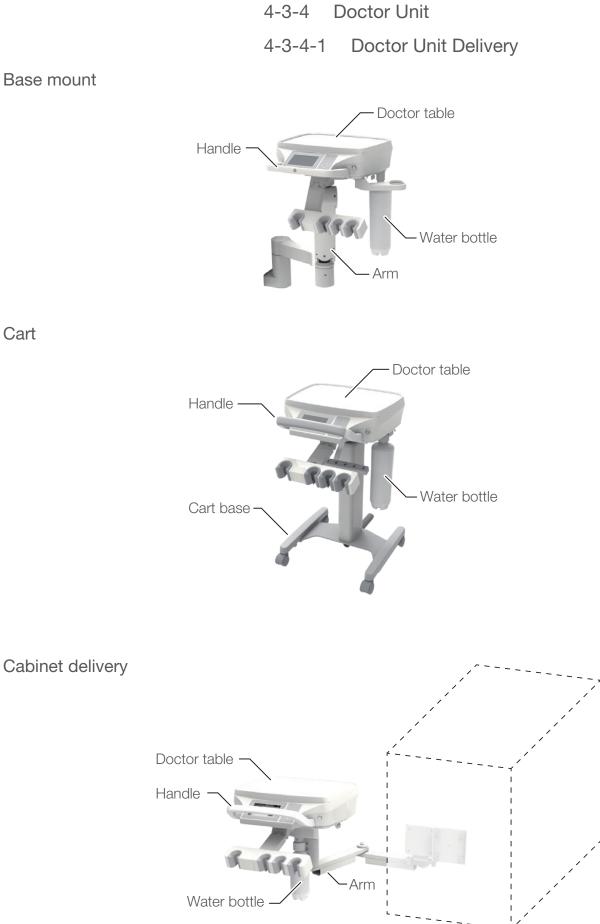


Base mount




Base mount


Holder

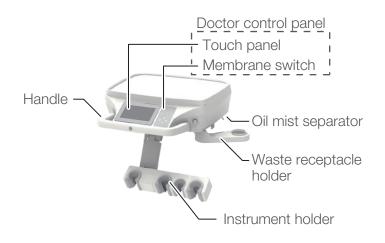


Place holder

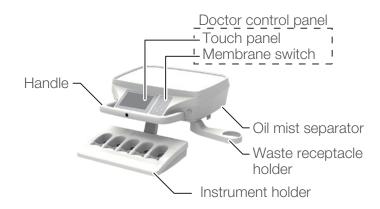



4-3-5 Cuspidor Unit

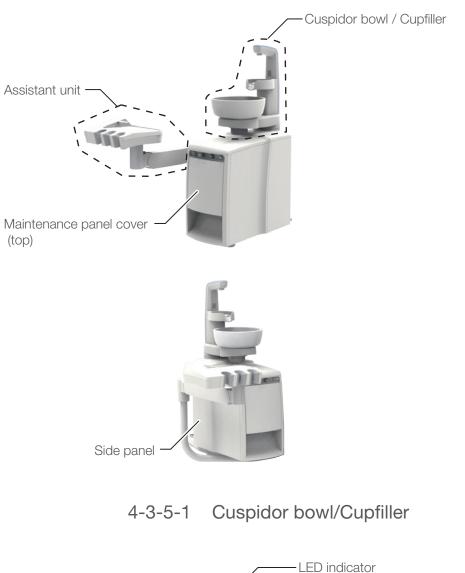

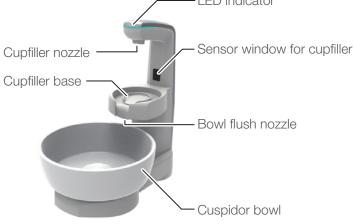



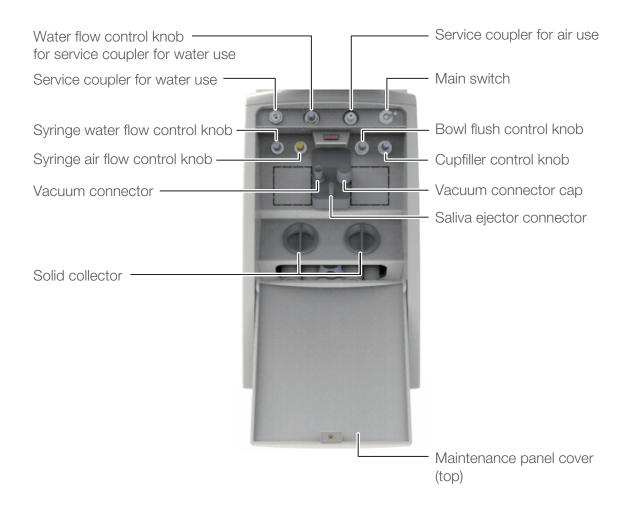

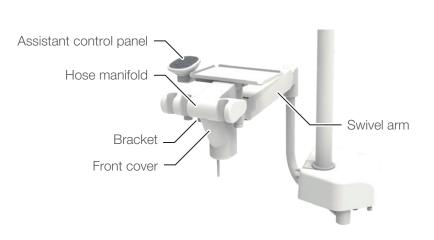


4-3-6

Vac Pac Unit

Refer to the DURR products' instructions for use for operating procedures.

4-3-7 Assistant Unit

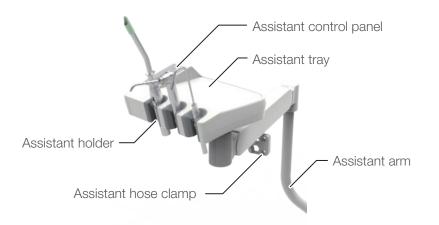

4-3-8 Junction Box






Main water valve knob





The valve is closed.

The valve is open.

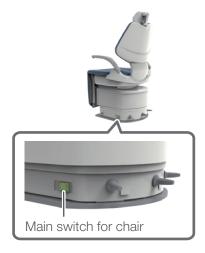

5-1 Preparation Before Use

1 Open the main water valve

Remove the junction cover. Turn the main water valve knob clockwise to lay it on its side.


2 Turn on the power for chair

Press the main switch for chair. * When turns on, it will light up.



Operate the main switch for chair by hand.

Cuspidor

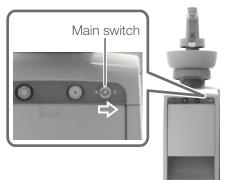

Vac Pac

In the case of a Vac Pac unit, the main switch is located under the doctor table.



3 Switch on EURUS.

Turn the main switch to the right (I)

Nothing should be placed on the cupfiller base when the main switch is turned on. Water may not be supplied normally during cupfiller operation.

When turning on the main switch, sensitivity of touch panel will be initialized. Do not turn on the power while touching the panel by fingers.

Turning on the power while touching the panel may lead to the reduction of the sensitivity of the panel.

If the detection is incorrect, turn off the power and wait for approx. 5 seconds. Then, turn on the power again.




Operate the main switch by hand.

Peel off the film attached to the touch panel before initial use. Using the touch panel with the film attached may lead to incorrect operation, resulting in injury.

4 Confirm the startup screen and flush the retained water

Turn on the main switch. The **[]** mark is displayed on the touch panel, the startup sound is heard, and the program version information is displayed for several seconds.

The program version information differs depending on the month of purchasing (manufacturing) and specifications.

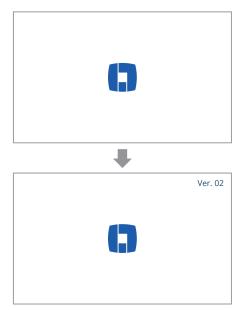

5 Operation

| Check pairing status to FC | | | | |
|--|--|--|--|--|
| Step on the pedal of Foot controller. | | | | |
| *Pressing skip without stepping on the FC will not activate the wireless operation of HP. | | | | |
| • | | | | |
| Flushing, Washing the vacuum line ? | | | | |
| Flushing | | | | |
| Washing the vacuum line | | | | |
| Flushing and washing the vacuum line | | | | |
| *This selection can also be made at of the top menu. | | | | |
| - | | | | |
| Flushing | | | | |
| Select type. | | | | |
| Short mode 00:40 | | | | |
| Long mode 10:00 | | | | |
| ОК | | | | |

Hold down the pedal of the wireless foot controller until the flushing screen appears.

If it does not happened while holding down the pedal of the foot controller, refer to the "Foot controller (wired/wireless)"[pages 129 to 132].

* The wireless pairing screen does not appear if the foot controller is wired.

Press the 'Flushing' switch.

If the washing function for vacuum line is not equipped, the screen shown in the left figure does not appear. The screen titled "Without washing function for vacuum line" will be displayed.

The flushing screen is displayed.

Perform flushing, referring to "Selection and implementation of flushing method" [pages 147 to 155].

* To cancel the process, press 🚮.

For Vac Pac unit type

| Flu | ushing | | 5 |
|--------|------------|-------|---|
| Select | t type. | | |
| | Short mode | 00:40 | |
| | Long mode | 05:00 | |
| | | 0 | |

Without washing function for vacuum line

| Flu | shing | ? | Skip | | |
|---------------------------------------|------------|-------|------|--|--|
| Select type. | | | | | |
| (| Short mode | 00:40 | | | |
| | Long mode | 10:00 | | | |
| *This selection can also be made at O | | | | | |

* For Vac Pac unit type, the screen shown in the left figure appears.

Home screen



Once the flushing process is completed, the home screen is displayed.

For the description of respective switches and indications, see "Doctor's unit operation panel" [pages 77 to 97].

5 Be sure to respond appropriately if an error occurs

Check mode E01:Communication w/cuspidor section E02:Communication w/FC E03: Communication w/water heater (HP) E04:Communication w/micro motor If an error occurs during the startup process, the check mode screen is displayed on the touch panel. See explanation on [pages 218 to 220], and respond appropriately according to the indicated method.

To display the next screen, press the skp switch. If five or more errors occur, press the skp switch to check errors other than the ones indicated on the first page.

6 Perform startup inspection

Confirm that the discharge of retained water has been completed.
 Operate the switches and the foot controller to confirm that the chair and the handpieces/syringes function normally.
 (Connect the handpieces/syringes before performing the operation check.)

7 Prepare for the procedure

①Put an cup on the cupfiller base of the cuspidor unit.②Attach the vacuum/saliva ejector tip to the vacuum/saliva ejector handpiece.



Be sure to perform maintenance inspection before use (startup inspection to confirm that the product functions normally). Operation without performing maintenance inspection may result in injury and damage to peripheral equipment.



5-2 After use

1 Turn off the power for chair When turned off, the light goes off.




Operate the main switch for chair by hand.

2 Power off the unit

Push down the main switch to the left side (\bigcirc).

Cuspidor

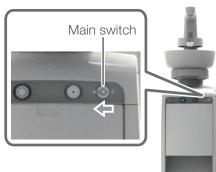



Operate the main switch by hand.

Vac Pac





Main water valve knob


00000

3 Close the water main valve

Turn the main water valve knob counterclockwise until the knob is positioned vertically.



To prevent accidental leakage, be sure to close the main water valve at the end of day.

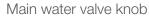

Junction box





The valve is open. The valve is

The valve is closed.

Touch panel



LED indicator



5-3 Operating the main switch for chair during treatment(when the main switch for chair is available)

1 Turn off the power for chair

Press the main switch for chair.

When turning off the chair, "E06: Communication with chair" is displayed on the touch panel because of the interruption of communication between unit and chair, then LED indicator of the cuspidor unit lights up in orange, which is not malfunction.

2 Turn on the power for chair


Press the main switch for chair.

Press the main switch for chair when 5 seconds or more have passed after turning off the main switch for chair.




The position and/or availability of the stick switch may differ according to the specification selected.

Doctor control panel Membrane switch

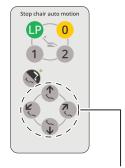


Switches for manual operation

Assistant control panel (Vac Pac Unit type)

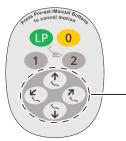

5-4 Raising/lowering the chair and raising/reclining the backrest (manual operation)

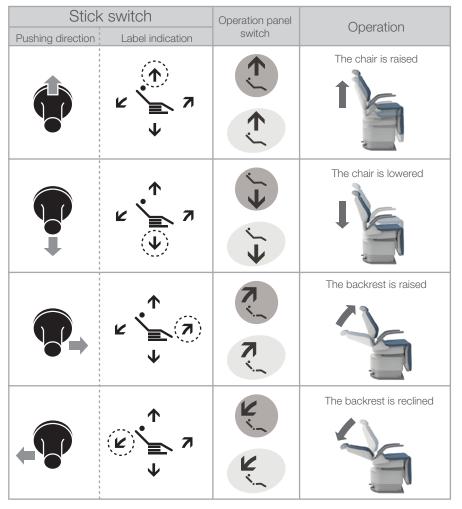

*The chair is operated while the stick switch is pushed down or the switch is pressed.



Before raising/lowering the chair, confirm that no human body part, limb, or obstacle is obstructing the chair.

Before reclining the backrest, confirm that the patient's arm or hand is not placed between the backrest and the seat.

Before raising the backrest, confirm that the patient's arm or hand is not placed between the backrest and the armrest.


[Reference] Precautions when a patient is seated [page 19]



Stick switch for automatic operation

The position and/or availability of the stick switch may differ according to the specification selected.

Doctor control panel Membrane switch

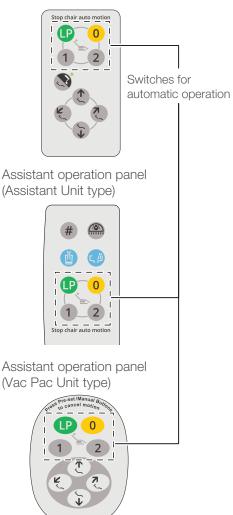

5-5 Moving the chair to the preset position (automatic operation)


| | Stick switch | | Operation | Operation |
|-----|-------------------|------------------|--------------|---|
| | Pushing direction | Label indication | panel switch | |
| I | | 1 2 0 | 1 | The chair is moved to the preset "Treatment position 1". |
| 11 | | 1 2) 0 | 2 | The chair is moved to the preset "Treatment position 2". |
| 111 | | 1 2 0 | 0 | The chair is moved to the preset "Entry/exit position". By pushing again after movement completion, the headrest is housed in the case of headrest (electrohydraulic). |
| IV | | 1 2 0 | LP LP | The chair is moved to the preset "mouth rinsing position". By pushing again after movement completion, the chair is moved to the position prior to the "mouth rinsing position". |

Do not hold down the preset switches 1 or 2, automatic return switch 0, or last position switch P for five seconds or longer. If pressed for five seconds or more, a buzzer sound is heard, and the chair position at that point is memorized as the set position.

[Reference] Automatic operation setting [page 143]



Before moving the chair, confirm that no human body part, limb, or obstacle is obstructing the chair.

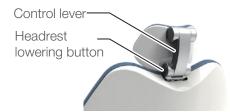

Angle and up/down of headrest



5-6 Operating the headrest (manual type)

Adjustment of the headrest

The headrest (manual type) is capable of flexibly adjusting the inclination of the patient's head for treatment in the upper/lower jaw. It is also possible to manually extend/fold the headrest in accordance with the patient's height.

Angle adjustment of the headrest

1, Insert the operator's hand beneath the headrest to support the patient's head.

2, Press in the control lever with the other hand to make the headrest movable.

3, Adjust the headrest to an angle that matches the required treatment, and release the control lever to lock the headrest.

Up/down adjustment of the headrest

1, To raise the headrest, pull the headrest up slowly while holding it.

2, To lower the headrest, apply the operator's hand to the headrest, and pull it down slowly while pressing the headrest lowering button.

Do not apply excessive load or impact to the headrest while it is extended. Do not hit the headrest from above. Otherwise, failure may result.



If the backrest is to be moved while the headrest is extended, confirm that there is no object obstructing or coming into contact with the chair. Otherwise, damage or injury may result.

When operating the headrest, be careful not to get the operator's hand or finger caught around the control lever or between the head-rest and the backrest.

When adjusting the angle or up/down, be sure to apply the operator's hand to the headrest and support the patient's head. Otherwise, an accident or injury may result.





Switch for manual operation

5-7 Operating the headrest (electrohydraulic type)

For operation with the manual operation switch, press the chair/headrest switch Sto switch over to the headrest function (the light is turned on).

| Stick | < switch | Operation panel switch | |
|-------------------|--|---------------------------|-----------|
| Pushing direction | Label indication | lamp is turned on | Operation |
| ¢ | ∠ × 7 | | Raise |
| | | ₹ | Lower |
| | ⊭ ♪ ⊘ | X X X | Raise |
| | A A | X ×. | Recline |

* The headrest is operated while the stick switch is pushed down or the switch is pressed.

Do not apply excessive load or impact to the headrest while it is extended. Do not hit the headrest from above. Otherwise, failure may result.

WARNING

Before moving the headrest, confirm that the patient's head is placed in the correct position. Operator must not take their eyes off the patient during movement.

Do not use the unit at an angle that causes discomfort to the patient.

When moving the headrest, confirm that there is no object obstructing. Be very careful not to get an object or human body part caught between the headrest and the backrest. [Reference] Precautions when a patient is seated [page 19]

5-8 Turning the armrest

If the armrest hinders entry/exit or treatment, it may be turned backward.

Only the right armrest can be turned.

(1) Pull the root of the armrest (back side) diagonally upward.



(3) To return the armrest to the original position, turn it forward in the same procedure.

The armrest is locked when it comes to the normal position.





When turning the armrest, confirm that there is no obstruction. Operator must be careful not to get their hand or finger caught.


Do not move the chair while the armrest has been turned backward. Return the armrest to the normal position before moving the chair.





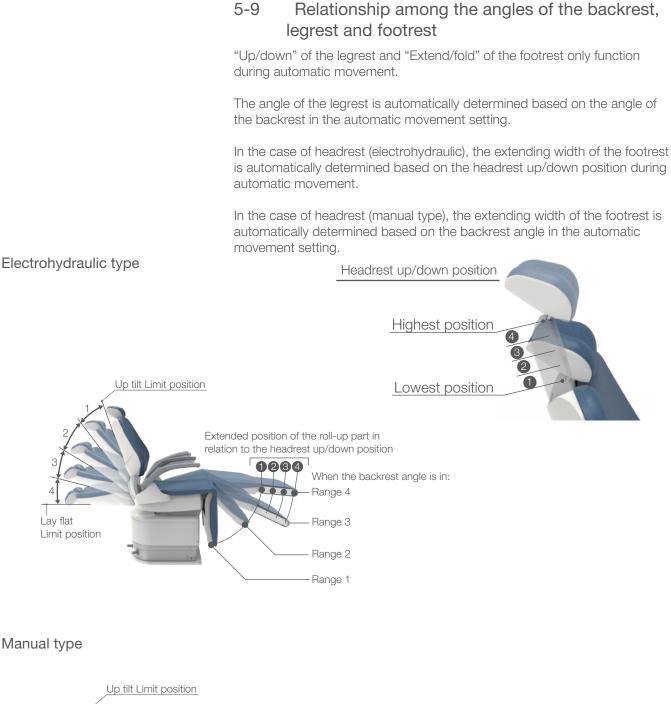

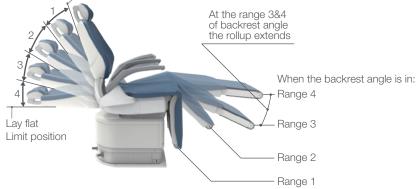


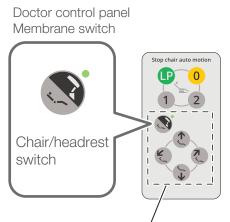

Switch for manual operation

5-10 Operating the legrest

By holding down the chair/headrest switch S for approx. one second or more, the unit is switched into the legrest control mode (the light is flashing), and the legrest becomes controllable.

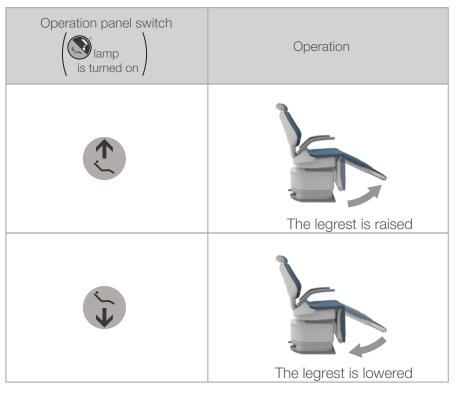

* The legrest can be moved while the switch is pressed down.

Headrest (manual type)

The extending width of the footrest is automatically determined based on the legrest angle.

Headrest (electrohydraulic type)

The extending width of the footrest is determined based on the legrest angle and the headrest height.

To cancel the legrest control mode, press the chair/headrest switch again.

Notification of the legrest control mode

When the chair/headrest switch S is pressed after powering on the unit, a popup screen notifying the legrest control mode is displayed only once.

Press the 🛛 switch to close the popup.


Before raising/lowering the legrest, confirm that no human body part, limb, or obstacle is obstructing the chair.









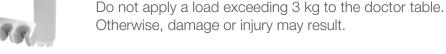

5-11

Base mount

Height adjustment of the doctor table

WARNING

The height of the doctor table can be adjusted in the range as shown in the left figure.

Operating the doctor table

Hold the handle and slowly move the doctor table.

Moving the doctor table forward/backward and right/left

To change the height, release the brake first by pressing the balance arm brake release switch at the center of the handle. Then, move the doctor table upward/downward.

Moving the doctor table upward/downward without releasing the brake may cause damage to the product.

Be sure to release the brake first by pressing the brake release switch. Then, move the doctor table upward/downward.

Cart

The height of the doctor table can be adjusted in the range as shown in the left figure.

To change the height, loosen the lock knob and pull up/down the doctor table.

After changing the height to the desired position, tighten the lock knob securely.

Loosen the lock knob \rightarrow turn it counterclockwise Tighten the lock knob \rightarrow turn it clockwise

Moving the doctor table upward/downward without loosening the lock knob may cause damage to the product. Be sure to loosen the lock knob first, then move the doctor table


upward/downward.







5-12 Control of the assistant tray

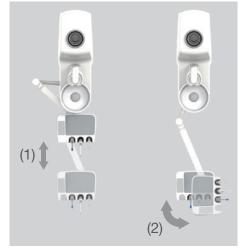

(1) Moving the assistant tray forward/backward Hold the assistant tray and slowly move it forward/backward.

(2) Turning the assistant tray Hold the assistant tray and slowly turn it. It is possible to turn the tray only.

When moving the assistant tray forward/backward, make sure there is nothing on the assistant tray.



Do not apply a load exceeding 1.5 kg to the assistant tray. Otherwise, damage or injury may result.

Height adjustment of the assistant tray

The height of the tray can be adjusted by stopping the arm angle in the range from 0 to 40 degrees (see the figure on the left). To change the height, move the tray while holding the arm part.

1) To raise the height

Hold the tray when moving it upward.

2) To lower the height

First move the arm upward while pressing the unlock button, and subsequently move it downward.

Move the tray to the desired height, and release the unlock button.


When adjusting the height of the assistant tray, make sure there is nothing on the assistant tray.

To lower the height, be sure to press down the unlock button during adjustment.

Lowering the height without pressing the unlock button may cause damage to the product.



Do not apply a load exceeding 1.5 kg to the assistant tray. Otherwise, damage or injury may result.

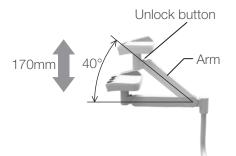



5-13 Handpiece

When a handpiece other than the syringe is picked up from the instrument holder, the operation screen for the handpiece is displayed on the touch panel. Subsequently, operate the foot controller to move the unit.

[Reference] Micromotor

Ultrasonic scaler[pages 12]Air Turbine /motor[pages 12]Vacuum handpiece[pages 12]Saliva ejector handpiece[pages 12]Foot controller (wired/wireless)[pages 12]

[pages 108 to 121] [pages 122 to 124] [pages 125 to 127] [pages 135] [pages 135] [pages 129]

| (Example of display) Micromotor MX2 |
|-------------------------------------|
|-------------------------------------|



| Instrument | Syringe | |
|------------|-----------|--------------------------|
| | Handpiece | Micromotor |
| | | Ultrasonic scaler |
| | | Air turbine |
| | | Air motor |
| | | Vacuum handpiece |
| | | Saliva ejector handpiece |



When the handpiece is returned to the instrument holder, the home screen is displayed.

Handpiece priority function (first priority)

Only the handpiece that is picked up first is operable.

If two or more handpieces are picked up from the instrument holder, and if the handpiece picked up first is returned to the holder, a message [C12: HP first priority] is displayed.


This message disappears if all handpieces are returned to the instrument holder.



5-14 Doctor operation panel

Touch panel (home screen)


The overview of indications and switches on the touch panel is as follows.

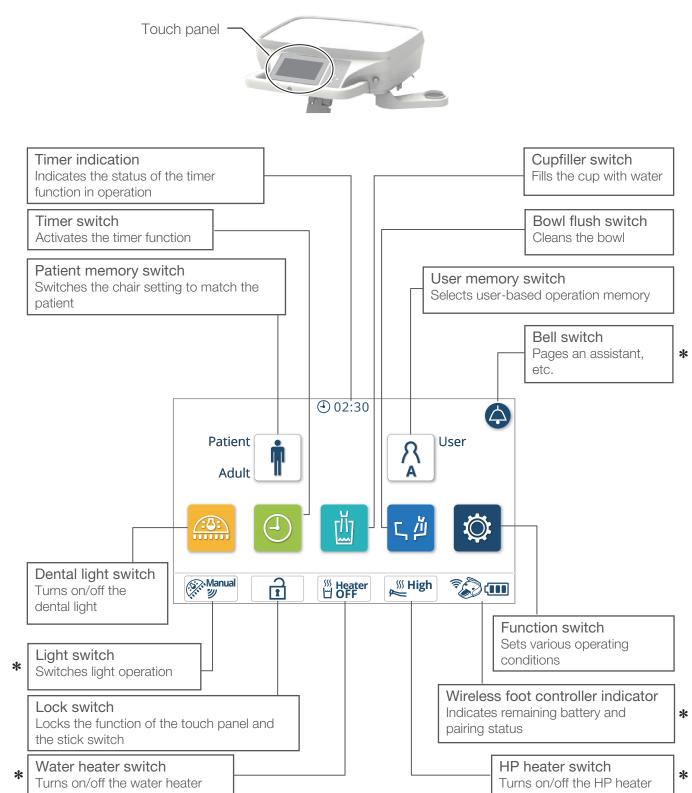

* Not indicated if the function is not installed

Touch panel (handpiece screen)

The overview of indications and switches on the touch panel is as follows.

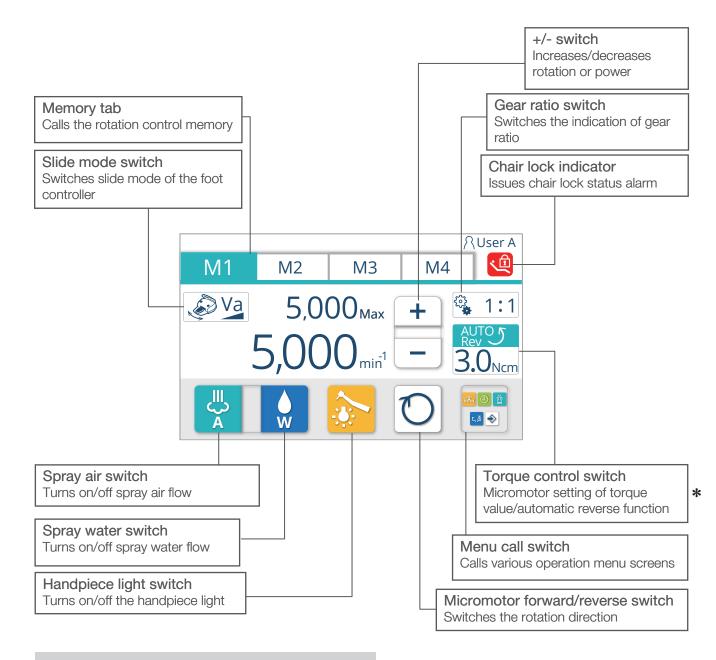


* Not indicated if the function is not installed

Membrane switch

The overview of indications and switches on the operation panel is as follows.



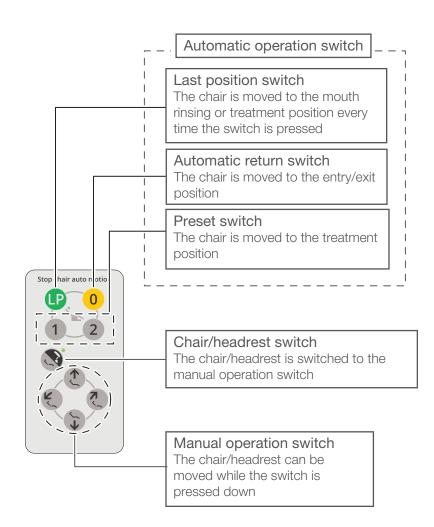





Last position switch

When pressed in the treatment position, the chair is moved to the mouth rinsing position.

When the switch is pressed again, the chair is returned to the treatment position prior to mouth rinsing.

Automatic return switch

The chair is moved to the entry/exit position. * In the case of headrest (electrohydraulic), by pushing this switch again after the chair comes to the complete stop, the headrest is housed.




Preset switch

When 1 is pressed, the chair is moved to treatment position 1.

When **2** is pressed, the chair is moved to treatment position 2.

Do not hold down the preset switch, automatic return switch, or last position switch for five seconds or longer.

If pressed for five seconds or more, a buzzer sound is heard, and the chair position at that point is memorized as the set position. [Reference] Automatic operation setting [page 143]

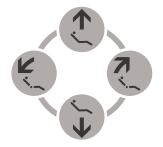

Manual operation switch

Manually operates "Raise/lower" and "Raise/recline" of the chair/headrest.

| Switch | Operation |
|--------|--|
| | The chair is raised / The headrest is raised |
| ₹ ↓ | The chair is lowered / The headrest is lowered |
| ×. | The backrest is reclined / The headrest is reclined |
| R. | The backrest is raised / The headrest is raised |

*The chair/headrest can be moved while the switch is pressed down

Chair/headrest switch

Switches the chair/headrest functions to the manual operation switch. When the headrest function is selected, the lamp is turned on.



When the headrest function is selected, the lamp is turned on



Before moving the chair, confirm that no human body part, limb, or obstacle is obstructing the chair.



Structure of user/patient memories


Up to six user memories (A to F) and up to three patient memories (Adult, Child and Elderly) can be set.



Ň

:Adult mode

A recommended mode for an adult patient of a standard figure; In addition to the automatic return switch (1) and the last position switch (1), the preset switches (1) (2) can be set for up to six user memories (A to F).

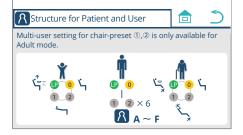

[Child] :Child mode

A recommended mode for a child patient or a patient of a small figure;

The automatic return switch **0**, the last position switch **(P)**, and the preset switches **1 (2)** can be set respectively.

In the case of a pedestal type, the height of the chair at mouth rinsing position can be set.

A treatment position without extending the legrest can be set, and the position around the legrest can be achieved.

[Elderly]

ŕ

:Elderly mode

A recommended mode for an elderly patient or a patient who requires special considerations; The automatic return switch 0, the last position switch 1, and the preset switches 1, and be set respectively.

The angles of the backrest and legrest can be limited, and other settings are possible.




| | | R | User A |
|--|---|---|--------|
| ⚠Selection of user | ? | | 5 |
| Selection of the user changes the set position $(0, 2)$ and handpieces to that | | | |
| A B | (| : | |
| D E | F | · | |
| Name Copy Copy | | C | к |

Calling the Structure of user/patient memories

1.Press the user memory switch Λ . The "Selection of user" screen is displayed.

2.On the "Selection of user" screen, press the ? switch. The "Structure for Patient and User" screen is displayed.

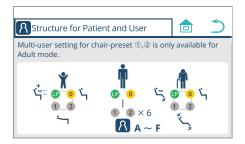

Press the figstheta switch to return from the present screen to the home screen.


Press the \bigcirc switch to return from the present screen to the previous screen.



Patient memory selection switch

The treatment position setting of the chair can be switched in accordance with the patient to be treated.

Every time the switch is pressed, the mode is toggled "Adult" > "Elderly" > "Child".

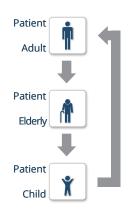




User memory switch

The preset chair position for each user and the initial settings for handpieces upon powering on can be memorized, and selected when necessary.



1. Press the user memory switch Å. The "Selection of user" screen is displayed.

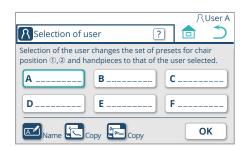

| | | R | User A |
|---|-----|---|--------|
| <u> </u> Selection of user | ? | | D |
| Selection of the user changes the set position \mathbb{O},\mathbb{Q} and handpieces to that | | | |
| A B | | : | |
| D E | - F | : | |
| Mame Сору | | C | ж |



2. On the "Selection of user" screen, press the memory switch for the user to be called for selection.


(In this case, user **B**_____ is selected.)

Press $\fbox{o\kappa}$ to call the settings for the newly selected user , and return to the home screen.

Name registration

A name can be registered for each user memory.

- Patient Adult
- 1. Press the user memory switch 🔝. The "Selection of user" screen is displayed.

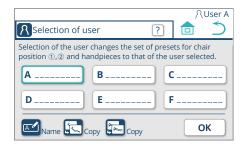

2. On the "Selection of user" screen, press the switch for the user memory for selection and name registration. (In this example, User memory A is selected.)

Press the name registration switch do view the "Registration of name" screen for the selected user .

| Registrat | ion of name | د 🖻 |
|-----------|-------------|-------|
| 123 | Α | ОК |
| QW | ERTYU | I O P |
| AS | DFGH | JKL |
| ZX | C V B N M | . 💌 |


3. Enter the name using the keyboard. Press **ОК** to view the "Selection of user" screen.

User name can be entered in alphabetical or numeric format up to nine digits.

When switching alphanumeric input, press the alphanumeric switch 123.

Press the backspace switch 🔄 to delete the previous (left) character at the cursor position during the character input.

In this example, the name "BELMONT" has been entered. The registered name is indicated for the User memory A box and displayed at the top right.

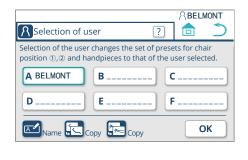

* The registered name is indicated to the right of the user memory switch on the home screen.

Copying the chair preset

The settings for the chair preset switches **1**, **2** in the Adult mode can be copied to another user memory.

| | {\User A | | | |
|--|----------|--|--|--|
| Selection of user ? | | | | |
| Selection of the user changes the set of presets for chair position $(_{0},_{2})$ and handpieces to that of the user selected. | | | | |
| A | c | | | |
| D | F | | | |
| Name Copy | ОК | | | |

1. On the "Selection of user" screen, press the chair copy switch 🔂.

- **R**Copy of chair preset \mathcal{D} Copied from Copied to ·1,2 Α В ОК Select the user copied from. Α В С D Е F
- 2. Select the "Copied from" user from [A] to [F].

The "Copied from" switch indication is changed to show the selected user.

∧User A Copy of chair preset Copied to Copied fro ·1,2 В ок Α Select the user copied to. В С Α D Е F

3. Press the "Copied to" switch and select the "Copied to" user from [A] to [F].

The "Copied to" switch indication is changed to show the selected user.

Press the **OK** switch to view the popup screen for confirmation.

| | R۱ ا | Jser A |
|---|---------|--------|
| Copy of chair preset | | 5 |
| $\begin{array}{c c} & \text{Copied from} & \text{Copied to} \\ \hline & & & \\ & & & & \\ & & & \\ & & & \\ & & & & \\ & & & \\ & & & \\ & & & \\ & & & \\ & & & \\ & & & \\ & & & \\ & & &$ | | |
| Once copied, memory of original settings for the user copied to will be lost. Are you sure to make a copy? | | |
| × | 0 | к |

4. To confirm, press ок.

The copying is completed, and the "Selection of user" screen is displayed.

Press \mathbf{x} to return to the previous screen without copying.

Copying the handpiece setting

The initial setting of a handpiece upon powering on can be copied to another user memory.

| | | R | User A | |
|---|---|---|--------|--|
| RSelection of user | ? | | 5 | |
| Selection of the user changes the set of presets for chair position $(), ()$ and handpieces to that of the user selected. | | | | |
| A B | С | | | |
| D E | F | | | |
| | | 0 | к | |

1. On the "Selection of user" screen, press the "Copy of handpiece setting" switch 🔙 .

- R Copy of handpiece setting C Copied from Copied to A 🔿 В ОК Select the user copied from. Α В С D Е F
- 2. Select the "Copied from" user from [A] to [F].

The "Copied from" switch indication is changed to show the selected user.

| | | RUser A |
|----------------------|---------------|---------|
| R Copy of handp | piece setting | |
| Copied from | Copied to | |
| <i>P</i> A | ⇒ <u>B</u> | ОК |
| Select the user copi | ed to. | |
| Α | В | С |
| D | E | F |

3. Press the "Copied to" switch and select the "Copied to" user from [A] to [F].

The "Copied to" switch indication is changed to show the selected user.

Press **OK** to view the popup screen for confirmation.

| RCopy of handpiece setting | RUser A |
|--|---------|
| Copied from Copied to | |
| Once copied, memory of original settings for the user copied to will be lost. Are you sure to make a copy? | |
| × | ОК |

4. To confirm, press ок .

The copying is completed, and the "Selection of user" screen is displayed.

Press \mathbf{x} to return to the previous screen without copying.



Dental light switch

Turns on/off the dental light; On/off is toggled;

- * Pressing and holding the switch while the light is on for 2 seconds or more changes the mode into Composite Safe.
- * The ceiling light and the track light are not operable.



Timer switch For the use and setting of the timer



<How to use the timer>
Time for the timer can be selected from four memories. **1. Press the timer switch** The "Timer" screen is displayed.

| Time | r | | 5 |
|------|---|-------|---|
| | Α | 01:00 | |
| | В | 02:00 | |
| | c | 03:00 | |
| | D | 05:00 | |

2. Select a memory (in this example, "Timer D: 5 minutes" is selected)

The operation/setting screen for Timer D is displayed.

Press the home switch finise to cancel the operation and return to the home screen.


To return to the previous operation screen, use the return switch).



3. Change the timer setting

 \sim : decrease the set time

Press the store switch $\textcircled{\begin{subarray}{c} \bullet \end{subarray}}$ to save the changed setting into the memory.

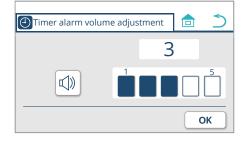

4. Change the alarm volume for the timer

When the alarm volume adjustment switch is pressed, the "Timer alarm volume adjustment" screen is displayed.

Press the touch bar graph _____ to adjust the alarm volume in five levels.

Press the alarm sound switch 1 to mute or de-mute the alarm. Once adjustment is completed, press the **ok** switch to return to the previous screen.

O 5:00
 O³
 Control
 Control
 Control
 Control
 Control
 Control
 Contr

5. Press the start **>** to start the countdown of the timer The subtraction timer is started.


Timer D

 02:30

Save

Press the pause III to pause the timer operation. To restart the timer, press the start III.

Press the stop • to cancel the timer operation, and return to the screen in 3.

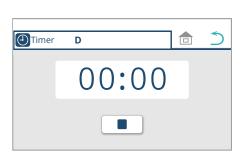

6. Once the timer countdown is completed, an alarm sounds Alarm sounds for the specified time, then it stops automatically.

Press the stop • to stop the alarm sound, and return to the home screen.

* If the timer operation is performed on the handpiece screen while the handpiece is picked up, the display returns to the relevant handpiece screen.

5 Operation



М3

min⁻¹

М3

M4

M4

+

+

🎭 1:1

유User A

🗣 1:1

High 🔊 🔊 💷

M2

25.000

M2

40,000_{Max}

25,000min¹

40.000_{Max}

M1

🔊 Va

M1

Switch

Ē

7. Returning to the home screen during timer operation Press the home switch a while the timer is operating to return to the home screen. The timer operation status is displayed at the top of the screen.

When the timer countdown is completed on the home screen, the screen in 6 is displayed.

* With the handpiece being picked up, even if the timer switch is

pressed after pressing the menu call switch $\begin{bmatrix} 1000\\ 000 \end{bmatrix}$, the timer will not

start, and the display returns to the handpiece screen.


Once the countdown of the timer is completed with the handpiece being picked up, timer switch (2) appears on the screen. If the timer switch (2) is pressed, handpiece display appears on the screen.

If the handpiece is returned without pressing the timer switch 0, the display shown in 6 appears.



8. Changing the mode into timer display

Press the timer switch 0 during the countdown process to show the timer display on the screen as shown in 5.

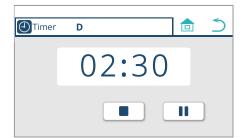

Heater



Cupfiller switch

This is a switch for filling the cup, installed separately from the sensor cupfiller.

When this switch is pressed, water is released from the cupfiller nozzle. Water is also released from the bowl flush nozzle to flush the cuspidor bowl.

Cupfiller operates by timer for a specified time period.

To stop cupfiller before the specified time is elapsed, press this switch again.

Before using cupfiller, put a cup on the cupfiller base. Otherwise, the wall and the chair may be spattered with water. Cupfiller operates for a specified time period, regardless of the water level in the cup. Be careful not to overflow the cup.



Bowl flush switch

Water is released from the bowl flush nozzle to flush the cuspidor bowl (approx. six seconds).

Hold down the switch for two seconds or longer for continuous flushing. Press this switch again to stop flushing.



Function switch For setting various operating conditions [pages 145-186]

5 Operation



| Sensor / manual mode | |
|-------------------------|-----------------------|
| Manual Manual Sensor | ON Keep light ON |
| Manual Manual Manual | OFF Keep light OFF |
| | ОК |
| | Heater High |

Light switch


Call the "Sensor/Manual switch mode" screen to switch the operation mode for the dental light.

Select an operation mode and press **OK** to change the setting. Press **X** to cancel the change and exit.

| Indication | Mode | Condition |
|---------------------------|------------------------|--|
| Manual Manual &Sensor | Sensor/ switch mode | The light can be turned on/off with the touchless switch of the dental light and with the dental light switch . |
| Manual Manual Manual only | Switch mode | The light can be turned on/off with the dental light switch |
| ON Keep light | ON mode | The dental light is continuously turned on at the maximum illuminance. * The light cannot be turned on/off with the touchless switch of the dental light and with the dental light switch |
| OFF Keep light | OFF mode | The dental light is continuously turned off * The light cannot be turned on/off with the touchless switch of the dental light and with the dental light switch 🙈 . |
| (No indication) | Not installed | The dental light is not installed in the product. |

5 Operation



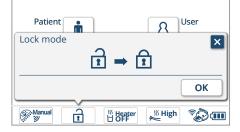

Lock switch

To clean the operation panel or to prevent mischief by child patients, the touch panel, membrane switch or the stick can be locked.

Call the "Lock mode" screen and press **OK** to lock the operation.

During locking, the locking message screen is displayed on the touch panel.

To cancel the operation lock, press and hold the chair/headrest switch reprox. one second or more.





Water heater switch

Call the "Water heater (Cup)" screen to set cupfiller water to warm water.

Select the heater mode (ON/OFF) and press **OK** to change the setting.

Press \mathbf{X} to cancel the change and exit.





Patient SUser Water heater (Handpiece)

HP heater switch

Call the "Water heater (Handpiece)" screen to set water released from the handpiece to warm water, and to set the temperature of water (two temperatures).

Select the heater mode (High, Low, OFF) and press $\circ\kappa$ to change the setting.

Press \mathbf{X} to cancel the change and exit.



Wireless foot controller indication

Indicates the remaining battery and pairing status of the wireless foot controller.

As the battery level declines, the indication changes from 1 to 1 After a warning popup screen is displayed, the indication is changed to 1.

Once the popup is displayed, promptly replace the battery with a new one. [Reference] How to replace batteries [page 131]

Press \mathbf{X} to close the popup.

If the wireless foot controller has not been paired,



is displayed, and the foot controller is not usable.


If it is not usable, turn off the main switch and reboot the unit, then perform pairing the wireless foot controller.



Bell switch

An assistant, etc. can be paged by using an externally connected device.

Air ON / Water ON (Spray ON)

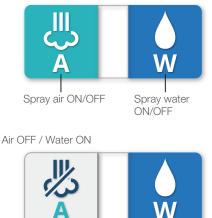

Air ON / Water OFF



Air OFF / Water OFF (Spray OFF)



Light on



Light off



Spray switch

Turns ON/OFF handpiece spray (water and air). Pick up a handpiece and press the switch to toggle between ON/OFF of A (Air) and W (Water).

In the case of ultrasonic scaler, the W (Water) is turned on regardless of the spray mode.

OFF cannot be selected.

In the case of micromotor / air turbine / air motor, two modes or four modes can be selected depending on the spray mode setting. When set to two modes, Spray ON/OFF is toggled. When set to four modes, ON/OFF can be respectively toggled for Air and Water.

[Reference] Spray mode setting [page 174]

Handpiece light switch

Turns on/off the handpiece light. The light is toggled between on and off.

[Reference] Setting the light activation timing [page 175]

Forward rotation (clockwise)



Reverse rotation (counterclockwise)



Micromotor forward/reverse switch

Switches the rotation direction of the micromotor.

Forward rotation (clockwise) and reverse rotation (counterclockwise) is toggled.

Rotation is set to forward immediately after powering on.

* The rotation direction cannot be switched while the micromotor is being rotated.







M1 M2 M3 M4 M1 M2 M3 M4 M1 M2 M3 M4 M1 M2 M3 M4

Menu call switch

Calls menu screens for performing the following operations while a handpiece is picked up.

Dental light on/off [Reference] Dental light switch [page 88]

Use and setting of timer [Reference] Timer switch [pages 88-90]

Cupfiller [Reference] Cupfiller switch [page 91]

Cuspidor bowl flush [Reference] Bowl flush switch [page 91]

Memory of handpiece operating condition setting [Reference] Store switch [page 96]



Store switch

This switch is displayed when the rotation, spray condition and other settings have been changed from the memory on the handpiece screen. Press the switch to overwrite the called memory (M1 to M4) with the changed setting.



Slide mode switch

Switches the slide mode (Va/Fix) of the foot controller. [Reference] Micromotor [page 108]

5 Operation





+/- switch

In the case of micromotor, the maximum rotation value is increased/decreased. [Reference] Micromotor [page 109]


Gear ratio switch

The indication can be switched to rotation that matches the contra angle gear ratio of the micromotor.

[Reference] Micromotor [page 111]

Automatic reverse function and Torque value 3.5 Ncm



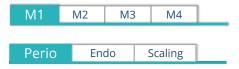



Torque control switch

The condition of torque value and automatic reverse function can be set in the case of micromotor that accepts setting in low-speed rotation range.


[Reference] Micromotor [page 113,114,116 to 119]

Memory tab

The preset memories of handpiece settings can be called and switched.

[References] Micromotor [page 108]

Ultrasonic scaler [page 122] Turbine [page 125]

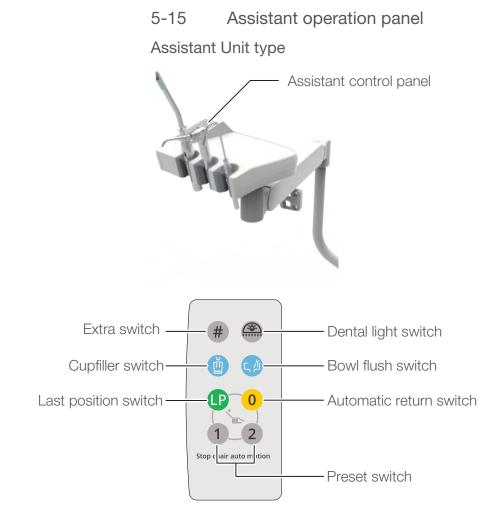



Dental light switch

Turns on/off the dental light; On/off is toggled;

- * Pressing and holding the switch while the light is on for 2 seconds or more changes the mode into Composite Safe.
- * The ceiling light and the track light are not operable.



Extra switch

When this switch is held down for two seconds or longer, the the touchless switch of the dental light is invalidated.

The LED indicator in the cuspidor unit is turned on in pink.

When this switch is held down for two seconds or longer, the dental light sensor is validated.

The LED indicator in the cuspidor unit is returned to the original color.



Cupfiller switch

This is a switch for filling the cup, installed separately from the sensor cupfiller.

When this switch is pressed, water is released from the cupfiller nozzle. Water is also released from the bowl flush nozzle to flush the cuspidor bowl.

Cupfiller operates by timer for a specified time period.

To stop cupfiller before the specified time is elapsed, press this switch again.

Before using cupfiller, put a cup on the cupfiller base. Otherwise, the wall and the chair may be spattered with water. Cupfiller operates for a specified time period, regardless of water level

in the cup. Be careful not to overflow the cup.

Bowl flush switch

Water is released from the bowl flush nozzle to flush the cuspidor bowl (approx. six seconds).

- Hold down the switch for two seconds or longer for continuous flushing.

- Press this switch again to stop flushing.

Last position switch

The chair is moved to the gargle position when this switch is pressed in the procedure position.

Press it again to return the chair to the procedure position prior to gargling.

Automatic return switch

The chair is moved to the entry/exit position.

* In the case of headrest (electrohydraulic), by pushing this switch again after the chair comes to a complete stop, the headrest is housed.

Preset switch

When 1 is pressed, the chair is moved to treatment position 1. When 2 is pressed, the chair is moved to treatment position 2.


Do not hold down the preset switch, automatic return switch, or last position switch for five seconds or longer. If pressed for five seconds or more, a buzzer sound is heard, and the chair position at that point is memorized as the set position.

[Reference] Automatic operation setting [page 143]



Before moving the chair, confirm that no human body part, limb, or obstacle is obstructing the chair.



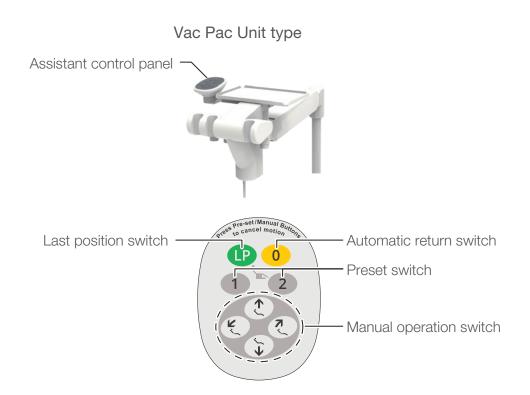





The chair is moved to the gargle position when this switch is pressed in the procedure position.

Press it again to return the chair to the procedure position prior to gargling.

Automatic return switch


The chair is moved to the entry/exit position.

* In the case of headrest (electrohydraulic), by pushing this switch again after the chair comes to a complete stop, the headrest is housed.

Preset switch


When 1 is pressed, the chair is moved to treatment position 1. When 2 is pressed, the chair is moved to treatment position 2.

Do not hold down the preset switch, automatic return switch, or last position switch for five seconds or longer. If pressed for five seconds or more, a buzzer sound is heard, and the chair position at that point is memorized as the set position.

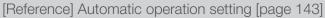



Before moving the chair, confirm that no human body part, limb, or obstacle is obstructing the chair.

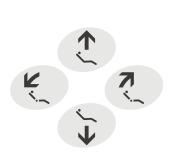

Manual operation switch

Manually operates "Raise/lower" and "Raise/recline" of the chair.

| Switch | Operation |
|-------------|--------------------------|
| ^ | The chair is raised |
| ۴ ۲° | The chair is lowered |
| 2 | The backrest is reclined |
| 7.5 | The backrest is raised |

*The chair/headrest can be moved while the switch is pressed down



Before moving the chair, confirm that no human body part, limb, or obstacle is obstructing the chair.



Stick switch



Assistant operation panel (Assistant Unit type)



Foot controller Step on the pedal.

5-16

operations.

Stick switch

Assistant operation panel Press one of the switches in the [__] frame.

Cancellation function

Operate one of the stick switches (indicated with O).

To stop the chair movement started by a switch for automatic operation (preset switch, automatic return switch, last position switch, or stick switch for automatic operation), perform one of the following

Assistant operation panel (Vac Pac Unit type)



Doctor control panel Membrane switch



Press one of the switches.

Membrane switch Press one of the switches.

5-17 Lock function

5-17-1 Chair lock function

When the lock function is activated, indications as listed below are displayed on the touch panel. (When force is applied to the rear link cover, the lock function is activated.)

| Home screen : | Interference with rear link cover |
|--------------------|-----------------------------------|
| | |
| | |
| | |
| | |
| | |
| | |
| | |
| Handpiece screen : | € ≜ |
| | |
| | |
| | |

When the lock function is activated, the chair movement is stopped, and a popup is opened to notify the abnormality. Check the popup and promptly eliminate the abnormality.

Eliminate the abnormality or press \mathbf{X} to close the popup.

The functions that become locked and the unlocking steps are as follows.

[Reference] [pages 104 to 106]

The LED indicator in the cuspidor unit is turned on in orange.

| Home screen | |
|-----------------------------------|--|
| Interference with rear link cover | |

"||

Ē

Patient

Adult

Manual



٢Ň

Heater

User

 \odot


°)) 💷


Å

ら 凶

∭ High

| Condition 1 | Touch panel indication |
|--|--|
| If force is applied to the footrest in the direction indicated with an arrow during the automatic movement or manual lowering of the chair, the lock function is activated, and the chair is raised for approx. one second and stopped. * The lock function is activated when the legrest angle is less than 45 degrees. | Remove the obstacle . |
| | How to unlock |
| | Remove the obstacle that has caused emergency stop, confirm safety, and operate the chair. |
| | Locked functions |
| | Automatic operations and manual operations of the chair (lowering/raising) |
| Condition 2 | Touch panel indication |
| When the foot controller pedal is stepped on during the automatic operation of the chair | Foot controller Step off the pedal. |
| | How to unlock |
| | Step off the pedal, check safety, and operate the chair. |
| | Locked functions |
| | All chair operations |

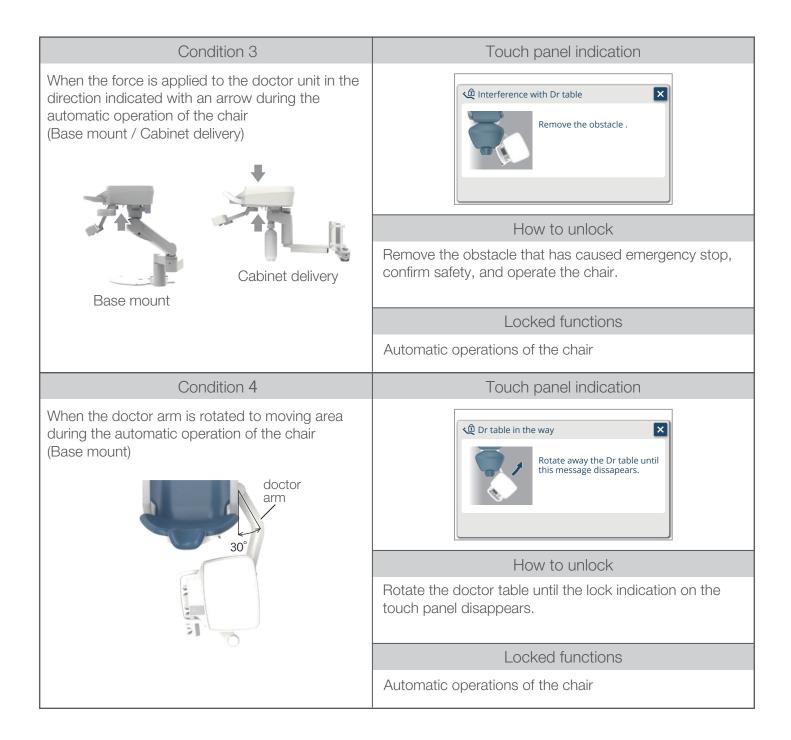

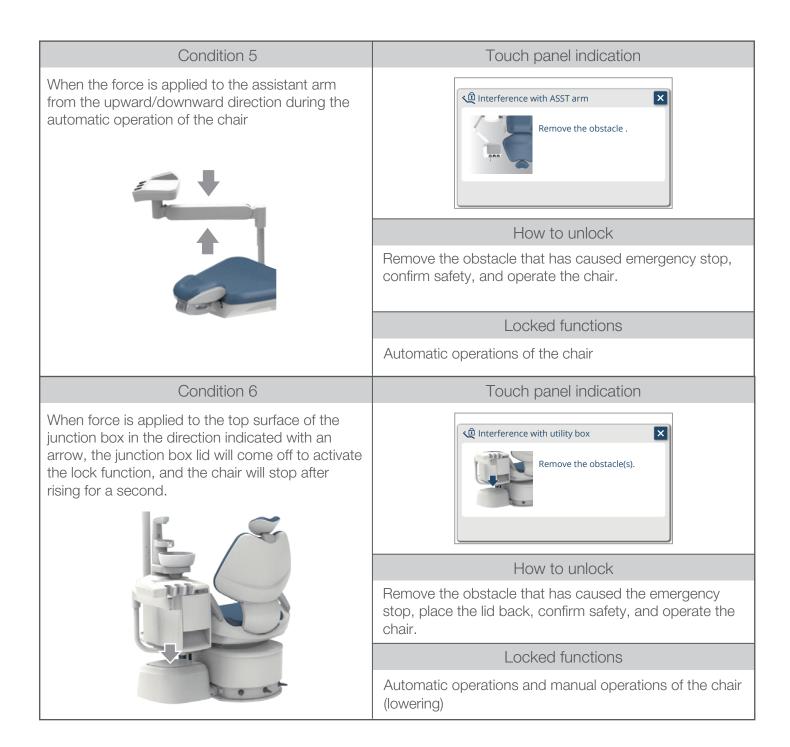

5-17-2 Handpiece lock function

When the lock function is activated, indications as listed below are displayed on the touch panel, and the handpiece cannot be used. The functions that become locked and the unlocking steps are as follows.

| Condition | Touch panel indication |
|--|--|
| A handpiece is picked up from the instrument holder while the foot controller pedal is stepped on. | E20:HP picked up while FC is on Patient Adult Adult User Adult C Manual |
| | How to unlock |
| | Step off the foot controller pedal, and step again on it. |
| | Locked functions |
| | Handpiece operations |

[Example of MX2]

Slide mode (Va/Fix)



5-18 Micromotor Setting the micromotor slide mode (NBX/NLX plus/NLX nano/MX2/MCX)

When the micromotor is picked up from the instrument holder, the micromotor screen is displayed.

Memory can be selected from four memories (M1 to M4). The memories can be toggled by pressing the memory tab.

The slide mode can be selected from Va and Fix.

Va (Variable)

By setting the maximum rotation, and sliding the foot controller pedal to the left or right, the rotation can be adjusted within the range of the maximum rotation.

The rotation is decreased by sliding to the left, and is increased by sliding to the right.

The rotation can be adjusted to the maximum rotation by stepping on the pedal.

Fix (Fixed)

The rotation value can be adjusted up to the setting by the degree of step-in, regardless of the foot controller pedal position.



1. Pick up the micromotor from the instrument holder.

2. Press the slide mode *y* to display the slide mode setting screen.





- 3. Select a slide mode.
- 4. Memorize the selection by pressing οκ

The micromotor screen is displayed.

Settings of the slide mode are memorized in the starting condition after powering on.

* To cancel the setting change, press 🛛 without pressing ок.

Setting the maximum rotation (NBX/NLX plus/NLX nano/MX2/MCX)

[Example of MX2]




Torque value

Automatic reverse function/ Automatic forward function 1. Pick up the micromotor from the instrument holder

2. Press the 🛨 🖃 switch to set the maximum rotation The setting range of the maximum rotation varies depending on the model of the micromotor (see the figure below).

Automatic reverse function / automatic forward function and torque value are only displayed when setting the low-speed rotation range (NLX plus/MX2).

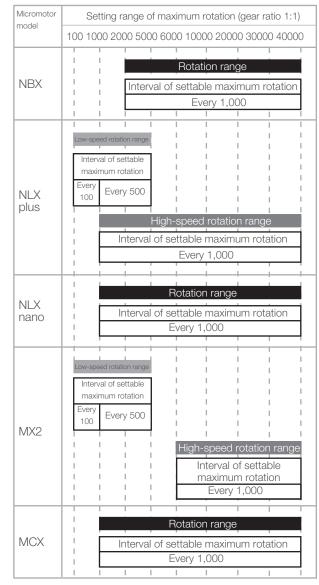

(Unit: min⁻¹)

[Example of MX2]



Memory setting for starting condition after powering on (NBX/NLX plus/NLX nano/MX2/MCX)

When the following setting of handpiece condition is changed, it can be memorized in the steps specified below.

- Maximum rotation

[Reference] Setting the maximum rotation [page 109]

- Spray condition (ON/OFF/Water/Air) [Reference] Spray switch [page 95]

- Handpiece light condition (ON/OFF) [Reference] Handpiece light switch [page 95]

- 1. Pick up the micromotor from the instrument holder
- 2. Select a memory tab M1 to M4 and press it

3. Change the handpiece condition (maximum rotation / spray / handpiece light condition).

4. Press the menu call switch [■]
5. Press the store switch

The memory setting screen for starting condition is displayed.



6. Confirm the displayed setting, and press **Οκ** to memorize it.

* If the \bigcirc K is not pressed but \boxtimes is pressed, the changed setting is valid until the main switch is turned off, but the setting returns to before the change once the main switch is turned off.

| | | | | \User A |
|-----------|--------|---------------------------------------|----|---------|
| M1 | M2 | M3 | M4 |] |
| Default m | iemory | | | × |
| 1 | | ast HP set-u OFF, light O r-on. | | |
| | | | | ок |

[Example of MX2]

| Memor | y tab | | | ear ratio |
|--------|---------------|------------------------------|----|-----------|
| M1 | M2 | M3 | M4 | (User A |
| 🔊 Va | 40,0 | ООмах | + | 1:1 |
| 2 | 5,00 | 0 _{min¹} | - | |
| ↓ A | ♦ W | | | 20 ↓ ◆ |

Gear conversion display function (NBX/NLX plus/NLX nano/MX2/MCX)

This function is for indicating a value close to the actual rotation by converting the ratio of contra angle handpiece/ straight handpiece attached to the micromotor.

The following four kinds of gear ratio are used for conversion.

1:5 (five-fold speed), 1:1 (same speed),

4:1 (decelerated to one-quarter speed),

16:1 (decelerated to one-sixteenth speed)

- 1. Pick up the micromotor from the instrument holder.
- 2. Select a memory from the tab switch M1 to M4 and press it.

3. Press the gear ratio switch setting screen.

4. Select a gear ratio.

5. Memorize the selection by pressing \bigcirc .

The micromotor screen is displayed.

Settings of gear ratio are memorized in the starting condition after powering on.

* To cancel the setting change, press 🛛 without pressing ок





Displayed when the overheating prevention function is activated



Overheating prevention function (NBX/NLX plus/MX2/MCX)

If the product is used for a long time at a high load, the overheating prevention function is activated and the torque is automatically decreased, in order to prevent the micromotor unit overheating. If this occurs, a buzzer sound is heard, and 🛃 is displayed. While this indication is displayed, the overheating prevention function is activated.


By stopping rotation or using the product at a lower load, the overheating prevention function is deactivated after a certain time has elapsed.



Do not turn off and on again the main switch to deactivate the overheating prevention function. This may cause burn out due to the overheating of the micromotor unit.



For proper use, be sure to carefully read the instructions for use attached to the relevant micromotor in advance.

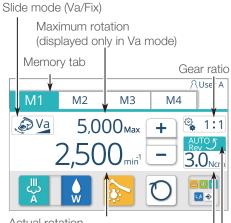

-Actual rotation

(when the foot controller is on) - Rotation in the present foot controller slide position

(when the foot controller is off)

Torque value

Automatic reverse function/ Automatic forward function







NLX plus

In the case of micromotor NLX plus, if the maximum rotation is set in the low-speed rotation range (100 to 5,000 min-1), the torque value, the applicability of automatic reverse function, and the applicability of automatic forward function can be set.

Automatic reverse is a function that automatically switches to reverse rotation (counterclockwise) when the load exceeds the preset torque value in the low-speed rotation range (0.3-3.0 Ncm).

Automatic forward is a function that automatically switches to reverse rotation (counterclockwise) when the load exceeds the preset torque value in the low-speed rotation range (0.3-3.0 Ncm), and then returns again to forward rotation (clockwise) when the load disappears.

Setting of a torque value

- 1. Pick up the micromotor from the instrument holder.
- 2. Select a memory from the tab switch M1 to M4 and press it.
- 3. Press the torque control switch $\frac{30}{3.0}$ to display the torque setting screen.

4. Press the + - switch to set to the required torque value.

<Value can be selected from:>

0.3/0.4/0.5/0.6/0.7/0.9/1.0/1.2/1.4/1.8/2.1/2.3/2.7/3.0

When + is pressed, the torque value toggled upward among the options above.


When — is pressed, the torque value is toggled downward among the options above.

5. Memorize the selection by pressing ок.

The micromotor screen is displayed.

Settings of torque value are memorized in the starting condition after powering on.

* To cancel the setting change, press 🛛 without pressing 🔼 .

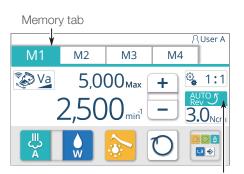

Automatic reverse function/ Automatic forward function

 Auto reverse
 Torque
 RUser A

 Rev
 3.0 Ncm

 Rev
 +

 AUTOS

 AUTOS

 OK
 0



Setting of automatic reverse function / automatic forward function

1. Pick up the micromotor from the instrument holder

2. Select a memory from the tab switch M1 to M4 and press it

3. Press the torque control switch 3.0 to display the automatic reverse function / automatic forward function setting screen

4. Select the required automatic reverse function / automatic forward function setting

- : Set to "No automatic reverse function"
- ₩IJ: Set to "Automatic reverse function"
- : Set to "Automatic forward function"

5. Memorize the selection by pressing $o\kappa$


The micromotor screen is displayed.

Settings of automatic reverse function and automatic forward function are memorized in the starting condition after powering on.

* To cancel the setting change, press 🛛 without pressing 🔼 ок



If a reverse screw-shaped file, etc. is attached to NLX plus, be sure to set to "No automatic reverse function", and switch the rotation direction using the forward/reverse switch on the micromotor. If that unit is set to "Automatic reverse function / automatic forward function", the file may get damaged, resulting in injury.

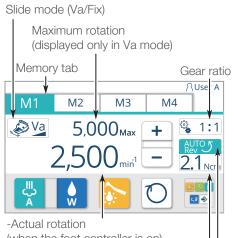

(when the foot controller is on) - Rotation in the present foot controller slide position

(when the foot controller is off)

Torque value

Automatic reverse function/ Automatic forward function

| | | | | ဂ User A |
|-----|--|----------|--------|-----------------|
| M1 | M2 | М3 | M4 | |
| Fix | | | | |
| | th files of Wave , or Bien · Air CA | | 00955) | Reciprocal mode |
| | × | * | Ø | |

MX2

In the case of micromotor MX2, if the maximum rotation is set in the low-speed rotation range (100 to 5000 min-1), the torque value, the applicability of automatic reverse function, the applicability of automatic forward function, and the time of automatic forward function duration can be set. The applicability of reciprocal mode can be set.

Automatic reverse is a function that automatically switches to reverse rotation (counterclockwise) when the load exceeds the preset torque value in the low-speed rotation range (0.4-3.5 Ncm).

Automatic forward is a function that automatically switches to reverse rotation (counterclockwise) when the load exceeds the preset torque value in the low-speed rotation range (0.4-3.5 Ncm), and then returns again to forward rotation (clockwise) after the specified time of duration (automatic forward duration).

In reciprocal mode, the micromotor oscillates at preset angle, acceleration speed, and torque value. This mode allows start or stop operation only.

*Rotation speed cannot be adjusted.

When stepping on the foot switch, Fix is displayed as shown in the left figure. When not stepping on the foot switch, Fix is displayed.



To use the reciprocal mode, designated contra-angle and files are necessary.

Designated contra-angle is shown below: CA ENDO (REF 1600955) by Bien Air

Designated files are shown below: WaveOne[®] Gold by Dentsply Maillefer

RECIPROC[®] by VDW

Auto reverse

AUTO B

AUTOD

Reciprocal mode

| | | | ٩ | User A |
|-------------|------|-------|----------|------------------|
| M1 | M2 | М3 | M4 | |
| 🔊 Va | 5,0 | ООмах | + | 1:1 |
| | 2,50 | | <u> </u> | 1 _{Ncm} |
| U A A | w | | 0 | 0 |

Torque 🗙

2.1_{Ncm}

╋

οк

AUTO 5

2.1_{Ncm}

0

5.¢ 🔶

Setting of a torque value

- 1. Pick up the micromotor from the instrument holder
- 2. Select a memory from the tab switch M1 to M4 and press it
- 3. Press the torque control switch 21 to display the torque setting screen

4. Press the + - switch to set to the required torque value <Value can be selected from:> ဂ User A 0.3/0.4/0.5/0.6/0.7/0.9/1.0/1.2/1.4/1.8/2.1/2.3/2.7/3.0 🔹 1:1

When the + switch is pressed, the torque value toggled upward among the options above.

When the ______ switch is pressed, the torque value is toggled downward among the options above.

5. Memorize the selection by pressing ок

The micromotor screen is displayed.

* To cancel the setting change, press 🛛 without pressing 🔽 ок

| | | | | ဂိUser A |
|-------------|---------------|-------|----------|---------------------------|
| M1 | M2 | М3 | M4 | |
| Va Va | $\sim - \sim$ | ООмах | + | 1:1 |
| | 2,50 | | <u> </u> |).7 _{Ncm} |
| U A A | w | | O | |



Automatic reverse function/ Automatic forward function

RUser A

🎕 1:1

AUTO52

2.1_{Ncm}

a 0 ŭ ⊲ ⇒

M4

+

| Auto reverse | Torque X AUser A |
|-----------------|--------------------|
| AUTO Rev 2 | 2.1 _{Ncm} |
| AUTO 5 Rev | + AUTO 5 |
| AUTOD Fwd | - 2.1 Ncm |
| Reciprocal mode | OK OK |

М3

5.000_{Max}

M1

🔊 Va

M2

2.500

Setting of automatic reverse function / automatic forward function

1. Pick up the micromotor from the instrument holder

2. Select a memory from the tab switch M1 to M4 and press it

3. Press the torque control switch 2.1 to display the automatic reverse function / automatic forward function setting screen

4. Select the required automatic reverse function / automatic forward function setting

- : Set to "No automatic reverse function"
- : Set to "Automatic reverse function"
- : Set to "Automatic forward function"
- Reciprocal mode
- : Set to "Reciprocal mode"

[Reference] Setting the reciprocal mode [page 119 to 121]

5. Memorize the selection by pressing or

The micromotor screen is displayed.

Settings of automatic reverse function, automatic forward function, and reciprocal mode are memorized in the starting condition after powering on.

* To cancel the setting change, press 🛛 without pressing ок



If a reverse screw-shaped file, etc. is attached to MX2, be sure to set to "No automatic reverse function", and switch the rotation direction using the forward/reverse switch on the micromotor. If that unit is set to "Automatic reverse function / automatic forward function", the file may get damaged, resulting in injury.



Automatic reverse function/ Automatic forward function

Setting of automatic forward function duration 1. Pick up the micromotor from the instrument holder

2. Select a memory from the tab switch M1 to M4 and press it

3. Press the torque control switch 2.1. to display the automatic reverse function screen

| Auto reverse | | Torque 🗙 | ∩User A |
|-----------------|--------------|--------------------|--------------------|
| AUTO P Rev 2 | 2.5 s | 2.1 _{Ncm} | 🔹 1:1 |
| AUTOS | + | + | AUTO 5 Rev |
| AUTO Fwd D | - | - | 2.1 _{Ncm} |
| Reciprocal mode | | ОК | |

 RUser A

 M1
 M2
 M3
 M4

 Image: Walk of the state of the sta

4. Select automatic forward function automatic forward function setting

5. Press the + - switch to select from 0.0-25.4 seconds The selection can be made at an interval of 0.1 second. If pressing and holding the + - switch, the time interval is 0.5 second.

6. Memorize the selection by pressing ок

The micromotor screen is displayed. Settings of automatic forward duration are memorized in the starting condition after powering on.

* To cancel the setting change, press 🛛 without pressing ок

Setting the reciprocal mode

Startup setting


1. Pick up the micromotor from the instrument holder

 M1
 M2
 M3
 M4

 Image: Main and the main

(Example) Micromotor MX2 Auto-Rev

| Auto reverse | Torque |
|-----------------|----------------------|
| AUTO Rev 2 | 3.0 _{Ncm} |
| AUTOS | + AUTO5 |
| AUTOD Fwd DD | - 3.0 _{Ncr} |
| Reciprocal mode | OK 🔮 |

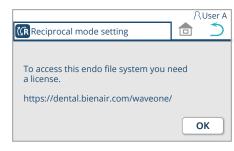

| CRID code 10 | 5 E9 1C ** ** ** ** ** | ∩User A |
|--------------|------------------------|---------|
| 123 | Activation | ОК |
| QWE | RTYU | ΙΟΡ |
| AS | DFGHJ | KL |
| ZXC | VBNM | . 🛛 |

| Fix 💽 | | | |
|--|--|----|------------------|
| ~ | | | |
| | | De | cip ro col |
| Valid only with files of RECIPROC [®] , or Bie | | | ciprocal mode |

- 2. Set the rotation speed at 5000 min-1 or less by pressing
- 3. Press the torque control switch 3.0

4. Press the reciprocal switch Reciprocal mode

When the reciprocal switch is selected, the frame color of it turns blue.

* Reciprocal mode switch is not displayed when the setting language is Japanese, or the micromotor except MX2 is used.

- * Pressing 🛛 switch returns the display to the micromotor screen.
- 5. Press or switch
 - The display requests you to obtain a license.
 - * Procedures 6 to 9 described below are displayed at first setting only.

After setting is completed, the micromotor screen is displayed.

- * Pressing S switch returns the display to the torque setting screen.
- 6. Press ок switch

The screen for entering activation code is displayed.

7. Obtain the activation code from the following website [Reference] How to obtain the activation code [Page 121] https://dental.bienair.com/waveone/

When obtaining the activation code, ID code shown at the top-left of screen is necessary.

8. Enter the activation code obtained in Procedure 7.

When switching alphanumeric input, press the alphanumeric switch ¹²³. Each press of the switch alternates the display in the order of ¹²³ - ^{ABC} - ¹²³.

9. After entering the code, press or switch

After a buzzer sound is heard, micromotor screen is displayed. Setting of the reciprocal mode is memorized in the starting condition after powering on.

| Enter the Activation code. | × |
|------------------------------------|---|
| Your Activation code is incorrect. | |
| Try again. | |
| | |

M3

×

Valid only with files of WaveOne[®] Gold, (RECIPROC[®], or Bien · Air CA ENDO (REF 1600955)

M1

🔊 Fix

₿

Auto reverse

AUTO Rev

AUTO Rev 5

AUTO5

Reciprocal mode

RUser A

Reciprocal

mode

🖹 🕘 🚺

s,å 🐟

∧User A

Reciprocal

mode

۵ 🙋

چ م

M4

Ø

Torque 🗙

3.0_{Ncm}

+

ок

- If the activation code is incorrect, the screen shown in the left is displayed.
- After pressing the \blacksquare switch, resume the setting from Procedure 8.

Canceling the reciprocal mode

- 1. Pick up the micromotor from the instrument holder Micromotor screen is displayed.
- 2. Press the Reciprocal switch.
- 3. Press one of the following switches, AUOS, AUOS, or AUOS

When the switch is selected, the frame color of it turns blue.

- * Pressing 🛛 switch returns the display to the micromotor screen.
- 4. Press oκ switch Micromotor screen is displayed.

(Example) Micromotor MX2 Auto-Rev



Website for obtaining activation code

| Welcome on the WaveOr | |
|-----------------------|------------------------|
| Please select ye | our product |
| iOptima INT | Dental unit |
| | Contact Birm-Air suppo |

Screen for entering information


| | Bien Alir Dental | | |
|--------------------------------|---|--|--|
| Welcome on the activation page | | | |
| WaveOne [®] Gold | | | |
| | | | |
| | Dental unit brand * | | |
| | Select your brand * •••• 1 | | |
| | Name * First name * | | |
| | Enter your name Enter your first name (2) | | |
| | Dental office* | | |
| | Email address * | | |
| | Enter your email address ···· (4) | | |
| | License code* | | |
| | Enter your license code | | |
| | ID code* | | |
| | Enter your ID code | | |
| | GENERATE THE ACTIVATION CODE | | |
| | Your Activation Code | | |
| | | | |
| | ··· () | | |
| | Please enter your activation code in the dental | | |
| | unit screen and get WaveOne activated. | | |
| | Thank you ! | | |
| | Contact Bien-Air support | | |

How to obtain the activation code

Select the "Dental unit" shown in the left figure.

- 1. Enter the following information into the box 1 to 6.
 - 1 Select "Belmont"
 - 2 Enter your name
 - 3 Enter your address
 - 4 Enter your email address
 - 5 Enter the license code contained in the box of contra-angle.
 - 6 Enter the ID code displayed on the touch panel of the doctor unit.
 - [Reference] Setting the reciprocal mode— Startup setting— 7. [Page 119]
- 2. Enter the information into the box 1 to 6 and press THE ACTIVATION CODE Activation code will be displayed in the box 7.
- Enter the obtained activation code into the box when prompted in Procedure 8— Startup setting—Setting the reciprocal mode [Page 119].

The content of display is subject to change without notice.

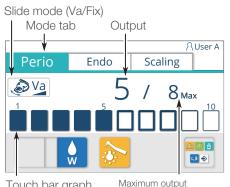

(displayed only in Va mode)

Touch bar graph

(Example of indication) Maximum output: 8; Output: 5









5-19 Ultrasonic scaler Perio/Endo/Scaling switch setting

When the ultrasonic scaler is picked up from the instrument holder, the electric scaler screen is displayed, and the mode is displayed at the top left.

The mode can be selected from "Perio", "Endo", and "Scaling". The mode is toggled by pressing the mode tab.

* When using NO PAIN, the mode cannot be selected.

The slide mode can be selected from Va or Fix.

Va (Variable)

By sliding the foot controller pedal to the left or right, the output can be adjusted within the range of the maximum output.

The output is decreased by sliding to the left, and is increased by sliding to the right.

The touch bar graph and output indication are displayed in accordance with the pedal slide position.

The maximum output setting can be changed using thetouch bar graph

Output of the electric scaler cannot be changed when the scaler is in operation. After turning off the scaler, change the output by using the touch bar graph.

Fix (Fixed)

The output is fixed only with the set value.

1. Pick up the ultrasonic scaler from the instrument holder

2. Press the slide mode switch *y* to display the slide mode setting screen

3. Select a slide mode


4. Memorize the selection by pressing ок

The electric scaler screen is displayed.

Settings of the slide mode are memorized in the starting condition after powering on.

* To cancel the setting change, press 🛛 without pressing 🤇 ОК Slide mode Va

Slide mode Fix

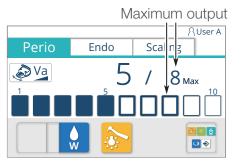

Setting of the maximum output in the slide mode Va

1. Pick up the ultrasonic scaler from the instrument holder

2. Press the touch bar graph **I** that corresponds to the required maximum output value, and select the maximum output between 1 and 10

Setting of the maximum output in the slide mode Fix

1. Pick up the ultrasonic scaler from the instrument holder

Perio Endo Scaling

2. Press the touch bar graph _____ that corresponds to the required output value, and select the output between 1 and 10



For proper use, be sure to carefully read the instructions for use attached to the relevant scaler in advance.

Memory setting for starting condition after powering on

When the following setting of handpiece condition is changed, it can be memorized in the steps specified below.

- Maximum output
- [Reference] Setting of the maximum output [page 123]
- Handpiece light condition (ON/OFF)

[Reference] Handpiece light switch [page 95]

- 1. Pick up the ultrasonic scaler from the instrument holder
- 2. Select a mode from the mode tab and press it
- 3. Change the handpiece condition (maximum rotation / spray / handpiece light condition).
- 4. Press the menu call switch 📴 to call the menu screen

 Perio
 Endo
 Scaling

 Default memory
 X

 Save the last scaler set-up (max power, light ON/OFF) for the next power-on.

 OK

6. Confirm the setting to be recorded, and press oκ to memorize it.

* If $\mathbf{o}\mathbf{\kappa}$ is not pressed but \mathbf{X} is pressed, the changed setting is valid until the main switch is turned off, but the setting returns to before the change once the main switch is turned off.



For proper use, be sure to carefully read the instructions for use attached to the relevant scaler in advance.



Endo

Perio


RUser A

 $\overline{\mathbf{v}}$

×


Scaling


Slide mode(Va/Fix)

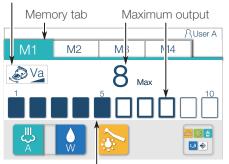



Touch bar graph(Upper limit output)


(Example of indication) Maximum output:10 / Operating output: 5

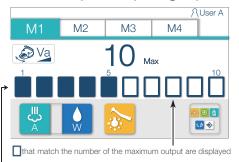

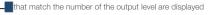

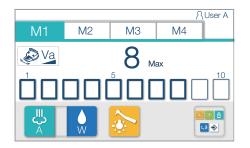

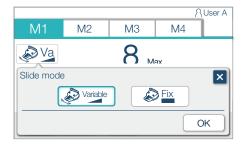

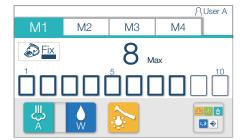

5-20 Air turbine/motor Setting of the slide mode

The initial setting of a handpiece upon powering on can be copied to another user memory.

When the air turbine/motor is picked up from the instrument holder, the turbine screen is displayed.

Memory can be selected from four memories (M1 to M4). The memory number can be switched by pressing the memory tab.

The slide mode can be selected either Va or Fix.

The memory of the air turbine/motor cannot be set individually for Va and Fix.

The Va and Fix setting must be common among M1 through M4.

Va (Variable)

Set the maximum output as explained in [Page 126]. In operation, the output can be adjusted within the range of this maximum output by sliding the foot control pedal to left (to decrease) or right (to increase) or by depressing (to increase) or releasing (to decrease) the pedal.

Fix

Set the maximum output as explained in [Page 126]. The upper limit output can be adjusted to the maximum output value by depressing the pedal regardless of the position of the pedal for the foot controller.

1.Pick up the air turbine/motor from the instrument holder

2.Press the slide mode switch 2 to display the slide mode setting screen

3.Select a slide mode

4.Memorize the selection by pressing οκ The turbine screen is displayed.

* To cancel the setting change, press 🛛 without pressing 🗌 ок

(Example of indication)

Maximum output : Change from 8 to 10

Maximum output



Setting of the maximum output

1. Pick up the air turbine/motor from the instrument holder

2. Press the touch bar graph That corresponds to the required maximum output value, and select the maximum output between 1 and 10

Memory setting for starting condition after powering on

When the following setting of handpiece condition is changed, it can be memorized in the steps specified below.

- Maximum output
- [Reference] Setting of the maximum output [page 126]
- Spray condition (ON/OFF/Water/Air)
- [Reference] Spray switch [page 95]
- Handpiece light condition (ON/OFF)
- [Reference] Handpiece light switch [page 95]
- 1. Pick up the air turbine/motor from the instrument holder


Select a memory from the tab switch M1 to M4 and press it
 Change the handpiece condition (maximum output / spray / handpiece light condition)

- 4. Press the menu call switch 🔤 to call the menu screen
- 5. Press the store switch \Rightarrow

The memory setting screen for starting condition is displayed.

6. Confirm the displayed setting, and press **ok** to memorize it

* If \mathbf{OK} is not pressed but \mathbf{X} is pressed, the changed setting is valid until the main switch is turned off, but the setting returns to before the change once the main switch is turned off.

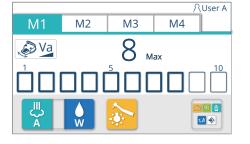



 All
 M2
 M3
 M4

 Default memory
 X

 Save the last HP set-up (max power, spray ON/OFF, light ON/OFF) for the next power-on.
 OK



For proper use, be sure to carefully read the instructions for use attached to the relevant air turbine/motor in advance.

5-21 Syringes


| BT14 | 3WAY syringe |
|-------------|----------------|
| 77-type | 3WAY syringe |
| LUZZANI | 6WAY syringe L |
| LUZZANI | 3WAY syringe |
| DCI syringe | |




For proper use, be sure to carefully read the instructions for use attached to the relevant syringe in advance.

77-type 3WAY syringe Operation

Depressing either or both buttons, this syringe offers air, water and spray. Syringe tip can be rotated freely.

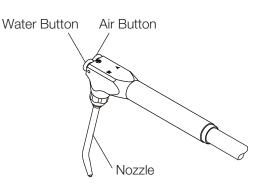

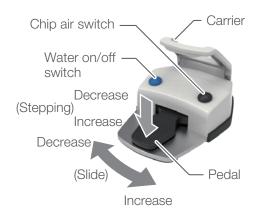

5-22 Foot controller (wired/wireless)

Pedal

<When using the micromotor>

The rotation and output can be adjusted by the levels of pedal sliding and stepping.

(Within the range of the indicated maximum rotation)

In Va mode, the rotation is decreased by sliding the pedal to the left, and is increased by sliding it to the right.

<When using the air turbine/motor>

Operation is only possible at a maximum output of 10 max, and on/off operation can be performed by stepping. It is not possible to adjust the output by the levels of sliding/stepping.

<When using the ultrasonic scaler>

The operation is turned on by stepping on the pedal.

It is not possible to adjust the output by the levels of sliding during the operation.

* Output of the electric scaler cannot be changed when the scaler is in operation. After turning off the scaler, change the output, then activate the scaler again.

The output can be adjusted by the level of pedal sliding.

It is not possible to adjust the output by the level of stepping.

(Within the range of indicated maximum output)

The present output is decreased by sliding the pedal to the left, and is increased by sliding it to the right.

Water on/off switch


The spray mode can be switched.

Chip air switch

Air is discharged from the chip of the micromotor or air turbine/motor.

Carrier

The foot controller can be moved by hanging it over the foot.

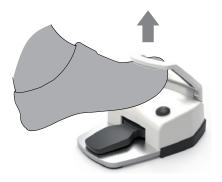


The batteries are not included. Insert the batteries referring to "How to replace batteries of the wireless foot controller" [page 131].

5 Operation

Cuspidor Unit type



Vac Pac Unit type

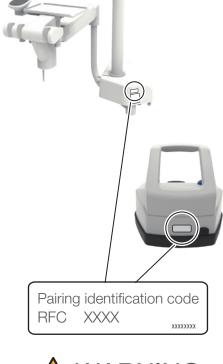



5-22-1 Confirmation of the pairing of the wireless foot controller

On the wireless foot controller and on the side of the cuspidor unit, stickers indicating the pairing identification code are attached. Confirm that the same pairing identification code is indicated.

Operate the wireless foot controller within one meter of the product.

If more than one unit of the product is used on the same floor, be careful not to operate a wireless foot controller that is paired with another unit.



Battery boxes

5-22-2 How to replace batteries of the wireless foot controller

When the battery level indication on the home screen becomes <a> , promptly replace the batteries with new ones.

- 1. Turn off the main switch
- 2. Remove the covers of the two battery boxes on the back of the wireless foot controller

Turn off the main switch when replacing batteries. Otherwise, switches or a pedal is accidentally pressed resulting in unintentional movement, leading to any damage or injury, or the product may not operate normally.





Be careful to put each battery in the correct direction (positive/negative).

4. Attach the cover of each battery box in order indicated with arrows (1, 2).




(2)

Do not use batteries other than the specified types. Do not mix new batteries with old ones, or mix different types of batteries (e.g. alkali dry batteries with NiMH batteries). Note that improper use of batteries causes leakage or rupture.

Take out batteries if the unit is not to be used for one week or longer.

5 Operation

Connection point There is a connection point where the rubber cap is removed.

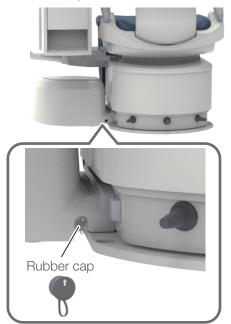


Wireless foot controller



Connector

Face the arrow mark upward

Hold this part while inserting

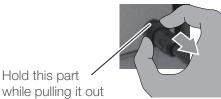

5-22-3 Cable connection of a wireless foot controller

If the wireless foot controller does not operate normally even after replacing the batteries, the unit can be used by connecting the cable contained in the package.

Electric wave used for the wireless foot controller could be interrupted depending on the conditions where the product is used, which might affect the functions of the handpiece. This is not the malfunction of the product.

Be sure to turn off the main switch before connecting the cable. Otherwise, the unit may fail to operate normally.

Before using the cable, take batteries out of the battery boxes. Otherwise, the unit may fail to operate normally.

When removing the cable, be sure to hold the indicated part while pulling it out. Otherwise, disconnection may result.

Cable connection is only for temporary use, and must not be used permanently.

How to connect

When connecting the cable to the chair or cuspidor unit, remove the rubber cap, and insert the connector until it clicks, with the arrow mark facing upward.


The same applies when connecting the cable to the wireless foot controller side.

* Keep the rubber cap of the wireless foot controller to avoid losing it.

How to remove

When removing the cable, be sure to hold the part indicated in the figure while pulling it out, and then attach the rubber cap.

Be sure to attach the rubber cap before using the unit. Use without attaching the rubber cap allows the entry of dust and water, resulting in failure.

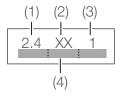

5-22-4 Operating precautions for the wireless foot controller

Radio-frequency interference with this product.

This product contains radio equipment that uses the full 2.4 GHz band ((1)), and interference with the mobile identifier band ((4)) is unavoidable. The FSK system ((2)) is employed as a modulation method, and the assumed interference distance is 10 meters ((3)). The assumed interference distance means the distance at which radio-frequency interference on premises radio stations or specified low power radio stations is anticipated.

Many other devices, including those described below, use the same frequency band as this product: Industrial devices such as microwave ovens, scientific and medical devices, premises radio stations for mobile object identification used for production lines in plants (radio stations that require a license), specified low-power radio stations (radio stations that require no license), and amateur radio stations (radio stations that require a license).

1) Check that no premise radio stations for mobile object identification or specified low-power radio stations are operating in the vicinity before starting to use this product.

2) If you notice that this product causes hazardous radio-frequency interference on a premises radio station for mobile object identification, promptly stop all emissions of radio waves and contact your local authorized Belmont dealer.

3) If you encounter problems such as a case in which this product causes hazardous radio-frequency interference on a specified low-power radio station for mobile object identification, contact your local authorized Belmont dealer.

Devices listed below may cause radio-frequency interference in the 2.4-GHz band:

- Wireless LAN devices

Receipt computer (used in the hospital wireless environment), mouse, keyboard (Bluetooth, 2.4-GHz band), microscope, intraoral camera, panoramic X-ray equipment, CT, etc.

- Mobile object identification apparatus (such as antitheft devices, entry and exit monitoring systems, etc.)

- Others (such as cautery knives, microwave ovens, etc.)

Certification of equipment

Wireless instruments built into this product are certified as radio-frequency devices for low-power communications in accordance with the Radio Act.

Certificates are indicated on the radio-frequency equipment.

Therefore, no license is required to use this product. If you take any of the actions listed below with this product, however, you may violate the law and be punished.

- Disassembling or modifying the radio-frequency instruments built into this product

- Removing the certificate labels from the radio-frequency instruments built into this product

Specifications of the wireless foot controller

Transmission system: F1D (FSK) Maximum transmission power: 1 mW Battery: 4 × AA batteries (Alkaline battery or NiMH (rechargeable) battery) Battery life: Approx. 1,640 hours


- Using AA Eneloop batteries
- Used at practices for 15 to 20 patients per day
- Used 5 days per week, one of which it is used only in the morning
- * The battery life varies according to the use environment (frequency of use, radio quality, etc.).
- * Eneloop is a registered trademark of the Panasonic Group.


5-23 Maintenance panel


- (1) Water service coupler
- This switch turns on the water supply to an external device.
- (2) Water supply control knob for the water service coupler

This knob adjust the quantity of water supplied to an external device. (3) Air service coupler

- Supplies air to an external device.
- (4) Main switch

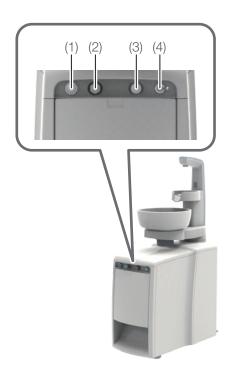

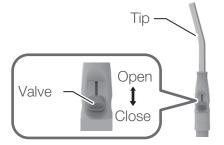

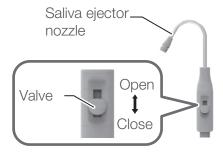

5-24 Handpiece (assistant holder)

Vacuum handpiece

Take the vacuum handpiece out of the assistant holder, and suction will start.

Since a delay circuit is provided, suction does not stop immediately when the vacuum handpiece is returned to the assistant holder. Suction will continue for approx. 4 seconds.

The suction volume may be controlled by opening or closing the valve.


Saliva ejector handpiece

Take the saliva ejector handpiece out of the assistant holder, and suction will start.

Since a delay circuit is provided, suction does not stop immediately when the saliva ejector handpiece is returned to the assistant holder. It will continue for approx. 4 seconds.

The suction volume may be controlled by opening or closing the valve.

5-25 Assistant hose clamp

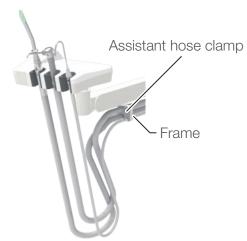

It is possible to fix the hose to prevent the handpiece hoses from contacting to the floor.


Assistant hose clamp is detachable from the frame, which is suitable for cleaning.

[Reference] Assistant hose clamp [page 201]


130



Touch panel

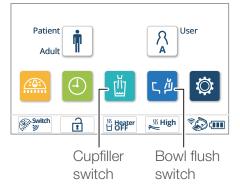

Assistant control panel

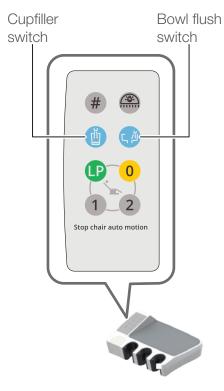

5-26 Filling a Cup

The following two methods can be selected to fill a cup: - Auto-cupfiller with sensor control: Place a cup on the base, and it

will be filled with water to the specified level.

(Cup, water level detection)

- Manual cupfiller: Press the cupfiller switch to fill with water for a specified time.

| | Cup-filling method |
|--|---|
| Auto-cupfiller with sensor control (Cup, water level detection) | When a cup is placed on the cupfiller base, a sensor detects the cup, and fills it with the specified quantity of water. To interrupt filling, press the cupfiller switch 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 |
| | the correct reduced quantity of water. * Even when a fully-filled cup is placed on the base, a small quantity water will be supplied to detect the water level. [Ref] Setting the cupfiller water level [Page 178] |
| Manual cupfiller | When the cupfiller switch 💆 🍈 on the touch panel or assistant control panel is pressed, water is supplied for the specified time. To interrupt filling, press the cupfiller switch 💆 🍅 again. |


If "cuspidor bowl flush" is set to "continuous flush," water is supplied from the bowl flush nozzle to flush the cuspidor bowl while the cup is filled.

To interrupt the flush, press the bowl flush switch 🧐 🗐 on the touch panel or assistant control panel.

If any droplet or dirt is attached on the surface of cupfiller sensor, the cupfiller may not work properly. In that case, wipe off the droplet or dirt after switching the sensor into disable mode.

Sensor disable mode (Cupfiller base maintenance mode)

The sensor disable mode can be activated by holding down the cupfiller switch is on the touch panel or assistant control panel for 2 seconds or more. In the sensor disable mode, continuous bleeps are emitted, and sensor control is disabled for the auto-cupfiller. The LED indicator on the cuspidor unit displays pink. To terminate the sensor disable mode, press the cupfiller switch is again. The LED indicator returns to its original color.

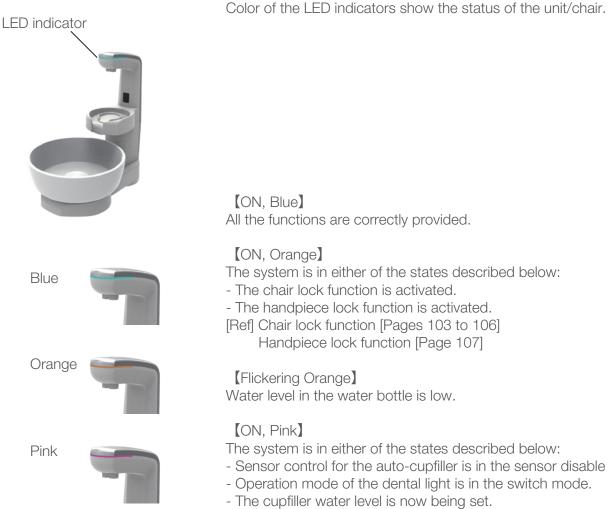


5-27 LED indicator

(ON. Blue) All the functions are correctly provided. (ON, Orange) The system is in either of the states described below: - The chair lock function is activated. - The handpiece lock function is activated. [Ref] Chair lock function [Pages 103 to 106] Handpiece lock function [Page 107] (Flickering Orange) Water level in the water bottle is low.

(ON, Pink)

- The system is in either of the states described below:
- Sensor control for the auto-cupfiller is in the sensor disable mode.
- Operation mode of the dental light is in the switch mode.
- The cupfiller water level is now being set.
- [Ref] Sensor disable mode [Page 136]
 - Operation mode of the dental light [Page 92] Setting the cupfiller water level [Page 178]

Since the color of the LED indicator gives an indication of the status of the chair/handpiece lock or mode transition, always check that the LED indicator is functioning correctly before starting daily work.

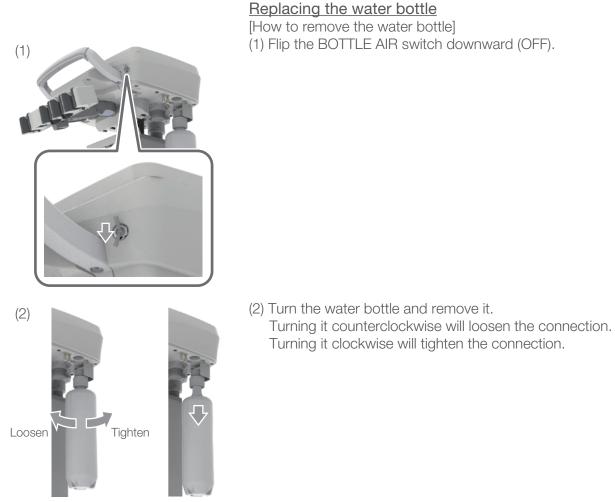

5-28

Water bottle

Refill water when the water bottle level becomes low.

[How to attach it]

- Reattach it in the reverse order of (1) to (4).
- * When not using the water from the water bottle, flip the WATER toggle switch backward (CITY).






5-29 USB port

The USB port accepts a doctor table for a USB-type intraoral camera or other device.

Connecting cable should be necessary inside doctor table as per the specification of the device to be added.

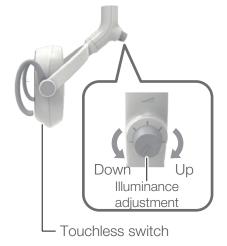



5-30 EURUS Light

Turn on the product's main switch; the dental light comes on and the indicator lights up.

The dental light can be turned on or off using the touchless switch or the dental light switch and the touch panel or assistant control panel.

Turning the dental light on or off with the touchless switch.

The touchless switch is activated by a sensor. The sensor detects an object of the same width as the sensor window at a distance of approx. 65 mm or closer.

Move a hand across the switch to turn it on, and again to turn it off. * If the surface of the touchless switch is dirty, it may degrade the sensitivity of the switch. If the surface is dirty, clean it with a soft cloth.

Adjusting illuminance in the treatment mode

When the light is turned on in the treatment mode, the operator can adjust the illuminance steplessly between 3,000 and 28,000 lx by turning the illuminance controller shown in the figure on the left. Turn it clockwise viewed from the rear of the light head to increase the illuminance or counterclockwise to decrease it. The illuminance cannot be adjusted when the light is turned on under ON mode.

Switching from the treatment mode to Composite-Safe mode

Hold your hand over the touchless switch within the 65-mm range for the specified time, or press the dental light switch a on the touch panel of the assistant control panel for the specified time to change operation to the Composite-Safe mode.


The illuminance cannot be adjusted when the light is turned on under Composite-Safe mode.

Repeat the above step to switch operation back to the treatment mode.

* When the light is turned off in the Composite-Safe mode and turned on again, it will be in the treatment mode.



Before use, always read the Instructions for each dental light to ensure correct usage.

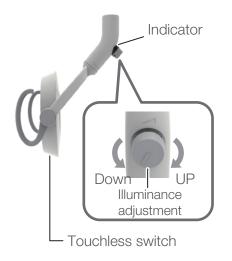



Approx. 65 mm or closer from the touchless switch (Effective range of the sensor)

5-31 900 Dental Light

Turn on the product's main switch; the dental light turns on and the indicator displays in green.

The dental light can be turned on or off using the touchless switch or the dental light switch and an the touch panel or assistant control panel.

Turning the dental light on or off with the touchless switch. The touchless switch is activated by a sensor. The sensor detects an object at distance of approx. 65 mm or closer from the switch. Move a hand across the switch to turn it on, and again to turn it off. * If the surface of the touchless switch is dirty, it may degrade the sensitivity of the switch. If the surface is dirty, clean it with a soft cloth.

Adjusting the illuminance in the treatment mode

The illuminance can be adjusted between 4,000 and 35,500 lx by turning the illuminance controller.

Switching from the treatment mode to Composite-Safe mode Hold your hand over the touchless switch within the 65-mm range for the specified time, or press the dental light switch and on the touch panel of the assistant control panel for the specified time to switch operation to the Composite-Safe mode; the indicator flickers in green.

The illuminance cannot be adjusted when the light is turned on under Composite-Safe mode.

Repeat the above step to switch operation to the treatment mode; the indicator turns steady green.

| Mode | Indicator |
|---------------------|------------------|
| Treatment mode | ON,Green |
| Composite-Safe mode | Flickering Green |

* When the light is turned off in the Composite-Safe mode and turned on again, it will be in the treatment mode.



Before use, always read the Instructions for Use supplied with each dental light to ensure correct usage.



5-32 Monitor bracket

The monitor can be swung to a position where you can easily see it. When adjusting the position, hold the monitor while you move it.

When moving the monitor, do not let it touch the wall or other object.



Do not apply an excessive load or shock to the monitor or monitor bracket. To avoid damage or injury, ensure that the monitor satisfies the following specifications:

Size: 21 inches Weight: 6 kg

5-33 Panoramic viewer


Turn the switch upward to turn on the screen and downward to turn it off.

Turn the luminance controller upward to increase the luminance and downward to lower it.

When it is not being used, turn it off.

The panoramic viewer is designed to supplement X-ray observation and is not for examination or diagnosis.


5-34 Dental viewer

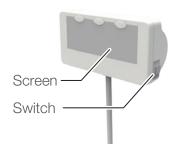


Press the switch to turn the screen on. Press it again to turn the screen off.

When it is not being used, turn it off.

The dental viewer is designed to supplement X-ray observation and not for examination or diagnosis.

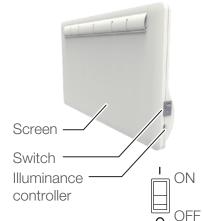

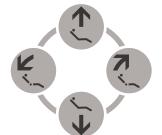


Manual control switches







Automatic return switch

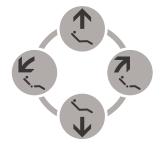

Manual control switches



Last position switch

6-1 Setting for automatic operation

6-1-1 Setting the treatment and entry/exit positions

1. Move the chair to the treatment position or entry/exit position using the manual control switch as required.

2. Press and hold the preset switch 1 or 2 (treatment position) or automatic return switch 0 (entry/exit position) for approx. 5 seconds. A beep sounds, and the chair is set to the position.

3. Move the chair away from the set position, and press the preset switch to check the chair moves to the set position.

* The angle of the legrest and length the footrest projects are determined automatically based on the angle of the backrest when set.

[Ref] Angular relationship between the backrest, legrest, and footrest [Page 71]

6-1-2 Setting the mouth rinsing position

1. Move the chair to the preset mouth rinsing position using the manual control switch.


2. Press and hold the last position switch (p) for approx. 5 seconds. A beep sounds, and the chair is set to the position.

3. Move the chair away from the set position, and press the last position switch (P) to check the chair moves to the set position.

* The angle of the legrest and the length the footrest projects are automatically determined based on the angle of the backrest when set.

[Ref] Angular relationship between the backrest, legrest, and footrest [Page 71]

Membrane switches on the doctor control panel

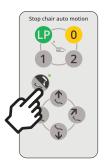

6-1-3 Freely setting the legrest angle

The legrest angle in auto mode is determined by the backrest angle. Change the legrest angle by the following the procedure below:

1. Move the seat of the chair and backrest to the desired position using the manual control switch or preset switch on the control panel. (The stick switch for manual/automatic operation can also be used to do the same.)

* A stick switch may not be installed depending on which specifications you selected.

2. Press and hold the chair/headrest membrane switch S on the control panel for 5 seconds or more.

3. While the bleeps are sounding, move the legrest to the desired angle.

Press the manual control switch O on the control panel, or turn the stick for manual control upward (in the \clubsuit direction) to raise the legrest. Press the manual control switch O on the control panel, or turn the stick for manual control downward (in the \checkmark direction) to lower the legrest.

4. Press and hold the switch for 5 seconds or more to set (automatic return switch ⁰, last position switch ¹, or preset switch ¹ or ² on the control panel) until a beep sounds, indicating that the setting has been saved.

(Or, keep turning the stick switch for automatic chair operation in the direction to set until a beep sounds, indicating that the setting has been saved.)

5. On completing the setting or to cancel the process without saving the setting, press the manual control switch or on the control panel.

(Or, turn the manual control stick switch to the right (in the \neg direction) or left (in the \checkmark direction).)

Bleeps stop, and the system returns to the normal state.



Spittoon valve cleaning

Handpiece

Language

6-2 Function switch

Press the function switch in the Home screen to display the Function setting screen.

Select an item to display the corresponding setting screen.

Press the initial switch to cancel operation and return from the current screen to the Home screen.

Press the \supset switch to cancel operation and return from the current screen to the previous screen.

Press the \vee switch to proceed to the next page.

Press the \wedge switch to return to the previous page.

| Ö Fu | nction setting | 5 |
|------|--------------------------------------|--------|
| | Cupfiller, bowl flush | |
| | Timer, control panel | |
| | Emergency cancellation of chair lock | |
| | Service mode | \vee |

6-2-1 Setting items with the function switch

| Setting items appearing on the touch panel | | | | |
|--|----------|--|--|--|
| Flushing, Washing the vacuum line | | | | |
| Flushing | Page 147 | | | |
| Washing the vacuum line | Page 156 | | | |
| Handpiece | | | | |
| Drive air pressure | Page 173 | | | |
| Spray mode | Page 174 | | | |
| Handpiece light activation | Page 175 | | | |
| Micromotor light brightness | Page 176 | | | |
| Cup, bowl | | | | |
| Cupfiller water level | Page 178 | | | |
| Interlock of cup-filling, bowl flush | Page 180 | | | |
| Bowl flush time | Page 181 | | | |
| Language | Page 182 | | | |
| Timer and control panel | | | | |
| Timer alarm sound | Page 183 | | | |
| Key touch volume | Page 184 | | | |
| Sleep mode time | Page 185 | | | |
| Emergency cancellation chair lock | Page 186 | | | |
| Service mode | * | | | |

*The service mode is designed exclusively for service engineers to carry out maintenance on this product. 6-2-2 Selecting the flushing method and carrying out flushing

Cuspidor Unit type

To flush the water circuit of the handpiece alone 1. Press the function switch (2). The Function setting screen appears.

2. Press the 'Flushing, Washing the vacuum line' switch.


If the washing function for vacuum line is not equipped, 'Flushing' appears on the screen instead. Procedures shown below are not applicable.

3. Press the 'Flushing' switch.

4. Check that 'Short mode 00:40' is selected, and press

* When using the timer, the screen shown in the figure on the left is displayed.



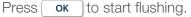

| Patier Adu | 🕅 | | ۲ A | Jser |
|---------------|---|--------|--------|------|
| :0: | | Ŭ | С, 겓 | |
| Manual U | | Heater | Migh N | °@@ |

| 🔅 Fu | nction setting | 5 |
|------|-----------------------------------|----------|
| | Flushing, Washing the vacuum line | |
| | Spittoon valve cleaning | \wedge |
| | Handpiece | |
| | Language | V |

| ຟູຟູ" Flu | shing Washing the vacuum line ? |
|-----------|--------------------------------------|
| (| Washing the vacuum line |
| ļ | Flushing and washing the vacuum line |
| | |

| Flu | ushing | ? 🗈 🌖 |
|--------|------------|-------|
| Select | type. | |
| | Short mode | 00:40 |
| | Long mode | 10:00 |
| | | ОК |


| Flushing | | 5 |
|--|---|---|
| Timer is in use. Stop the timer and start flushing? | | |
| × | 0 | ж |



- 5. Pick up the handpiece you want to flush, and set it in the cuspidor bowl.
 - The number of the handpiece you have picked up appears. (Handpieces are numbered 1, 2, 3 and 4 from front left.)

The screen does not return to the Home screen even when the \bigcirc switch is pressed.

Always flush the handpiece with the main body of the ultrasonic scaler attached. Flushing without it may cause a malfunction.

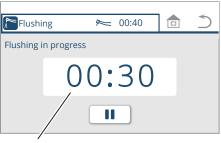

Remaining time (e.g. 30 sec.)

| Flushing | 00:40 | 5 |
|----------------|-------|---|
| Flushing in pa | use | |
| | 00:30 | |
| × | | |

| Flushing 🛌 | : 00:40 |
|--|-------------|
| Flushing is completed. | |
| Return the handpieces to the handpiece holder. | |
| | 3 2 2 |

6. Step on the pedal of the foot controller.

When the pedal is released, the handpiece continuously discharges water for 40 seconds.

The screen does not return to the previous screen even when the \supset switch is pressed.

* To suspend flushing, press the **II** switch on the touch panel, the membrane switch, or either switch on the assistant panel, or step on the foot controller pedal.

* To resume flushing, press the switch.

* To terminate flushing prematurely after pressing , press /membrane switch/assistant operation panel switch, or step on the foot controller.

* To terminate flushing prematurely without pressing [, press the membrane switch/assistant operation panel switch, or step on the foot controller.

7. Once flushing is completed, return the handpiece to the instrument holder.

When the handpiece is returned, the home screen appears.

Water is not automatically discharged from the syringe. Press the W lever/button for 40 seconds to discharge.

| Patien Adul | 🛄 | | الم A | Jser |
|----------------|---|--------|----------|---------|
| <u>:0:</u> | | Ш | ς, μ | |
| Manual | 1 | Heater | ∭ High | °@ (111 |

Flushing the water circuits of the handpiece and the cupfiller/cuspidor bowl

- * Place a cup on the cupfiller base
- 1. Press the function switch @ .
- The Function setting screen appears.

5 Function setting Flushing, Washing the vacuum line \wedge Spittoon valve cleaning Handpiece Language

Flushing, Washing the vacuum line ?

Washing the vacuum line

Flushing and washing the vacuum line

Flushing

5

2. Press the 'Flushing, Washing the vacuum line' switch.

If the washing function for vacuum line is not equipped, 'Flushing' appears on the screen instead. Procedures shown below are not applicable.

3. Press the 'Flushing' switch.

| Flu | ushing | ? 💼 🌖 |
|--------|------------|-------|
| Select | type. | |
| | Short mode | 00:40 |
| | Long mode | 10:00 |
| | | ОК |

| 4. Ch | eck that | 'Long | mode | 10:00' | is selected, | and | press | [|
|-------|----------|-------|------|--------|--------------|-----|-------|---|
|-------|----------|-------|------|--------|--------------|-----|-------|---|

ок

| Select | type. | | |
|--------|------------|-------|----|
| | Short mode | 00:40 | |
| | Long mode | 10:00 | |
| | | | ОК |
| | | | |
| | | | |


| Į | Flushing | | 5 |
|---|--|---|---|
| | Timer is in use. Stop the timer and start flushing? | | |
| | × | (| ж |

* When using the timer, the screen shown in the figure on the left is displayed.



to start flushing. Press ОК

6 Setting/Adjustment

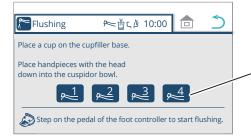


- 5. Pick up the handpiece you want to flush, and set it in the cuspidor bowl.
 - The number of the handpiece you picked up appears. (Handpieces are numbered 1, 2, 3 and 4 from front left.)

The screen does not return to the Home screen even when the \bigcirc switch is pressed.

Before starting flushing, always place a cup on the cupfiller base. Without a cup, water will splash onto the wall or chair.

Always flush the handpiece with the main body of the ultrasonic scaler attached. Flushing without it may cause a malfunction.

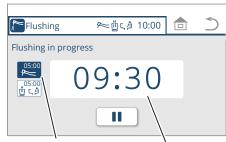

Status of handpiece flushing

Remaining time (e.g. 9 min. 30 sec.)

| Flushin | g 🍋 🖄 10:00 | 5 |
|--------------------------|-------------|---|
| Flushing in | pause | |
| 05:00 205:00 世 5 道 | 04:30 | |
| | × | |

Cupfiller and bowl flush status

| 🎦 Flushing 🖉 ក្ម៉េត្ | 10:00 | | 5 |
|---|-------|---|---|
| Flushing is completed. Return the handpieces to the handpiece holder. | 1 | | |
| | 3 | 4 | |

6. Step on the pedal of the foot controller.

When the pedal is released, the handpieces continuously discharge water for 5 minutes. Then the cupfiller nozzle and bowl flush nozzle discharge water for 5 minutes.

The screen does not return to the previous screen even when the switch $\stackrel{\curvearrowleft}{\supset}$ is pressed.

* To suspend flushing, press the switch on the touch panel, the membrane switch, or either switch on the assistant panel, or step on the foot controller pedal.

* To resume flushing, press [🕨]

* To terminate flushing prematurely after pressing
 press × /membrane switch/assistant operation panel switch, or step on the foot controller.

* To terminate flushing prematurely without pressing , press the membrane switch/assistant operation panel switch, or step on the foot controller.

7. Once flushing is completed, return the handpiece to the instrument holder.

When the handpiece is returned, the home screen appears.

Water is not automatically discharged from the syringe. Press the W lever/button for 40 seconds to discharge.



Flushing, Washing the vacuum line

Spittoon valve cleaning

Flushing, Washing the vacuum line ?

Washing the vacuum line

Flushing and washing the vacuum line

Handpiece

Language

Flushing

© Function setting

 \wedge

V

To flush the water circuit of the handpiece alone (Short mode)1. Press the function switch I. Press the function setting screen appears.

2. Press the 'Flushing, Washing the vacuum line' switch.

If the washing function for vacuum line is not equipped, 'Flushing' appears on the screen instead. Procedures shown below are not applicable.

3. Press the 'Flushing' switch.

Vac Pac Unit type

- Flushing ?
 Select type.
 Short mode 00:40
 Long mode 05:00
 OK
- Flushing

4. Check that 'Short mode 00:40' is selected, and press or

* When using the timer, the screen shown in the figure on the left is displayed.




Press oκ to start flushing.



- 5. Pick up the handpiece you want to flush, and set it in the container, such as a bucket.
 - The number of the handpiece you have picked up appears. (Handpieces are numbered 1, 2, 3 and 4 from front left.)

The screen does not return to the Home screen even when the \bigcirc switch is pressed.

Always flush the handpiece with the main body of the ultrasonic scaler attached. Flushing without it may cause a malfunction.

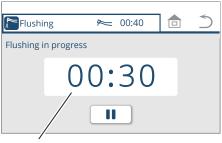

Remaining time (e.g. 30 sec.)

| Flushin | g | Į, | 00:40 | 5 |
|-------------|-------|----|-------|---|
| Flushing in | pause | | | |
| | 0 | 0: | 30 | |
| : | × | | | |

| Flushing 🛌 | : 00:40 |
|--|-------------|
| Flushing is completed. | |
| Return the handpieces to the handpiece holder. | |
| | 3 2 2 |

6. Step on the pedal of the foot controller.

When the pedal is released, the handpiece continuously discharges water for 40 seconds.

The screen does not return to the previous screen even when the \supset switch is pressed.

* To suspend flushing, press the **II** switch on the touch panel, the membrane switch, or either switch on the assistant panel, or step on the foot controller pedal.

* To resume flushing, press the switch.

* To terminate flushing prematurely after pressing , press /membrane switch/assistant operation panel switch, or step on the foot controller.

* To terminate flushing prematurely without pressing [, press the membrane switch/assistant operation panel switch, or step on the foot controller.

7. Once flushing is completed, return the handpiece to the instrument holder.

When the handpiece is returned, the home screen appears.

Water is not automatically discharged from the syringe. Press the W lever/button for 40 seconds to discharge.



Flushing, Washing the vacuum line

Spittoon valve cleaning

Handpiece

Language

 \wedge

V

Function setting

- To flush the water circuit of the handpiece alone (Long mode) 1. Press the function switch O . The Function setting screen appears.
- 2. Press the 'Flushing, Washing the vacuum line' switch.

If the washing function for vacuum line is not equipped, 'Flushing' appears on the screen instead. Procedures shown below are not applicable.

- Flushing Washing the vacuum line ?

 Flushing
 Washing the vacuum line
 Hushing and washing the vacuum line
- 3. Press the 'Flushing' switch.

| Flu | ushing | ? 💼 🍏 |
|--------|------------|-------|
| Select | type. | |
| | Short mode | 00:40 |
| | Long mode | 05:00 |
| | | ОК |

4. Check that 'Long mode 05:00' is selected, and press oκ

| 4 | Flushing | | 5 |
|---|--|---|---|
| | Timer is in use. Stop the timer and start flushing? | | |
| | × | C | ж |


- * When using the timer, the screen shown in the figure on the left is displayed.
- Press × to cancel flushing. Press σκ to start flushing.



- 5. Pick up the handpiece you want to flush, and set it in the container, such as a bucket.
 - The number of the handpiece you have picked up appears. (Handpieces are numbered 1, 2, 3 and 4 from front left.)

The screen does not return to the Home screen even when the \bigcirc switch is pressed.

Always flush the handpiece with the main body of the ultrasonic scaler attached. Flushing without it may cause a malfunction.

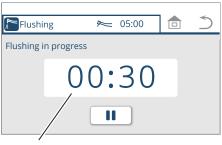

Remaining time (e.g. 30 sec.)

| Flushing | ; 🍋 05:00 | |
|-------------|-----------|--|
| Flushing in | pause | |
| | 00:30 | |
| × | | |

| 🎢 Flushing 🛛 🗮 | 05:00 | | Ś |
|--|-------|---|---|
| Flushing is completed. | | | _ |
| Return the handpieces to the handpiece holder. | 1 | | |
| | | 4 | |

6. Step on the pedal of the foot controller.

When the pedal is released, the handpiece continuously discharges water for 5 minutes.

The screen does not return to the previous screen even when the \supset switch is pressed.

* To suspend flushing, press the **II** switch on the touch panel, the membrane switch, or either switch on the assistant panel, or step on the foot controller pedal.

* To resume flushing, press the switch.


* To terminate flushing prematurely after pressing , press /membrane switch/assistant operation panel switch, or step on the foot controller.

* To terminate flushing prematurely without pressing [, press the membrane switch/assistant operation panel switch, or step on the foot controller.

7. Once flushing is completed, return the handpiece to the instrument holder.

When the handpiece is returned, the home screen appears.

Water is not automatically discharged from the syringe. Press the W lever/button for 40 seconds to discharge.

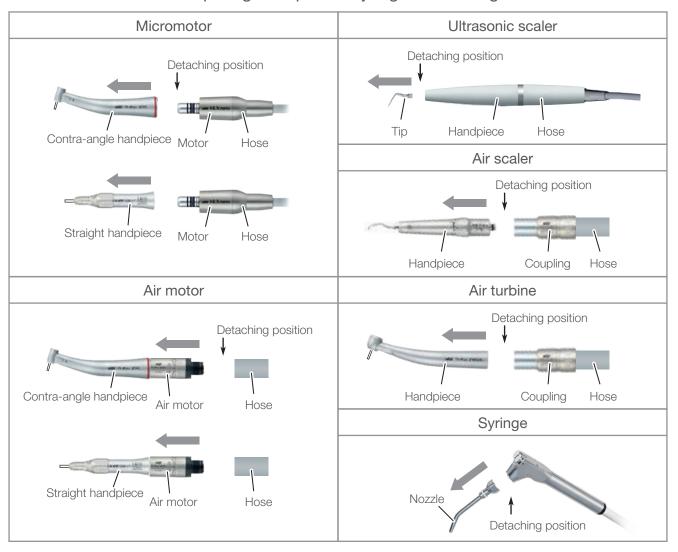

Preparing handpieces/syringe for flushing

After flushing, wipe the handpiece with a soft cloth to remove excess moisture.

When picking up the handpiece after flushing, remaining air may be discharged. This is normal.

6-2-3 Selecting the vacuum line washing method and conducting washing

Washing the vacuum line only

The following three options are available: Rinse the vacuum line+wash the vacuum line with cleaner (refer to page 158), Rinse the vacuum line (refer to page 161), and Wash the vacuum line with cleaner (refer to page 163).

1. Press the function switch (2). The Function setting screen appears.

| Ö Fu | nction setting | 5 |
|------|-----------------------------------|----------|
| (| Flushing, Washing the vacuum line | ~ |
| (| Spittoon valve cleaning | \wedge |
| | Handpiece | |
| | Language | × |

2. Press the 'Flushing, Washing the vacuum line' switch.

- 3. Press the 'Washing the vacuum line' switch.
- * To display the explanation of flushing and washing the vacuum line, press the ? switch (refer to page 157).
- * If you select 'Washing the vacuum line', the amount of remaining cleaner will be checked. If the amount of cleaner is sufficient, the screen shown in the left figure appears.

If the amount of cleaner is insufficient, the screen shown in the left figure appears. Refill the cleaner. [Reference] How to refill the cleaner [pages 172]

After refilling the cleaner, press the \mathbf{X} switch to display the 'Washing the vacuum line' screen.

| | Handpiece | |
|--------|--------------------------------------|---|
| | Language | |
| | | |
| (۳)"FL | ishing, Washing the vacuum line ? | 5 |
| | Flushing | |
| | Washing the vacuum line | |
| | Flushing and washing the vacuum line | |

| Mwa | shing the vacuum line | | 5 | |
|-----|---|--|---|--|
| • | Orotol plus | | | |
| | Rinse the vacuum line+wash the vacuum line with cleaner | | | |
| | Rinse the vacuum line | | | |
| | Wash the vacuum line with cleaner | | | |

| Washing the vacuum line | × |
|---|---|
| Cleaner is almost empty. Vacuum line may not be fully washed. Refill the cleaner. | |
| Without the cleaner, vacuum line will be washed only with water. | |
| | |

| 🕅 Flushing, Washing the vacuum line 🛛 💼 | 5 |
|---|----------|
| Flushing | |
| Short mode: to flush the water through the handpiece circuits for every patient. | \wedge |
| Long mode: to flush the water through water supply circuit to discharge the retained water in the unit before starting everyday treatment. | V |
| | |
| M [®] Flushing, Washing the vacuum line | 5 |
| Washing the vacuum line | |
| Rinse and wash the vacuum line with cleaner: cleaning, deodorizing, and washing the vacuum circuits. Do it once or twice a day. | Λ |
| Rinsing: use it when 60 minutes or more have passed after cleaner-washing, or in the next morning as a last process of washing. Also, it can be used to remove the blood or powder in the vacuum circuits after invasive treatment or using the powder jet handpiece. | V |
| | |
| | |

Washing the vacuum line

Washing with cleaner: cleaning, deodorizing, and washing the vacuum circuits. Do it for every patient.

 \wedge

Flushing and washing vacuum line

This is to conduct rinsing both flushing circuits (Long mode) and vacuum circuits before treatment every day.

| Pushing, Washing the vacuum line | | 5 |
|---|-----------|----------|
| Washing the vacuum line | | |
| Rinse and wash the vacuum line with cleaner: cleaning, deodorizing, and washing the vacuum circuits. Do it once or twice a day. | | \wedge |
| Rinsing: use it when 60 minutes or more have passe after cleaner-washing, or in the next morning as a la process of washing. Also, it can be used to remove t blood or powder in the vacuum circuits after invasiv treatment or using the powder jet handpiece. | ist he | V |
| | | |

| Flushing, Washing the vacuum line | 5 | |
|--|----------|--|
| Flushing | | |
| Short mode: to flush the water through the handpiece circuits for every patient. | \wedge | |
| Long mode: to flush the water through water supply circuit to discharge the retained water in the unit before starting everyday treatment. | | |

Explanation of flushing and washing the vacuum line

1. Press the ∨ switch.

The display moves to screen 2.

The screen provides an explanation about washing the vacuum line; "Rinse and wash the vacuum line with cleaner", and "Rinsing".

2. Press the \vee switch.

The display moves to screen 3.

The screen provides an explanation about washing the vacuum line; "Washing with cleaner" and "Flushing and washing the vacuum line".

3. Press the \wedge switch.

The display returns to screen 2.

The screen provides an explanation about washing the vacuum line; "Rinse and wash the vacuum line with cleaner", and "Rinsing".


4. Press the \wedge switch.

The display returns to screen 1.

The screen provides an explanation about flushing: Short mode and Long mode.

5. Press the 🔵 switch.

The display returns to screen 3 shown on page 156.

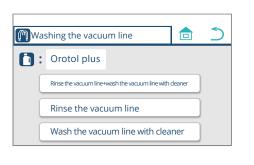

| Rinse the vacuum line+wash the vacuum line with deaner |
|---|
| Orotol plus Refill the cleaner Remove the chip/nozzle of the handpieces for vacuum/saliva ejector and insert the handpieces for vacuum/saliva ejector to each connector. Cover the connector when handpiece for vacuum/saliva is not inserted. 2 Do not use cleaner except the designated one. |
| |

Rinsing the vacuum line+washing the vacuum line with cleaner

1. Press the 'Rinse the vacuum line+wash the vacuum line with cleaner' switch.

* If the amount of cleaner is insufficient, the screen shown in the left figure appears. Refill the cleaner.

[Reference] How to refill the cleaner [pages 172]

2. Start rinsing the vacuum line+washing the vacuum line with cleaner.

(1) Open the maintenance panel cover (top) of the cuspidor unit. Pick up the vacuum and saliva ejector handpieces from the assistant holder and insert them into the connector.

[Reference] Procedure for washing with cleaner [page 171] (At this time, the suction is not activated.)

| Rinse the vacuum line+wash the vacuum line with deaner |
|---|
| i): Orotol plus emove the chip/nozzle of the handpieces for acuum/saliva ejector and insert the handpieces for acuum/saliva ejector to each connector. 1 Cover the connector when handpiece for vacuum/saliva is not inserted. 2 Do not use cleaner except the designated one. |
| |
| Rinse the vacuum line+wash the vacuum line with deaner |
| insing the vacuum line+washing the vacuum line with cleaner are in process. |

8 min.

X

about


| 2 Press the | | switch. |
|-------------|--|---------|
|-------------|--|---------|

* Before pressing the switch, make sure to insert the vacuum and saliva ejector handpieces into the connector.

The screen does not return to the home screen even when the $\ensuremath{\widehat{}}$ switch is pressed.

- * To stop washing the vacuum line in progress, press the
 - × switch.

6 Setting/Adjustment

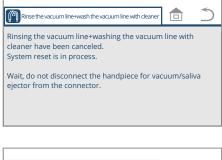

Rinsing the vacuum line+wash the vacuum line with deaner in the vacuum line with deaner washing the vacuum line with deaner have been canceled. Pressing OK button starts the suction. Return the handpieces for vacuum/saliva ejector to the assistant holder.

| Minse the vacuum line+wash the vacuum line with deaner |
|---|
| Rinsing the vacuum line+washing the vacuum line with cleaner have been canceled. |
| Pressing OK button starts the suction. Return the handpieces for vacuum/saliva ejector to the assistant holder. |
| |

* If the x switch is pressed during washing, the screen shown in the left figure appears.


If the handpiece is removed from the connector while the screen on the left is displayed, the cleaner will be left inside the handpiece and handpiece hose. Leaving the cleaner standing may cause a bacterial growth.

Press the **OK** switch.

* After the suction starts, return the vacuum and saliva ejector handpieces to the assistant holder.

When the vacuum and saliva ejector handpieces are returned to the assistant holder, the screen returns to the home screen.



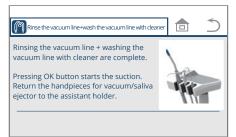

3. When you have finished rinsing the vacuum line+washing the vacuum line with cleaner, press the ок switch.

* After the suction starts, return the vacuum and saliva ejector handpieces to the assistant holder.

When the vacuum and saliva ejector handpieces are returned to the assistant holder, the screen returns to the home screen.

After the suction starts, remove the vacuum and saliva ejector handpieces from the connector.

Dripping of the cleaner can be prevented.

If the cleaner drips from the connector, wipe it off immediately. Places where the cleaner adheres may become discolored.

| E44:No water in the bottle | | 5 | |
|--|--|---|--|
| Rinsing the vacuum line+washing the vacuum line with cleaner have been canceled. System reset is in process. | | | |
| Wait, do not disconnect the handpiece for vacuum/saliva ejector from the connector. | | | |

4. In the event of an error state, take appropriate measures. In the event of an error state before/upon/during washing with cleaner/rinsing, the system will be reset and the error screen will appear.

"Message on the touch panel and measures to be taken" Refer to pages 218 to 220 and take measures in accordance with the remedies given.

e.g. E44: No water in the bottle

 F44:No water in the bottle.
 Image: Constraint of the sector of the assistant holder.

 Pressing OK button starts the suction.
 Return the handpieces for vacuum/saliva ejector to the assistant holder.

 Image: Constraint of the sector of the sector of the sector of the assistant holder.
 Image: Constraint of the sector of the

1IT

Pressing OK button starts the suction. Return the handpieces for vacuum/saliva ejector to the assistant holder. 5. Press the **OK** switch.

* After the suction starts, return the vacuum and saliva ejector handpieces to the assistant holder.

When the vacuum and saliva ejector handpieces are returned to the assistant holder, the screen returns to the home screen.

| Rinsing | the | vacuum | line |
|---------|-----|--------|------|
| | | | |

- 1. Press the 'Rinse the vacuum line' switch.

2. Start rinsing the vacuum line.

① Open the maintenance panel cover (top) of the cuspidor unit. Pick up the vacuum and saliva ejector handpieces from the assistant holder and insert them into the connector.

[Reference] Procedure for washing with cleaner [page 171](2) Press the switch.

* Before pressing the switch, make sure to insert the vacuum and saliva ejector handpieces into the connector.

The screen does not return to the home screen even when the $ilde{a}$ switch is pressed.

* To stop washing the vacuum line in progress, press the switch.

* If the x switch is pressed during washing, the screen shown in the left figure appears.

Press the **OK** switch.

* After the suction starts, return the vacuum and saliva ejector handpieces to the assistant holder.

When the vacuum and saliva ejector handpieces are returned to the assistant holder, the screen returns to the home screen.

| Rinse the vacuum line | 5 |
|--|---|
| Remove the chip/nozzle of the handpieces for vacuum/saliva ejector and insert the handpieces for vacuum/saliva ejector to each connector. *1 Cover the connector when handpiece for vacuum/saliva is not inserted. | |
| Flushing completes first, then washing vacuum line follows. | |

| Rinse | 5 | |
|---------|--------------|--|
| Rinsing | | |
| | about 4 min. | |
| | × | |

| Mainse the vacuum line | | 5 |
|---|----------|-----|
| Rinsing has been canceled. System reset is in process. | | |
| Wait, do not disconnect the handpiece for va ejector from the connector. | cuum/sal | iva |
| | | |
| | | |

| | _ |
|---|-------|
| Rinse the vacuum line | |
| Rinsing has been canceled. | 1 |
| Pressing OK button starts the suction. Return the handpieces for vacuum/saliva ejector to the assistant holder. | THE P |
| | ОК |

| Rinse the vacuum line | |
|---|----|
| Rinsing is complete. | 1 |
| Pressing OK button starts the suction. Return the handpieces for vacuum/saliva ejector to the assistant holder. | |
| | ОК |

3. After rinsing of the vacuum line is completed, press the switch. OK

Rinse the vacuum line Rinsing is complete. Pressing OK button starts the suction. Return the handpieces for vacuum/saliva ejector to the assistant holder.

* After the suction starts, return the vacuum and saliva ejector handpieces to the assistant holder.

When the vacuum and saliva ejector handpieces are returned to the assistant holder, the screen returns to the home screen.

After the suction starts, remove the vacuum and saliva ejector handpieces from the connector.

Dripping of the cleaner can be prevented.

If the cleaner drips from the connector, wipe it off immediately. Places where the cleaner adheres may become discolored.



4. In the event of an error state, take appropriate measures. In the event of an error state before/upon/during washing with cleaner/rinsing, the system will be reset and the error screen will appear.

"Message on the touch panel and measures to be taken" Refer to pages 218 to 220 and take measures in accordance with the remedies given.

5. Press the ОК switch.

* After the suction starts, return the vacuum and saliva ejector handpieces to the assistant holder.

> When the vacuum and saliva ejector handpieces are returned to the assistant holder, the screen returns to the home screen.


🕅 E44:No water in the bottle Rinsing has been canceled. Refer to the instructions for use. Pressing OK button starts the suction

E44: No water in the bottle

ejector from the connector

e.a.

Return the handpieces for vacuum/saliva ejector to the assistant holder. OK

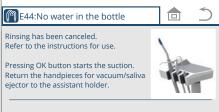



163



Mash the vacuum line with cleaner

i Orotol plus Refill the cleaner

Remove the chip/nozzle of the handpieces for vacuum/saliva ejector and insert the handpieces for vacuum/saliva ejector to each connector. *1 Cover the connector when handpiece for

vacuum/saliva is not inserted. *2 Do not use cleaner except the designated one

Washing the vacuum line with cleaner

1. Press the 'Wash the vacuum line with cleaner' switch.

* If the amount of cleaner is insufficient, the screen shown in the left figure appears. Refill the cleaner. [Reference] How to refill the cleaner [pages 172]

2. Start washing the vacuum line with cleaner.

① Open the maintenance panel cover (top) of the cuspidor unit. Pick up the vacuum and saliva ejector handpieces from the assistant holder and insert them into the connector. [Reference] Procedure for washing with cleaner [page 171] (At this time, the suction is not activated.)

🕅 Wash the vacuum line with cleaner

2 Press the switch.

* Before pressing the base of switch, make sure to insert the vacuum and saliva ejector handpieces into the connector.


The screen does not return to the home screen even when the switch is pressed.

* To stop washing the vacuum line in progress, press the × switch.



×

6 Setting/Adjustment

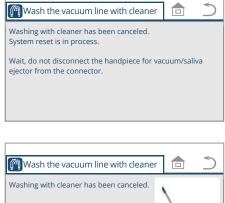

Pressing OK button starts the suction. Return the handpieces for vacuum/saliva ejector to the assistant holder.

| Mash the vacuum line with cleaner |
|---|
| Washing with cleaner has been canceled. |
| Pressing OK button starts the suction. Return the handpieces for vacuum/saliva ejector to the assistant holder. |
| |

OK


* If the x switch is pressed during washing, the screen shown in the left figure appears.

If the handpiece is removed from the connector while the screen on the left is displayed, the cleaner will be left inside the handpiece and handpiece hose. Leaving the cleaner standing may cause a bacterial growth.

Press the **OK** switch.

* After the suction starts, return the vacuum and saliva ejector handpieces to the assistant holder.

When the vacuum and saliva ejector handpieces are returned to the assistant holder, the screen returns to the home screen.

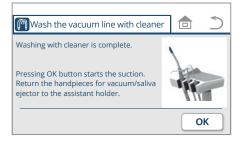

| Wash the vacuum line with cleaner | r | | 5 |
|---|---|-----|---|
| Washing with cleaner is complete. | 1 | | |
| Pressing OK button starts the suction. Return the handpieces for vacuum/saliva ejector to the assistant holder. | í | THE | |
| ejector to the assistant holder. | | 17 | 1 |

3. When you have finished washing the vacuum line with cleaner, press the $o\kappa$ switch.

* After the suction starts, return the vacuum and saliva ejector handpieces to the assistant holder.

When the vacuum and saliva ejector handpieces are returned to the assistant holder, the screen returns to the home screen.

After the suction starts, remove the vacuum and saliva ejector handpieces from the connector.

Dripping of the cleaner can be prevented.


If the cleaner drips from the connector, wipe it off immediately. Places where the cleaner adheres may become discolored.

| 🖗 E44:No water in the bottle | | \mathbf{i} | |
|---|--|--------------|--|
| Washing with cleaner has been canceled. System reset is in process. | | | |
| Wait, do not disconnect the handpiece for vacuum/saliva ejector from the connector. | | | |

4. In the event of an error state, take appropriate measures. In the event of an error state before/upon/during washing with cleaner/rinsing, the system will be reset and the error screen will appear.

"Message on the touch panel and measures to be taken" Refer to pages 218 to 220 and take measures in accordance with the remedies given.

| e.g. | | | | | |
|------|----|-------|----|-----|--------|
| E44: | No | water | in | the | bottle |

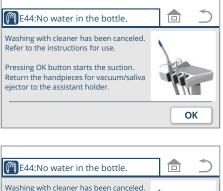

5. Press the

ОК

switch.

* After the suction starts, return the vacuum and saliva ejector handpieces to the assistant holder.

When the vacuum and saliva ejector handpieces are returned to the assistant holder, the screen returns to the home screen.

Washing with cleaner has been canceled. Refer to the instructions for use. Pressing OK button starts the suction. Return the handpieces for vacuum/saliva ejector to the assistant holder. TI

| Patient Adult | | <u>م</u> | lser |
|------------------|--------|----------|------|
| | Ŭ | ۲ ۵ | |
| Manual C | Heater | ∭ High | °&) |

| Ö Fund | ction setting | 5 |
|--------|-----------------------------------|---|
| | Flushing, Washing the vacuum line | |
| | Spittoon valve cleaning | ~ |
| | Handpiece | |
| | Language | V |

Flushing and rinsing the vacuum line

1. Press the function switch 🙆 . The Function setting screen appears.

2. Press the 'Flushing, Washing the vacuum line' switch.

| (¶″ Flu | shing, Washing the vacuum line ? | 5 |
|---------|--------------------------------------|---|
| | Flushing | |
| | Washing the vacuum line | |
| | Flushing and washing the vacuum line | |
| | | |

- 3. Press the 'Flushing and washing the vacuum line' switch.
- * To display the explanation of flushing and washing the vacuum line, press the ? switch (refer to page 157).

| | | _ | | |
|--|---------------|---|---|--|
| Flushing | 严茴٢칠 10:00 | | 5 | |
| Place a cup on the cu | pfiller base. | | | |
| Place handpieces with the head down into the cuspidor bowl. | | | | |
| | | 4 | | |
| | | 0 | ж | |

- 4. Pick up the handpiece you want to flush, and set it in the cuspidor bowl.
 - The number of the handpiece you have picked up appears. (Handpieces are numbered 1, 2, 3 and 4 from front left.)

The screen does not return to the Home screen even when the \bigcirc switch is pressed.

Always flush the handpiece with the main body of the ultrasonic scaler attached. Flushing without it may cause a malfunction.

5. Press the **OK** switch.



6. Open the front panel of the cuspidor unit. Pick up the vacuum and saliva ejector handpieces from the assistant holder and insert them into the connector.

Press the **b** switch.

Flushing and rinsing of the vacuum line will start.

* Before pressing the switch, make sure to insert the vacuum and saliva ejector handpieces into the connector.

The screen does not return to the home screen even when the $\ensuremath{\widehat{}}$ switch is pressed.

 Flushing
 ⇐ 茴 ⊑ 10:00
 ♪
 The

 Flushing in progress
 for
 10:00
 ↓

 09:30
 ↓
 ↓
 ↓

 05:00
 ↓
 ↓
 ↓

 09:30
 ↓
 ↓
 ↓

 0
 ↓
 ↓
 ↓

 0
 ↓
 ↓
 ↓

 0
 ↓
 ↓
 ↓

 0
 ↓
 ↓
 ↓

 0
 ↓
 ↓
 ↓

 0
 ↓
 ↓
 ↓

 0
 ↓
 ↓
 ↓

 0
 ↓
 ↓
 ↓

 0
 ↓
 ↓
 ↓

 0
 ↓
 ↓
 ↓

 0
 ↓
 ↓
 ↓

 0
 ↓
 ↓
 ↓

 0
 ↓
 ↓
 ↓

 0
 ↓
 ↓
 ↓

 0
 ↓
 ↓
 ↓

 0
 ↓
 ↓
 ↓

 0
 ↓
 ↓
 ↓

 ↓
 ↓
 ↓

Remaining time (e.g. 9 min. 30 sec.)

Status of handpiece flushing


Water flows out of the handpiece for five minutes. Then, water flows out of the cupfiller nozzle and the bowl flush nozzle for five minutes.

If the cuspidor bowl is not equipped, rinsing the vacuum line starts 5 minutes after flushing the water circuits of handpieces.

If you need to start rinsing the vacuum line immediately after flushing the water circuits of handpieces, press then press . Rinsing the vacuum line starts. [Refer to page 161]

The screen does not return to the previous screen even when the \supset switch is pressed.

* To suspend flushing, press the switch on the touch panel, the membrane switch, or either switch on the assistant panel, or step on the foot controller pedal.

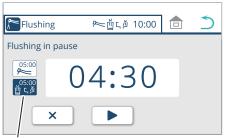

Cupfiller and bowl flush status

* To resume flushing, press the **>** switch.

* To terminate flushing prematurely after pressing , press /membrane switch/assistant operation panel switch, or step on the foot controller.

* To terminate flushing prematurely without pressing \square , press the membrane switch/assistant operation panel switch, or step on the foot controller.

Flushing

Flushing is completed. Return the handpieces to the handpiece holder.

p = 1

ም──□」「□ 10:00

To display the screen in step 6, press the \supset switch.

To display the screen shown in the left figure, press the \mathbf{x} switch.

When flushing is completed, rinsing of the vacuum line will start.

* To stop rinsing the vacuum line in progress, press the

| M Rinse t | he vacuum line 1 | | 5 |
|--|---|------|--------|
| Rinsing *Flushing is returned to | completed. The cleaned handping the holder. | eces | can be |
| | about 4 min. | | |
| | × | | |

 ρ^2 ρ^3

| Rinse the vacu | um line | | \supset |
|---|--------------|--|-----------|
| Rinsing has been cano System reset is in pro- | | | |
| Wait, do not disconnect the handpiece for vacuum/saliva ejector from the connector. | | | |
| | | | |
| | | | |
| Flushing | ஊ∐்டி∄ 10:00 | | 5 |

Flushing is completed.

Return the handpieces to the handpiece holder.

* If the x switch is pressed during rinsing, the screen shown in the left figure appears.

Press the **ok** switch.

switch.

×

* After the suction starts, return the vacuum and saliva ejector handpieces to the assistant holder.

When the vacuum and saliva ejector handpieces are returned to the assistant holder, the screen returns to the home screen.



ок

 \mathbb{A}^{1} \mathbb{A}^{2} \mathbb{A}^{3}

| Flushing | ⅇ₫ւ⊉ | 10:00 | | 5 |
|---|--------|-------|---|---|
| Flushing is completed. | | J | | _ |
| Return the handpieces handpiece handpiece holder. | to the | - | | |
| | 2 | 3 | 4 | |

 p_1^2 p_2^3 p_4^4

OK

Flushing

Place a cup on the cupfiller base.

Place handpieces with the head down into the cuspidor bowl.

7. After rinsing of the vacuum line is completed, return the handpieces for doctor unit to the instrument holder.

After flushing, if the handpieces are returned to the instrument holder while the vacuum line is being rinsed, the screen shown in the left figure does not appear.

* Water is not automatically discharged from the syringe. Press the W lever/button of the syringe for 40 seconds to discharge.

To display the screen shown in the left figure, press the \supset switch.

Press the oκ switch. The screen in step 6 appears.

To display the screen in step 3, press the \bigcirc switch.

8. Press the **ok** switch.

Rinse the vacuum line

Rinsing is complete.

Pressing OK button starts the suction.
Return the handpieces for vacuum/saliva
ejector to the assistant holder.

OK



* After the suction starts, return the vacuum and saliva ejector handpieces to the assistant holder.

When the vacuum and saliva ejector handpieces are returned to the assistant holder, the screen returns to the home screen.

* If the handpieces for doctor unit are returned after the vacuum and saliva ejector handpieces, the home screen appears instead of the screen in step 8.

After the suction starts, remove the vacuum and saliva ejector handpieces from the connector.

Dripping of the cleaner can be prevented.

If the cleaner drips from the connector, wipe it off immediately. Places where the cleaner adheres may become discolored.

| 🖗 E44:No water in the bottle | | 5 |
|--|----------|-----|
| Rinsing has been canceled. System reset is in process. | | |
| Wait, do not disconnect the handpiece for vac ejector from the connector. | :uum/sal | iva |
| | | |

9. In the event of an error state, take appropriate measures. In the event of an error state before/upon/during washing with cleaner/rinsing, the system will be reset and the error screen will appear.

"Message on the touch panel and measures to be taken" Refer to pages 218 to 220 and take measures in accordance with the remedies given.

e.g. E44: No water in the bottle


| E44:No water in the bottle | | J | 5 |
|---|---|---|--------------|
| Rinsing has been canceled. Refer to the instructions for use. | 1 | | |
| Pressing OK button starts the suction. Return the handpieces for vacuum/saliva ejector to the assistant holder. | 4 | | |
| | (| 0 | к |
| | | | |
| | | | |
| 🕅 E44:No water in the bottle | | J | \mathbf{i} |
| Rinsing has been canceled. | | | |

10. Press the ОК switch.

* After the suction starts, return the vacuum and saliva ejector handpieces to the assistant holder.

When the vacuum and saliva ejector handpieces are returned to the assistant holder, the screen returns to the home screen.

Refer to the instructions for use. Pressing OK button starts the suction. Return the handpieces for vacuum/saliva ejector to the assistant holder.

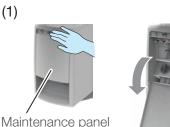


Maintenance pane cover (top)



Procedures for washing with cleaner

(1) Open the maintenance panel cover (top) of the cuspidor unit.

(2) Pick up the vacuum and saliva ejector handpieces from the assistant holder. Remove the vacuum and saliva ejector chips.

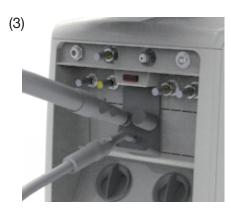

(3) Insert (2) into the connector.

A vacuum connector cap (hereinafter, 'connector cap') should be attached to a connector that is not being used.

Do not remove the connector cap attached to a connector that has no function or a connector that is not being used. Its removal weakens the suction and may hinder suction capacity. The connector cap may be lost when removed.



* Do not remove the solid collector while washing. Without the solid collector, the suction does not work.

If the solid collector is removed during washing, the washing will not be performed and water may burst forth.


* Do not remove the handpiece from the connector during washing. An error may occur.

The washing may take longer than the indicated time depending on the vacuum capacity of the machine room.




Do not pour any cleaner except our designated cleaner into the cleaner bottle. The bottle can hold up to 500 ml of cleaner, enough for approximately 25 washes.

Place the cap on the remaining bottle. Use the bottle for replacement. Use the cleaner remaining in the replaced bottle when refilling the cleaner the next time.

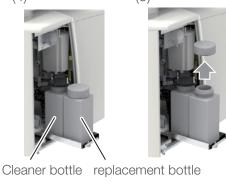

(6)

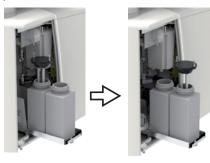

(7)



User Patient R N Adult ٢Ī١ Й \odot F. 📈 High 🔊 🔊 💷 Manual [∭] Heater ☐ OFF Î Distance in the setting Function setting Flushing, Washing the vacuum line \wedge Spittoon valve cleaning

Handpiece

Language

Drive air pressure

Handpiece light activation Micromotor light brightness

Spray mode

🚝 Handpiece

6-2-4 Displaying the drive air pressure

Display the handpiece drive air pressure.

1. Press the function switch . The Function setting screen appears.

2. Press 'Handpiece.'

3. Press 'Drive air pressure.'The Drive air pressure screen appears.

 \mathbf{V}

| Prive air pressure | |) |
|---------------------------------------|--------------------|---|
| Pick up handpiece. | | |
| Step on the pedal of Foot controller. | | |
| | 130 _{kPa} | |

4. Pick up the handpiece to measure the pressure from the instrument holder.

5. Step on the pedal of the foot controller. The air pressure is displayed.

During the pedal of foot controller being stepped on, pressing will not bring the mode to home screen; pressing \bigcirc will not bring the mode to the previous screen.

6. After checking the pressure, return the handpiece to the instrument holder.

7. Press the \bigcirc switch to display the Home screen.

6-2-5 Setting the spray mode

Two mode: Spray ON/OFF

Four mode: Air and water ON/OFF

* The ultrasonic scaler discharges water regardless of whether in two mode or four mode.

1. Press the function switch 🖄 . The Function setting screen appears.



| Ö Fu | nction setting | 5 |
|------|-----------------------------------|---|
| | Flushing, Washing the vacuum line | |
| | Spittoon valve cleaning | |
| | Handpiece | |
| | Language | |

- Handpiece
 Drive air pressure
 Spray mode
 Handpiece light activation
 Micromotor light brightness
- 2. Press 'Handpiece.'

3. Press 'Spray mode.' The Spray Mode setting screen appears.

- 4. Pick up the handpiece you want to set.
 - 5. Select 'Two mode' or 'Four mode.'
 - 6. Press **Ο**κ to save the setting. The previous screen appears.
 - * To cancel the setting, press \bigcirc without pressing $\bigcirc \kappa$
 - 7. Return the handpiece to the instrument holder.
 - 8. Press the 💼 switch to display the Home screen.

6-2-6 Setting the light activation timing

Two types of light activation timing can be selected.

'When picking up a handpiece'

Turns on when the handpiece is picked up from the instrument holder and will turn off when returned to the holder.

* For NO PAIN, the light turns off after the afterglow period is completed. 'When stepping on the foot controller'

Turns on while the handpiece is activated and off after the after-glow period (approx. 5 seconds) is completed after the handpiece deactivates.

- * For NO PAIN, the light turns off after the afterglow period is completed.
- * Contact your local authorized Belmont dealer when the setting of afterglow period for NEWTRON/ VARIOS170 should be changed. Setting of afterglow period for NO PAIN cannot be changed.
- 1. Press the function switch 😳.

The Function setting screen appears.

R Π Adult ٢Ň сŬ Manual Ъ [∭] Heater ☐ OFF High 🗞

User

Patient

- Function setting Flushing, Washing the vacuum line Spittoon valve cleaning Handpiece Language
- 2. Press 'Handpiece.'

3. Press 'Handpiece light activation.' The Handpiece light activation setting screen appears.


4. Select 'When picking up a handpiece' or 'When stepping on the foot controller.'

ОК 5. Press to save the setting. The previous screen appears.

* To cancel the setting, press Ⴢ without pressing 🗌 ок

6. Press the 🚖 switch to display the Home screen.



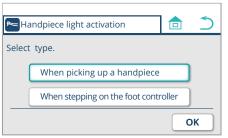

Patient User Å Π Adult ٢Ā Д (デー・Manual ジージー 📈 High °&) 💷 Ð Heater 5 🔅 Function setting Flushing, Washing the vacuum line \wedge Spittoon valve cleaning

6-2-7 Setting the micromotor brightness

Handpiece light brightness (illuminance) can be changed.

NLX plus

1. Press the function switch 🙆 . The Function setting screen appears.

2. Press 'Handpiece.'

3. Press 'Micromotor light brightness.' The micromotor light brightness setting screen appears.

| М На | ndpiece | Ć |
|-------------|-----------------------------|---|
| (| Drive air pressure | |
| | Spray mode | |
| (| Handpiece light activation | |
| | Micromotor light brightness | |

Handpiece Language

RUser A Micromotor light brightness \mathbb{D} Pick up handpiece. <u>1</u> Λ

4. Pick up the handpiece to set.

 $\mathbf{\vee}$ ____ switches to adjust the bright-5. Press the ness. Adjust the brightness while checking actual brightness.

- * It is not necessary to press [**ok**] to save this setting. It is memorized when the brightness (illuminance) is set.
- 6. Press the 💼 switch to return to the Home screen.

6 Setting/Adjustment



MX2/MCX 1. Press the function switch 🔯 . The Function setting screen appears.

| 🔅 Fu | nction setting | 5 |
|------|-----------------------------------|--------|
| (| Flushing, Washing the vacuum line | |
| (| Spittoon valve cleaning | \sim |
| | Handpiece | |
| | Language | V |

2. Press 'Handpiece.'

3. Press 'Micromotor light brightness.' The micromotor light brightness setting screen appears.

| Ma Ha | indpiece | 5 |
|-------|-----------------------------|---|
| (| Drive air pressure | |
| (| Spray mode | |
| (| Handpiece light activation | |
| (| Micromotor light brightness | |

| Micromotor light brightness | | ∖User A |
|--|----|-----------|
| Pick up a handpiece. Select the memory number and the level of brightness. | 10 | 1 5 °0 |
| M1 V | | ^ |
| | | ок |

4. Pick up the handpiece you want to set.


5. Press the memory switch M1 to select [M1], [M2], [M3], or [M4]. These four switches appear sequentially every time the [M1] switch is pressed.

6. Press the v switches to adjust the brightness between [0] and [15]. The brightness can be adjusted by [1] increment.

7. Press or to save the setting. The previous screen appears.

* To cancel the setting, press \bigcirc without pressing \bigcirc .

8. Press the 💼 switch to return to the Home screen.

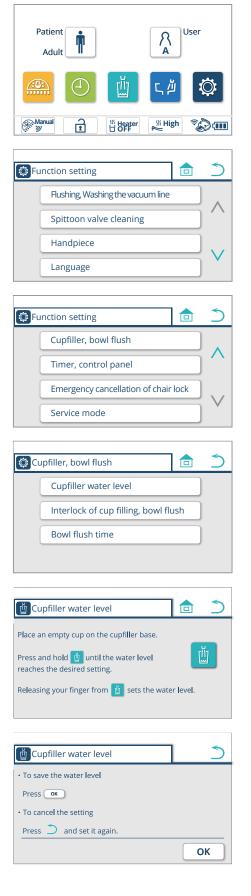

6-2-8 Setting the cupfiller water level

The cupfiller water level setting is set on shipment from the factory but may be adjusted as required.

The cupfiller water level and filling duration when using the cupfiller switch (forced cup-filling) can be adjusted.

1. Press the function switch 🔯 .

The Function setting screen appears.

2. Press the \lor switch.

The second page of the Function setting screen appears.

3. Press 'Cupfiller, bowl flush.'

4. Press 'Cupfiller water level.' The Cupfiller water level setting screen appears.

5. Place an empty cup on the cupfiller base, and press and hold the 🖄 switch to fill the cup to the desired level.

* Water is supplied while the <u>u</u> switch is pressed and stops when it is released.

- * To memorize the cupfiller water level, press OK. After pressing OK, the previous screen appears.
- * To cancel the setting, press ⊃ without pressing ок. After pressing ⊃, the previous screen appears.

6. Press the 💼 switch to return to the Home screen.

If the water supplied exceeds the desired water level, set it again from the beginning.

To cancel the setting in the middle, press \bigcirc or \boxdot .

6-2-9 Setting the cup-filling/bowl flush interlock

The following two modes can be selected: 'Interlock'

Cuspidor bowl flush starts at the same time as cup-filling. 'No interlock'

The cuspidor bowl is not flushed even when cup-filling is started.

1. Press the function switch O .

The Function setting screen appears.

2. Press the \lor switch. The second page of the Function setting screen appears.

3. Press 'Cupfiller, bowl flush.'

- 4. Press 'Interlock of cup filling, bowl flush.' The Interlock of cup filling, bowl flush screen appears.
- 5. Select 'Interlock' or 'No interlock.'

6. Press or to save the setting. The previous screen appears.

- * To cancel the setting, press \bigcirc without pressing \bigcirc
- 7. Press the \bigcirc switch to return to the Home screen.



| 🔅 Fu | nction setting | 5 |
|------|-----------------------------------|----------|
| (| Flushing, Washing the vacuum line | |
| | Spittoon valve cleaning | \wedge |
| | Handpiece | ~ / |
| | Language | V |
| | | |

| 🔅 Fu | nction setting | 5 |
|------|--------------------------------------|---|
| (| Cupfiller, bowl flush | |
| | Timer, control panel | |
| | Emergency cancellation of chair lock | |
| | Service mode | V |

| Ö Cu | pfiller, bowl flush | 1 | 5 |
|------|--------------------------------------|---|---|
| (| Cupfiller water level | | |
| | Interlock of cup filling, bowl flush | | |
| | Bowl flush time | | |
| | | | |

| 🚺 In | terlock of cup filling/bowl flush | | 5 |
|--------|-----------------------------------|---|---|
| Select | type. | | |
| | Interlock | | |
| | No interlock | | |
| | | C | ж |

Patient

🔅 Function setting

Cupfiller, bowl flush

Timer, control panel

Service mode

6-2-10 Setting the bowl flush time

Duration of the bowl flush can be set.

Å 1 Adult ٢Ň Manual F Heater 🖉 High **~**@ Function setting Flushing, Washing the vacuum line \wedge Spittoon valve cleaning Handpiece Language

User

 \bigcirc

Λ

1. Press the function switch $\textcircled{\circ}$. The Function setting screen appears.

- 2. Press the \vee switch. The second page of the Function setting screen appears.
- 3. Press 'Cupfiller, bowl flush.'

🔅 Cupfiller, bowl flush Cupfiller water level Interlock of cup filling, bowl flush Bowl flush time

Emergency cancellation of chair lock

| て ^資 Bowl flush time | Ć 🗈 |
|--------------------------------|--------------------|
| Set the duration time. | 6.0s ¹⁵ |
| | |
| | ОК |

- 4. Press 'Bowl flush time.' The Bowl Flush Time setting screen appears.
- 5. Press the 🗸 _____ switches to set the time between [0] and [15] seconds.
- 6. Press or losave the setting. The previous screen appears.
- * To cancel the setting, press \bigcirc without pressing $\bigcirc \kappa$
- 7. Press the 💼 switch to return to the Home screen.

6-2-11 Setting the language

Language displayed on the touch panel can be changed.

1. Press the function switch 🍄 . The Function setting screen appears. User Patient R Ň Adult ٢Ň こ 凶 \odot 📈 High 🗞 💷 (深い Manual ジ Ð [∭] Heater □ OFF 🔅 Function setting 2. Press 'Language.' The Language setting screen appears. Flushing, Washing the vacuum line \wedge Spittoon valve cleaning Handpiece \vee Language 3. Press the switches to select your language. \sim \wedge Language Select type. Japanese 4. Press ок to save the setting. The previous screen appears. \sim \wedge * To cancel the setting, press \bigcirc without pressing $\bigcirc \kappa$ 1. ок 5. Press the a switch to return to the Home screen.

6-2-12 Setting the timer alarm tone

Alarm tone emitted when the timer reaches zero can be set. Five tones are available and can be set to distinguish between different chairs in the same room.

1. Press the function switch 🔅 .

The Function setting screen appears.

| Ö Fu | nction setting | 5 |
|------|-----------------------------------|----------|
| | Flushing, Washing the vacuum line | |
| | Spittoon valve cleaning | \wedge |
| (| Handpiece | |
| (| Language | • |

🔅 Function setting

Cupfiller, bowl flush Timer, control panel

Service mode

2. Press the \lor switch. The second page of the Function setting screen appears.

3. Press 'Timer, control panel.'

| Ö Tii | ner, control panel | 5 |
|-------|-----------------------|---|
| | Timer alarm selection | |
| | Key touch volume | |
| | Sleep mode time | |

Emergency cancellation of chair lock

| 4. Press 'Timer alarm selection.' |
|---|
| The Timer Alarm Selection setting screen appears. |

| Timer alarm selection | <u></u> |
|-----------------------|------------|
| Select type. | ♪ 3 |
| | v ^ |
| | ОК |

5. Press the \checkmark \land switches to select a tone from five types (' 1' to ' 5').

6. Press or to save the setting. The previous screen appears.

* To cancel the setting, press \bigcirc without pressing $\bigcirc \kappa$.

7. Press the \bigcirc switch to return to the Home screen.

| | The tone emitted when a switch is pressed can be set. |
|---|---|
| Patient Adult Adult Muser Adult Muser Company Compa | 1. Press the function switch 🙆 . The Function setting screen appears. |
| Function setting Flushing, Washing the vacuum line Spittoon valve cleaning Handpiece Language | 2. Press the ∨ switch. The second page of the Function setting screen appears. |
| Function setting Cupfiller, bowl flush Timer, control panel Emergency cancellation of chair lock Service mode | 3. Press 'Timer, control panel.' |
| Timer, control panel Timer alarm selection Key touch volume Sleep mode time | 4. Press 'Key touch volume.' The Key touch volume setting screen appears. |
| Key touch volume | 5. Press touch bar graph ↓ to set the volume in five levels ('1' to '5'). The ♦ switch makes the sound of the tone. The ♦ switch mutes the tone. To mute the tone, press ♦ and the display changes to ♦ . 6. Press • K to save the setting. The previous screen appears. * To cancel the setting, press • without pressing • K . |
| | |

6-2-13 Setting the key touch volume

The tone emitted when a switch is pressed can be set.

7. Press the \Box switch to return to the Home screen.

Patient

Adult

Manual

'n

Ъ

6-2-14 Setting the sleep mode time

When no switch on the control panel is used for a while, the touch panel screen goes dark and enters the sleep mode. The time before entering sleep mode can be adjusted.

1. Press the function switch $\textcircled{\circ}$.

The Function setting screen appears.

 Function setting
 Image

 Hushing, Washing the vacuum line
 Image

 Spittoon valve cleaning
 Image

 Handpiece
 Image

ſŪ

Heater

 Function setting
 Image: Cupfiller, bowl flush

 Cupfiller, bowl flush
 Image: Cupfiller, bowl flush

 Timer, control panel
 Image: Cupfiller, bowl flush

 Emergency cancellation of chair lock
 Image: Cupfiller, bowl flush

 Service mode
 Image: Cupfiller, bowl flush

Timer alarm selection Key touch volume Sleep mode time

Service mode

User

°20-00

Å

Jigh 🖉

The second page of the Function setting screen appears.

3. Press 'Timer, control panel.'

2. Press the \bigvee switch.

4. Press 'Sleep mode time.' The Sleep mode time setting screen appears.

| Sleep mode time | <u> </u> |
|-----------------|--|
| Set the timer. | 10 min ¹⁵ ₅ |
| | |
| | ОК |

5. Press the v switches to select one of three durations ('5 min,' '10 min,' and '15 min').

6. Press oκ to save the setting. The previous screen appears.

* To cancel the setting, press \bigcirc without pressing \bigcirc .

- 7. Press the \bigcirc switch to return to the Home screen.
- * To cancel the sleep mode, touch the screen.

198

Patient

Adult

Manual

Π

Ð

6-2-15 Setting emergency cancellation of the chair lock

The chair lock function, when activated, can be temporarily cancelled. Use this temporary cancellation only when the chair lock function must be disabled to resolve a malfunction.

1. Press the function switch 🔯 .

The Function setting screen appears.

| 🔅 Fu | nction setting | Ć |
|------|-----------------------------------|---|
| (| Flushing, Washing the vacuum line | |
| (| Spittoon valve cleaning | |
| (| Handpiece | |
| (| Language | V |

٢Ň

[∭] Heater □ OFF User

🖉 High 🗞 💷

 \wedge

ОК

Interference with rear link cover

R

гЙ

| 🔅 Fu | nction setting | 5 |
|------|--------------------------------------|--------------|
| | Cupfiller, bowl flush | |
| | Timer, control panel | |
| | Emergency cancellation of chair lock | \mathbf{N} |
| | Service mode | V |

Emergency cancellation of chair lock

Select the cause of chair lock.

2. Press the ∨ switch. The second page of the Function setting screen appears.

3. Press 'Emergency cancellation of chair lock.' The Emergency cancellation of chair lock setting screen appears.

4. Press the <u>switches to cancel the chair lock</u>. [Reference] Chair lock function[page 104 to 106]

5. Press the oκ switch to save the setting. The previous screen appears.

- * To cancel the setting, press \supset without pressing \bigcirc .
- 6. Press the 🖻 switch to return to the Home screen.


* When the main switch is turned off, emergency cancellation of the chair lock will be disabled, and the chair lock function will be reactivated.



6-3 Adjusting the quantity of water/air supplied from the doctor unit

Adjusting the quantity of water /air supplied to the syringe spray The quantity of water and air supplied from the doctor unit can be adjusted with the knobs (1).

Blue cap (W side): Quantity of water supplied Yellow cap (A side): Quantity of air supplied

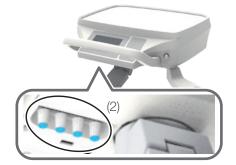

Adjusting the quantity of water supplied to the handpiece spray The quantity of water supplied to the handpiece spray from the doctor unit can be adjusted with the knobs (2) (with blue caps).

Up

Holder



Place holder



All the knobs decrease the flow rate when turned clockwise and increase it when turned counterclockwise.

Do not close the adjusting knob too tightly. It is used to increase or decrease flow rate and is not a stop valve. Note that it will idle if you close it too tightly.

* Correspondence between a holder and its adjusting knob (2) depends on the position of the syringe.

(See the figure on the left) Operate the system and notice the correspondence between the adjusting knob Nos. and the handpiece.

Adjusting the quantity of water supplied to the ultrasonic scaler The quantity of water supplied to the ultrasonic scaler can be adjusted with the knob (3).

* The knob (3) is only installed on systems with an ultrasonic scaler.

Fully open the adjusting knob (2) for the ultrasonic scaler when using it.



Maintenance panel cover (top)





6-4 Adjusting the quantity of water/air supplied to the cuspidor unit

Maintenance panel cover Press the maintenance panel cover (top) to open it.

(1) Water supply control knob for the syringe Adjust the quantity of water supplied to the syringe.

(2) Air supply control knob for the syringe Adjust the quantity of air supplied to the syringe.

(3) Water supply control knob for the water service coupler Adjust the quantity of water supplied from the water service coupler.

(4) Bowl flush adjusting knob

(5) Cupfiller adjusting knob*

* Knobs (5) are provided for maintenance. They have already been adjusted to ensure the optimum water supply on installation.

The water level can be adjusted as required. [Reference] Setting the cupfiller water level [page 178]

All the knobs decrease the flow rate when turned clockwise and increase it when turned counterclockwise.

Do not close the adjusting knob too tightly. It is used to increase or decrease flow rate and is not a stop valve. Note that it will idle if you close it too tightly.

7-1 Exterior

7-1-1 Cleaning and disinfecting the surfaces

To clean and disinfect the exterior of the product (except the touch panel), wipe the surface with a soft cloth or paper towel moistened with FD366 manufactured by Dürr, and then wipe it with a dry cloth.

If the exterior is excessively dirty, moisten a soft cloth with water containing approx. 10% of neutral detergent, and wipe the exterior with the cloth. Then, wipe it with a cloth moistened with water and dry it completely with a dry cloth.

CAUTION

To clean and disinfect the touch panel, use ethanol for disinfection. Using other disinfectant may lead to incorrect operation, resulting in injury.

Never use any of the products listed below:

Volatile chemicals such as paint thinner, butanol, isopropyl alcohol, nail-varnish remover, gasoline, or kerosene; acid, alkaline or chlorine detergents; highly corrosive disinfectants (povidone-iodine such as lsodine, sodium hypochlorite, etc.); abrasive polishing wax or abrasive sponge

Do not use a brush or scrubber for maintenance. All of these may leave scratches and/or ingrained dirt.

If water or detergent is left on the surface, wipe it off immediately. Moisture or detergent may cause rust or failure of electrical parts.

Do not spray detergent directly onto the exterior.

Clean the surface with a soft cloth or paper towel moistened with detergent, and wipe it dry with a cloth.

If liquid enters the product, it may cause a malfunction or failure.

Confirm that the detergent has completely dried off before activating the product.



7-2 Chair

7-2-1 Leather part

Wipe the surface with a soft cloth or paper towel moistened with FD360 manufactured by Dürr, and then wipe it with a dry cloth.

If it is excessively dirty, moisten a soft cloth with water containing approx. 10% neutral detergent, and wipe it with the cloth. Then, wipe it with a cloth moistened with water and then dry it completely with a dry cloth.

[Reference] Precautions for use of synthetic leather [Page 39]

Do not use organic solvents, detergents containing abrasives, or bleach to clean the leather part.

All of these may deteriorate the surface and change texture or gloss.

Do not spray detergent directly onto the leather part or exterior. Clean the surface with a soft cloth or paper towel moistened with detergent, and wipe it with a dry cloth.

The liquid may enter the product through gaps and cause a malfunction or failure.

Confirm that the detergent has completely dried off before activating the product.






7-3 Doctor unit

7-3-1 Touch panel

Turn off the main switch to to prevent accidental depression of a switch during cleaning or disinfection.

When cleaning or disinfection while powering on the main switch, lock the touch panel of the doctor unit.

(Example) Cleaning the surface of the touch panel during the procedure.

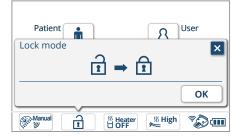

| Lock mode | |
|--------------------------------|--|
| 1 | |
| ЖТо unlock, press and hold 🔊 . | |
| | |

1. Open the [Lock mode] screen, and press ок. . The touch panel is locked.

While it is locked, a sub-window indicating the locked state is displayed on the touch panel.

2. Wipe the touch panel with a soft cloth moistened with detergent.

3. Press and hold the membrane chair/headrest switch s for one second or more.

This will unlock the touch panel operation.

Do not spray detergent directly onto the exterior. Clean the surface with a soft cloth or paper towel moistened with detergent, and wipe it with a dry cloth. If liquid enters the product, it may cause a malfunction or failure.



Be certain to wipe off all water and detergent. Otherwise, it may cause operational failure, malfunction, resulting in injury.

7-3-2 Silicone mat

Autoclave them at a temperature of 134°C for 3 minutes. Or, wipe off the surface with soft cloth moistened with disinfectant FD366 manufactured by Dürr.

Before sterilization, remove dirt on the surface and wash them under running water.

Be sure to put them in a sterilization bag during the autoclave process.

Do not perform dry processing.

Repeating autoclave sterilization may cause degradation or discoloration, but doesn't affect the product functions.

If chemical solutions are spilled on the equipment, immediately wipe them off. Otherwise, discoloration may occur.






Holder type



Handle cover

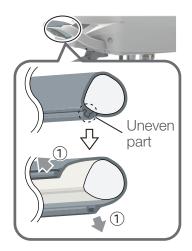



7-3-3 Handle cover

Handle cover is detachable for cleaning.

Wipe it well with a soft cloth moistened with FD366 manufactured by Dürr.

Handle cover can be autoclaved at a temperature of 134°C for 3 minutes.

Before sterilization, remove dirt on the surface and wash it under running water.

Be sure to put it in a sterilization bag during the sterilization process. Do not perform dry processing exceeding the sterilization temperature.

[Method to detach] Holder type

①Firstly, remove the uneven part at the edge of handle cover.②Remove the handle cover.

* Be sure to remove it by hands.

[Method to reattach]

Reattach it in the reverse order of detaching procedures.


* Align the uneven part of the handle cover and then reattach it.

Repeating autoclave sterilization may cause degradation or discoloration, but doesn't affect the product functions.

Do not perform dry processing exceeding the sterilization temperature.

Otherwise, deformation or discoloration may occur.

If chemical solutions are spilled on the equipment, immediately wipe them off. Otherwise, discoloration may occur.

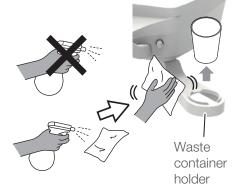

7-3-4 Waste container holder

When the waste container becomes full, replace a paper cup for the waste container holder.

It the holder becomes dirty, wipe it well with a soft cloth moistened with FD366 manufactured by Dürr.

Holder



7-3-5 Instrument holder

Wipe the surface with a soft cloth or paper towel moistened with FD366 manufactured by Dürr, and then wipe it with a dry cloth.

If the exterior is excessively dirty, moisten a soft cloth with water containing approx. 10% neutral detergent, and wipe the exterior with the cloth. Then, wipe it with a cloth moistened with water and then dry it completely with a dry cloth.

Never use any of the products listed below:

Volatile chemicals such as paint thinners, butanol, isopropyl alcohol, nail-varnish remover, gasoline, or kerosene; acid, alkaline, or chlorine detergents; highly-corrosive disinfectants (povidone-iodine such as lsodine, sodium hypochlorite, etc.); abrasive polishing wax, or abrasive sponge

Do not use a brush or scrubber during maintenance. All of these may leave scratches and/or ingrained dirt.


If water or detergent is left on the surface, wipe it off immediately. Moisture or detergent may cause rust or failure of electrical parts.

Do not spray detergent directly onto the leather part or exterior. Clean the surface with a soft cloth or paper towel moistened with detergent, and wipe it with a dry cloth.


If liquid may enter the product through gaps and cause a malfunction or failure.

Confirm that the detergent has completely evaporated before activating the product.



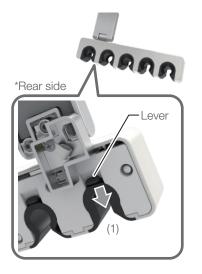

Instrument holder

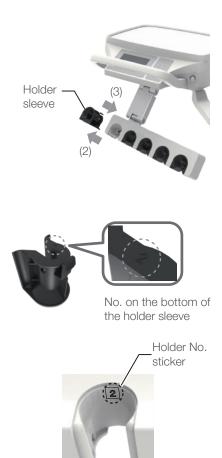

Holder sleeve

The holder sleeves of the instrument holder are detachable for cleaning.

Wipe it well with a soft cloth moistened with FD366 manufactured by Dürr.

Sleeve No. A1 to A3 can be autoclaved at a temperature of 134°C for 3 minutes.

Sleeve No.1 to 4 cannot be autoclaved.

Before sterilization, remove dirt on the surface and wash it under running water.

Be sure to put it in a sterilization bag during the sterilization process. Do not perform dry processing exceeding the sterilization temperature.

Regarding the sleeve No. of each handpiece / syringe, refer to the table on the next page.

[Method to detach/reattach it]

- 1. Pull the lever at the lower section of the holder sleeve toward you (1).
- 2. While pulling the lever, push the holder sleeve upward to remove it (2).
- 3. To reattach it, insert it from upper side until it 'clicks' in (3).

Each holder sleeve has its No. on the bottom, and a sticker showing the same No. is attached to the instrument holder. Make sure you attach the holder sleeve to the holder with the same No.

Make sure you attach the holder sleeves to the holders with the corresponding Nos. (to the original positions) until they 'click' in. A mismatched combination will stop the product from functioning correctly.

Do not autoclave the sleeve No. 1 to 4. Autoclave sterilization may deform or degrade them.

Sleeve No. A1 to A3 can be autoclaved at a temperature of 134°C for 3 minutes.

Repeating autoclave sterilization may cause degradation or discoloration, but doesn't affect the product functions.

Do not perform dry processing exceeding the sterilization temperature.

Otherwise, deformation or discoloration may occur.


If chemical solutions are spilled on the equipment, immediately wipe them off. Otherwise, discoloration may occur.

| Non-sterile | type |
|-------------|------|
|-------------|------|

| Sleeve No. | Applicable handpiece/syringe | |
|------------|------------------------------|--------------------------------------|
| 1 | Syringe | BT14 |
| | | LUZZANI 3WAY (MINI LIGHT) |
| | | LUZZANI 6WAY (MINI LIGHT) |
| 2 | Syringe | 77-type 3WAY |
| | | LUZZANI 3WAY (MINI MATE) |
| | Air turbine | Air turbine |
| | Ultrasonic scaler | NSK VARIOS VS170 SCALER |
| | Air motor | BIEN AIR Aquilon 830 |
| | | UNIFIX with LIGHT |
| | | PM1132 |
| | | NSK EX-203 |
| | | NSK EX-6 |
| 3 | Micromotor | NSK NBX |
| | | NSK NLX plus |
| | | NSK NLX nano |
| | | BIEN AIR MX2 (DMX3) |
| | | BIEN AIR MCX |
| | Ultrasonic scaler | SATELEC Xinetic |
| | | SATELEC SP4055 NEWTRON |
| | | EMS NO PAIN |
| | Curing light | SATELEC MINI LED STD OEM |
| 4 | Syringe | DCI |
| | Ultrasonic scaler | NSK VARIOS VS170 LUX SCALER |
| | | DENTSPLY CAVITRON SCALER (TYPE G139) |
| | | SATELEC SP4055 NEWTRON SLIM |

Sterilizable type

| Sleeve No. | Applicable handpiece/syringe | | |
|------------|------------------------------|--------------------------------------|--|
| A1 | Syringe | 77-type 3WAY | |
| | Air turbine | Air turbine | |
| A2 | Syringe | BT14 | |
| | | LUZZANI 3WAY (MINI LIGHT) | |
| | | LUZZANI 6WAY (MINI LIGHT) | |
| | Micromotor | BIEN AIR MCX | |
| | | BIEN AIR MX2 (DMX3) | |
| | | NSK NBX | |
| | | NSK NLX plus | |
| | | NSK NLX nano | |
| | Ultrasonic scaler | NSK VARIOS VS170 SCALER | |
| | | SATELEC Xinetic | |
| | | SATELEC SP4055 NEWTRON | |
| | | EMS NO PAIN | |
| | Curing light | SATELEC MINI LED STD OEM | |
| A3 | Syringe | DCI | |
| | | LUZZANI 3WAY (MINI MATE) | |
| | Ultrasonic scaler | NSK VARIOS VS170 LUX SCALER | |
| | | DENTSPLY CAVITRON SCALER (TYPE G139) | |
| | | SATELEC SP4055 NEWTRON SLIM | |

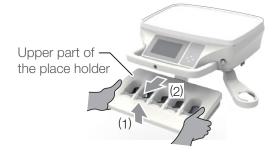




Place holder

Upper part of the place holder is detachable for cleaning.

[Method to detach/reattach it]

- 1. Hold the upper part of the place holder with both hands and pull it upward slightly. (1)
- 2. Pull it toward you and remove it. (2)
- 3. To reattach it, insert it to the original position from the upper side. (3)

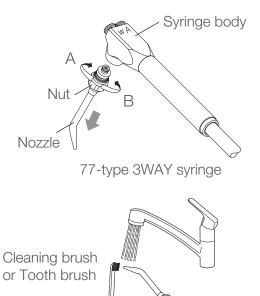

Cleaning 77-type 3WAY syringe Nozzle



Sterilization pouch

7-3-6 Cleaning of 77-type 3WAY syringe

Disassembly

Remove the nozzle from syringe by turning it in direction A. Cleaning by hand

- 1. Wipe off the surface contamination by a cloth while rinsing the surface by running clean warm water at 40±5 °C. Scrub the tip and joint part of nozzle by a cleaningbrush or by a tooth brush with running clean warm water at 40±5 °C.
- 2.Check whether contamination is removed or not after cleaning. Continue the cleaning if contamination is remained.
- 3.Immersed with an alkaline disinfection or detergent for 5 minutes. (We recommend to use ID212 made by Dürr)
- 4. Rinse thoroughly by distilled water at ordinary temperature or by clean water for more than 1 minute.

After sterilizing the handpiece, keep it in the sterilization pouch and store in a dark and cool place.

AUTION

Cleaning must be done every after use to patients.

Cleaning must be done within 1 hour after use.

Replace with a new syringe nozzle if contamination and solid material attached to syringe nozzle can not be removed.

Sterilization

Sterilization must be done every after use to patients.

The nozzle can be sterilized with autoclave. Sterilization with autoclave is permitted up to 250 times.

1.Insert the handpiece in a sterilization pouch and seal it.

2.Autoclave for 3 min. at 134°C and dry for 15 min.

Apply Vaseline thinly and evenly to the two O-rings after sterilization.

Tighten the nut firmly in direction B to undo the nozzle.

Storage:

After sterilizing the pouch and store in a dark and cool place. Recommended cleaner : Orotol Plus made by Dürr.



Sterilization must be done every after use to patients.


Sterilization by class B cycles.

Sterilization temperature is 135°C or less.

Dry naturally if the temperature for drying process is to exceed 135°C.

If damage occurs to the sterilization pouch, discard and sterilize again using a new pouch.

Before use, make sure that the nut is firmly tightened.

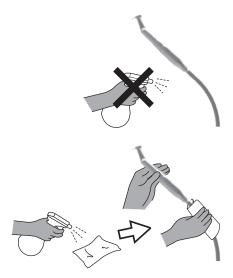


7-3-7 Handpiece hose

When the handpiece becomes dirty, wipe it well with a soft cloth moistened with FD366 manufactured by Dürr.

Do not spray detergent directly onto the exterior.

Clean the surface with a soft cloth or paper towel moistened with detergent, and wipe it with a dry cloth.

If liquid enters the product, it may cause a malfunction or failure. Confirm that the detergent has completely evaporated before activating the product.

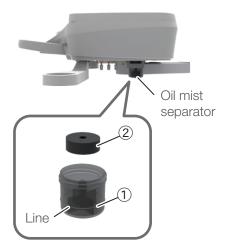

7-3-8 Oil mist separator

This unit collects oil from the exhaust air from the handpiece.

When oil reaches to the line on the oil reservoir (1), make sure you discard the oil.

Turn the oil reservoir counterclockwise to remove.

If the hygroscopic sponge (2) (consumable) is excessively dirty or has excessive oil on it, replace it.

Contact your local authorized Belmont dealer for a replacement for the hygroscopic sponge. (Replacement will be charged for.)





7-4 Cuspidor unit

7-4-1 Assistant holder

Wipe the surface with a soft cloth or paper towel moistened with FD366 manufactured by Dürr, and then wipe it with a dry cloth.

If the exterior is excessively dirty, moisten a soft cloth with water containing approx. 10% neutral detergent, and wipe the exterior with the cloth. Then, wipe it with a cloth moistened with water and then dry it completely with a dry cloth.

Never use any of the products listed below:

Volatile chemicals such as paint thinners, butanol, isopropyl alcohol, nail-varnish remover, gasoline, or kerosene; acid, alkaline, or chlorine detergents; highly-corrosive disinfectants (povidone-iodine such as Isodine, sodium hypochlorite, etc.); abrasive polishing wax or abrasive sponge.

Do not use a brush or scrubber during maintenance. All of these may leave scratches and/or ingrained dirt.

If water or detergent is left on the surface, immediately wipe it off. Any remaining moisture or detergent may cause rust or failure of electric parts.

Do not spray detergent directly onto the exterior. Clean the surface with a soft cloth or paper towel moistened with

detergent, and wipe it with a dry cloth.

If liquid enters the product, it may cause a malfunction or failure. Confirm that the detergent has completely evaporated before activating the product.

Holder sleeve

The holder sleeves of the instrument holder are detachable for cleaning.

Wipe it well with a soft cloth moistened with FD366 manufactured by Dürr.

Sleeve No. A1 to A4 can be autoclaved at a temperature of 134°C for 3 minutes.

Sleeve No.1 to 6 cannot be autoclaved.


Before sterilization, remove dirt on the surface and wash it under running water.

Be sure to put it in a sterilization bag during the sterilization process. Do not perform dry processing exceeding the sterilization temperature.


Regarding the sleeve No. of each handpiece / syringe, refer to the table on the next page.

[Method to detach/reattach it]

- 1. Pull the lever at the lower section of the holder sleeve toward you (1).
- 2. While pulling the lever, push the holder sleeve upward to remove it (2).
- 3. To reattach it, insert it from upper side until it 'clicks' in (3).

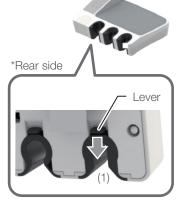

Assistant holder

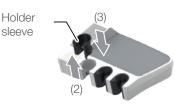



No. on the bottom of the holder sleeve



Each holder sleeve has its No. on the bottom, and a sticker showing the same No. is attached to the assistant holder.

Make sure you attach the holder sleeve to the holder with the same No.

Make sure you attach the holder sleeves to the holders with the corresponding Nos. (to the original positions) until they 'click' in. A mismatched combination will stop the product from functioning correctly.

Do not autoclave the sleeve No. 1 to 6. Autoclave sterilization may deform or discolor them.

Sleeve No. A1 to A4 can be autoclaved at a temperature of 134°C for 3 minutes.

Repeating autoclave sterilization may cause degradation or discoloration, but doesn't affect the product functions.

Do not perform dry processing exceeding the sterilization temperature. Otherwise, deformation or discoloration may occur.


If chemical solutions are spilled on the equipment, immediately wipe them off. Otherwise, discoloration may occur.

Non-sterile type

| Sleeve No. | Applicable handpiece/syringe | | |
|------------|------------------------------|---------------------------|--|
| 1 | Syringe | BT14 | |
| | | LUZZANI 3WAY (MINI LIGHT) | |
| | | LUZZANI 6WAY (MINI LIGHT) | |
| 2 | Syringe | 77-type 3WAY | |
| | | LUZZANI 3WAY (MINI MATE) | |
| 3 | Curing light | SATELEC MINI LED STD OEM | |
| 4 | Syringe | DCI | |
| | Saliva ejector handpiece | DURR | |
| 5 | Vacuum handpiece | VH-18 | |
| | | DURR | |
| 6 | Saliva ejector handpiece | BT06 | |

Sterilizable type

| Sleeve No. | Applicable handpiece/syringe | | |
|------------|------------------------------|---------------------------|--|
| A1 | Syringe | 77-type 3WAY | |
| A2 | Syringe | BT14 | |
| | | LUZZANI 3WAY (MINI LIGHT) | |
| | | LUZZANI 6WAY (MINI LIGHT) | |
| | Curing light | SATELEC MINI LED STD OEM | |
| A3 | Syringe | DCI | |
| | | LUZZANI 3WAY (MINI MATE) | |
| | Saliva ejector handpiece | DURR | |
| A4 | Vacuum handpiece | VH-18 | |
| | | DURR | |

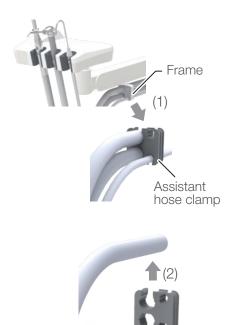

7-4-2 Assistant hose clamp

Assistant hose clamp of the assistant unit is detachable from the frame, which is suitable for cleaning.

When it becomes dirty, wipe it well with a soft cloth moistened with FD366 manufactured by Dürr.

[Method to detach]

- (1) Remove the assistant hose clamp from the frame.* Be sure to remove it by hands.
- (2) Remove the hose from the assistant hose clamp.

[Method to reattach]

Reattach it in the reverse order of detaching procedures.

Do not autoclave them. Autoclave sterilization may deform or discolor them.



Autoclave them at a temperature of 134°C for 3 minutes. Or, wipe off the surface with soft cloth moistened with disinfectant FD366 manufactured by Dürr.

Before sterilization, remove dirt on the surface and wash them under running water.


Be sure to put them in a sterilization bag during the autoclave process.

Do not perform dry processing.

Repeating autoclave sterilization may cause degradation or discoloration, but doesn't affect the product functions.

If chemical solutions are spilled on the equipment, immediately wipe them off. Otherwise, discoloration may occur.



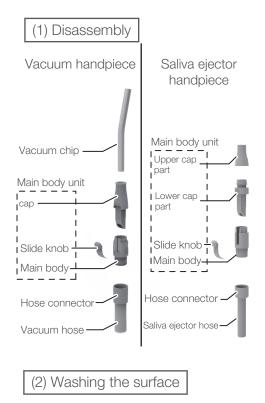




(3) Washing the interior and sliding part

Scrubbing brush or toothbrush



Sliding groove

(4) Washing parts inaccessible with a brush

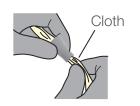

7-4-4 Washing and sterilizing the vacuum handpiece and saliva ejector handpiece

Wash and sterilize the handpieces between patients.

To properly sterilize the vacuum and saliva ejector handpieces, it is necessary to wash them to remove dirt and immerse them in detergent.

Then, rinse them to remove any remaining detergent.

Follow the procedure below to wash and sterilize the handpieces.

(1) Disassembly

To prepare for washing, disassemble the handpieces as shown in the figure.

Hold the hose connector, and pull the vacuum hose and saliva ejector hose to detach them from the main body.

(2) Washing the surface

Rinse the surface with clean water (tap water) warmed to a temperature of $40 \pm 5^{\circ}$ C, and rub it with a cloth to remove dirt. When all dirt is removed, wipe it dry.

(3) Washing the interior and sliding groove

Rinse the interior and sliding groove of the main body with clean water (tap water) warmed to a temperature of $40 \pm 5^{\circ}$ C, and scrub them with a scrubbing brush or toothbrush. When all dirt is removed, wipe them dry.

(4) Washing parts inaccessible with a brush

If some parts are inaccessible with a brush, rub them with a cloth. Rinse them well with clean water (tap water) warmed to a temperature of $40 \pm 5^{\circ}$ C (for at least 1 minute).

Immerse the parts in ID212 manufactured by Dürr or alkaline cleaner for 5 minutes. Then, rinse them well with clean water (tap water) warmed to a temperature of $40 \pm 5^{\circ}$ C (for at least 1 minute).

Inspect the parts for any visible dirt. If any visible dirt remains, repeat the above washing process.


Wash them immediately after use.

If the parts are in the condition described below after washing, do not autoclave them. Replace them.

A hole/holes are clogged, and dirt cannot be removed.

Dirt on some parts or solid dirt cannot be removed.

Caution when assembling vacuum handpiece before sterilization VH-18

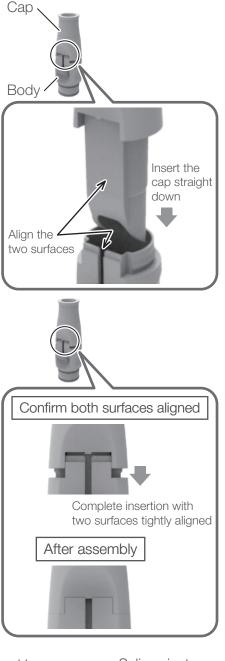

Vacuum handpiece Saliva ejector handpiece



Sterilization bag

(5) Sterilization

The vacuum handpiece and saliva ejector handpiece are autoclavable. Reassemble the main body unit, and autoclave the vacuum and saliva ejector handpieces.

When assembling the vacuum handpiece, align the two surfaces; a flat surface of the cap and the surface of the sliding groove of the body. Then, slide the cap into the body straight. * Do not insert it twisted.

- 1. Put the handpiece in a sterilization bag, and seal the opening.
- 2. Autoclave sterilization

Sterilization temperature and sterilization time for each vacuum handpiece are as follows.

DURR : 134°C for 5 minutes.

VH-18 / BT06 : 134°C for 3 minutes.

If the temperature is to exceed 135°C during the drying process, do not perform the drying process and dry naturally. The handpieces can be autoclaved up to 250 times.

Storage method: After sterilization, store the handpiece in the sterilization bag in a dark, cool place.

Vacuum Saliva ejector handpiece handpiece Apply Vaseline Apply Vaseline Vacuum handpiece Cup Saliva ejector handpiece Cup Vacuum hose Saliva ejector hose

Joint

ejector After autoclave sterilization, disassemble the handpieces, apply biece Vaseline, and reassembly before use.

If the handpieces are used without Vaseline, a packing may be damaged, resulting in leakages.

Vacuum handpiece / Saliva ejector handpiece

Suction a cup of water (approx. 100 ml) or more into the vacuum and saliva ejector handpieces after use by a patient. This is to clean and dilute the medicine used and to prevent the handpieces deteriorating.

Some medicines used for the procedure may cause deterioration of the handpiece. It may dissolve, deform, or damage part of the handpiece, possibly resulting in leaks from the handpiece or suction failure. This will ultimately makes the handpiece unusable. Please wash them properly to ensure long-term use.

Vacuum hose / Saliva ejector hose

The vacuum hose and saliva evacuation hose can be removed by holding the joint and pulling the hose.

- The exterior of the hose can be washed under tap water.
- * For washing the hose interior (of the vacuum and saliva ejector lines), refer to [Page 205].

To reattach the hose, insert the joint into the receptacle.

Always turn off the main switch before pulling the hose off to wash it.



7-4-5 Vacuum line / Saliva ejector line

Handpieces for vacuum and saliva ejector suction the secretions, saliva, or blood that contains bacteria. Therefore, always wash and sterilize them using Orotol plus® manufactured by Dürr after the procedure.

Do not use any detergent except our designated one. Otherwise, strong acidic detergents or alkaline drain preparations may cause clog, damage, or metal corrosion.

7-4-6 Cleaning of 77-type 3WAY syringe

[Reference] Cleaning of 77-type 3WAY syringe [page 197]

7-4-7 Sensor window for cupfiller

Wipe the surface with a soft cloth or paper towel moistened with FD366 manufactured by Dürr, and then wipe it with a dry cloth.

If the exterior is excessively dirty, moisten a soft cloth with water containing approx. 10% neutral detergent, and wipe the exterior with the cloth. Then, wipe it with a cloth moistened with water and then dry it completely with a dry cloth.



7 Maintenance and Cleaning



7-4-8 Cuspidor bowl

Cuspidor bowl / Cupfiller unit

Use MD550 manufactured by Durr as detergent for cuspidor bowl.

(1) Cuspidor bowl

The cuspidor bowl is detachable and can be washed under tap water. Lift it in the direction shown with the arrow to detach it. After reattaching it, check that it is securely mounted before use.

A CAUTION

Put on the cleaning gloves when detaching cuspidor bowl. Cuspidor bowl is made of ceramic that might cause injury when broken.

Always turn off the main switch before detaching the cuspidor bowl.

(2) Strainer (3) Drain outlet cover

Since the strainer easily gets clogged, detach the (3) drain outlet cover and clean the filter after consulting hours every day.

(4) Cupfiller base (5) Sensor window for cupfiller

Before cleaning, turn off the main switch, or switch the mode to the sensor disabled mode. [Page 136]

(6) LED indicator

Wipe the LED indicator with a soft cloth moistened with FD366 manufactured by Dürr to remove dirt and water droplets.

(7) Lower section of the drain trap

If the drain trap has not been completely drained, dirt may accumulate on the inside wall. Turn the cuspidor bowl by 90 degrees as shown in the figure, turn the (7) lower section of the drain trap counterclockwise to detach, and clean it. When detaching the (7) lower section of the drain trap, dirty water may overflow from the inside. Always detach the (2) strainer and (3) drain outlet cover, and flush out the cuspidor bowl with a cup of water before detaching the (7) lower section of the drain trap. (This reduces the quantity of dirty water in the drain trap.)

To reattach it, rotate the lower section of the drain trap clockwise until the vertical line meets the triangle mark.

When the lower section of the drain trap is detached, do not flush water in the cuspidor bowl.

To avoid the odor from the drain, flush the cuspidor bowl after cleaning.

Do not use sandpaper, wire wool, abrasive sponge, or abrasive cleaner.

Do not use any detergent except our designated one. Otherwise, strong acidic detergents or alkaline drain preparations may cause clog, damage, or metal corrosion.

Do not allow any metal object to come into contact with the cuspidor bowl when detaching or reattaching it since contact may leave a metal mark. If the metal object marks the bowl, use a metal mark removing agent (commercially available).

* A metal mark is a black or silver line caused by strong contact and friction between the ceramic surface and the metal object.

Before detaching or reattaching the cuspidor bowl, always turn off the main switch. When cleaning the cupfiller base, turn off the main switch, or switch the mode to the sensor disabled mode. When detaching or reattaching the bowl, the cupfiller sensor may detect your hand or other object, causing water for flushing the bowl or cupfiller base to splash around, which can cause metal parts to rust.

The cuspidor bowl is made of ceramic. It may break if it receives a hard knock. Do not wash it with boiling water, as this may break it.

Since the cuspidor bowl can be easily detached or reattached, do not apply excessive force except when washing. (Do not knock it, hit it, or drop it.)

Do not pour water over the cupfiller base when cleaning it. It is not waterproof, and excessive water may cause it to malfunction. For cleaning the cupfiller base, refer to the [page 206].

Do not let water flow onto the cupfiller base. The drainage function of the cupfiller base is designed to drain away any overflow water when filling a cup.

7-4-9 Solid collector

After consulting hours, remove the filter from the solid collector in the cuspidor unit, and wash it.

Accumulated debris may degrade the suction force of the vacuum handpiece or saliva ejector handpiece.

[Method to detach/reattach the filter]

1. Press the maintenance panel cover (top) of the cuspidor unit to open it.

Maintenance panel cover (top)



2. Pull off the solid collector.


З.

5.

ection.)


1.


2.

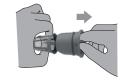





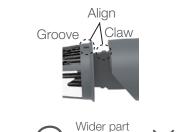

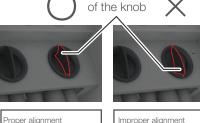

The wider part of the knob is located in the upper

Improper alignment The wider part of the knob s not located in the upper ection.)

- 3. Detach the solid collector cap from the filter. Pull the filter to detach the solid collector cap as shown in the figure on the left.
- 4. Wash the solid collector cap and the filter under running water.
- 5. Align the claw on the solid collector cap with the groove on the filter to attach them.
- 6. Attach the solid collector in its original position with the cap aligned as shown in the left figure.

Press the solid collector into the end of the receptacle with the proper alignment. If it is improperly mounted, the vacuum handpiece and saliva ejector handpiece may not function properly.

7-4-10 Spittoon valve

Clean the Spittoon valve twice in a day at the end of morning treatment and evening treatment. (At the end of evening treatment if cleaning once in a day.)

1. Press the function switch 🖄 . The Function setting screen appears.

2. Press 'Spittoon valve cleaning'. Spittoon valve cleaning screen appears.

3. Add 1 liter of diluted solution of "Orotol plus" into the cuspidor bowl, then press ок.

- 4. Spittoon valve cleaning starts 'Spittoon valve in cleaning' message appears on the screen.
- 5. Spittoon valve cleaning finishes 'Spittoon valve in cleaning' message disappears from the screen.




| 🔅 Fund | ction setting | 5 |
|--------|-----------------------------------|--------------|
| | Flushing, Washing the vacuum line | ~ |
| | Spittoon valve cleaning | \sim |
| | Handpiece | \mathbf{v} |
| | Language | • |

| Spittoon valve cleaning | | 5 | |
|--|----------|----|--|
| Add 1 liter of diluted solution of "Orotol plus" cuspidor bowl. | into the | 5 | |
| The cleaning will start after pressing or . | | | |
| | | | |
| | | ок | |







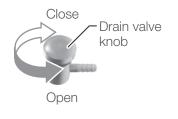

7-4-11 Air filter

Empty any water that has collected in the air filter at least once a week.

- 1. Open the Junction cover.
- 2. Prepare a cup to put the water in.
- 3. Turn the drain valve knob counterclockwise, and discharge the water into the cup.
- 4. When no more water comes out, turn the drain valve knob clockwise to close it.

After closing the drain valve knob, water remained in the tube may come out.

Wipe the water with cloth to prevent the water from coming out.

If any water gets into the air line, it may cause a failure of the product.

7-4-12 Air compressor

For maintenance of the air compressor, follow the Instructions for Use supplied with the air compressor.

7-4-13 Vacuum pump

For maintenance of the vacuum pump, follow the Instructions for Use supplied with the vacuum pump.

7-5 How to detach battery of the doctor unit


Instrument delivery: Holder/Place holder



1. Remove the four screws under the doctor table and detach the doctor table.



2. Remove the seven screws and detach the table top.

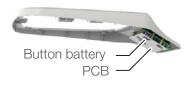

3. Detach the button battery (CR1632) on the PCB.

7-6 Maintenance and inspection Notes on daily maintenance and inspection (by the user)

It is the responsibility of the user (medical institution) to ensure that the medical device is correctly maintained and inspected. To ensure safe use of this product, the unit must be inspected at the specified intervals as described in the table below:

| No | Inspection item | Inspection interval | Inspection procedure and criteria | Outcome if inspection is not conducted | Maintenance required when the inspection criterion is not satisfied |
|----|---|-------------------------------|--|--|--|
| 1 | Check the cancel function | Before consulting hours | When the chair is in automatic operation, it must stop when any of the operation switches shown on [Page 102] is pressed or stepped on. | The chair unexpectedly moves during procedure, resulting in an injury. The patients may be caught between the doctor unit and the chair, resulting in an accident. | If the chair does not stop, please contact your local authorized Belmont dealer. |
| 2 | Check for water, air and oil leaks | Before consulting hours | Check that no water, air or oil leaks out from the product. | The product does not function properly, preventing proper treatment or procedure. | If water, air or oil leaks out, please contact your local authorized Belmont dealer. |
| 3 | Cup-filling | Before consulting hours | Place a cup on the cupfiller base, and check that the cup is detected and correctly filled. * Use a cup with a weight of 3 grams or more. | The cup is not filled with water in some cases. | Follow the description under 'Cupfiller,' and inspect it again.[Page 136] If it cannot be solved after re-inspection, please contact your local authorized Belmont dealer. |
| 4 | Check the functioning of each handpiece | Before consulting hours | Check that the turbine rotates properly and that the correct quantities of water and air are supplied. Check that the micromotor rotates properly and that the correct quantity of water is supplied. Check that the ultrasonic scaler vibrates properly and that the correct quantity of water is supplied. Check that the correct quantity of water is supplied. Check that the correct quantities of water and air are supplied from syringe. | The patient may receive an injury to their mouth, or the handpiece may malfunction. | Adjust the quantity of water or air. [Page 187] For other failures, please refer to the Instructions for Use for the respective handpieces. If the problem still persists, please contact your local authorized Belmont dealer. |
| 5 | Check the correct burr for the turbine, air motor, and micromotor is mounted. | Before each patient | Check that the correct burr is securely mounted. Make sure you refer to the Instructions for Use for the turbine, air motor and micromotor. Check that the burr is free of any defect (damage or deformation). | The burr will not rotate freely, resulting in an accident. | If the burr is damaged, deformed or has some other defect, replace the burr by following the Instructions for Use for the turbine, air motor and micromotor. |
| 6 | Check the scaler tip | Before each patient | Check that the correct tip is securely mounted and properly used. Refer to the Instructions for Use for the scaler. Check that the tip is free from any defect (wear or deformation). | The tip will not vibrate properly, resulting in an accident. | If the tip is worn or deformed, replace it by following the Instructions for Use for the scaler. For other defects, please contact your local authorized Belmont dealer. |
| 7 | Check that the syringe nut is tight | Before each patient | Check that the nut that secures the nozzle of the Type 77 / Radis 3-way syringe is securely tightened. | The nut may come off, resulting in an accident. | Follow the Instructions for Use for the Type 77 3-way syringe to tighten the nut that secures the nozzle and confirm secure tightening. |
| 8 | Debris on the micromotor | After consulting hours | Check that no excess oil from the handpiece adheres to the motor unit. | The motor unit may not function properly, resulting in a malfunction. | Follow the Instructions for Use for the micromotor to maintain it. |
| 9 | Maintenance Vacuum handpiece / saliva ejector handpiece | After consulting hours | Wash the vacuum lines and saliva ejector lines. | Suction is defective. | Wash the vacuum lines. [Page 205] |

7 Maintenance and Cleaning

| No | Inspection item | Inspection interval | Inspection procedure and criteria | Outcome if inspection is not conducted | Maintenance required when the inspection criterion is not satisfied |
|----|---|------------------------------|---|---|--|
| 10 | Maintenance Cuspidor bowl | After consulting hours | Check that the cuspidor bowl does not contain any dirt (or excrescence). Check that no dirt has collected on the dirt filter. | Draining is defective. | Clean the cuspidor bowl and dirt filter. [Pages 206, 207] |
| 11 | Maintenance Solid collector | After consulting hours | Check that no dirt has collected on the solid collector. | Suction power of the vacuum handpiece or saliva ejector handpiece has decreased. | Clean the filter. [Page 208] |
| 12 | Maintenance Exterior | After consulting hours | Check that no chemical solution or dirty water adheres to or remains on the exterior of the product. | Any liquid remaining will causes discoloration or change the properties of the exterior or cause metal parts to rust. | Clean it. [Page 189] |
| 13 | Check the main switch and main water valve | After consulting hours | Check that the main switch is turned off, and the main water valve is closed . | May result in a short circuit or water leaks. | If the main switch cannot be turned off or the main water valve cannot be closed, please contact your local authorized Belmont dealer. |
| 14 | Movable parts of the product | Once a week | When operating the product, check that no movable parts emit any abnormal noise. | The product does not function properly, preventing proper treatment or procedure. | If any movable parts emit an abnormal noise, please contact your local authorized Belmont dealer. |
| 15 | Maintenance Drain valve | Once a week | Drain water from the drain valve on the air filter. | Water enters the air line, resulting in a malfunction. | Always drain the air filter. [Page 210] |
| 16 | Check water and pneumatic pressures | Once a month | Check the water and pneumatic pressures using the pressure gauge on the maintenance panel. Main water pressure : 0.1 to 0.2 MPa Main air pressure : 0.45 to 0.5 MPa | The product does not function properly, preventing proper treatment or procedure. | If the pressure is out of the range of the main water pressure/main air pressure, please contact your local authorized Belmont dealer. |
| 17 | Check the doctor unit | Once a month | The doctor table is level and stops at the specified position. | Objects fall from the doctor table, resulting in an injury or accident. | If the doctor table is not level or does not stop at the specified position, please contact your local authorized Belmont dealer. |
| 18 | Oil mist separator | Once a month | The drain oil does not reach the line on the oil mist separator. | The handpiece may not function properly due to an exhaust failure. | Empty the oil. [Page 198] |



Always refer to this Guide and the Instructions for Use supplied with each device (such as the dental light and handpieces) before conducting daily maintenance and inspection. If you do not conduct daily maintenance or inspection, use of the product may result in injury or damage to nearby devices.

Notes for periodic inspection

The product contains parts that stop functioning or wear depending on the use frequency, and therefore it is important to carry out maintenance in a periodic inspection once a year (including replacement of consumables) and safety checks.

Service parts required for the periodic inspection (including consumables) are listed in the table below. However, depending on the specifications of your device, there may be alternative parts available that differ from those listed in the table below.

Maintenance and inspection can be outsourced to qualified persons such as authorized repairers of medical devices.

If you have any question about periodic inspection, contact your local authorized Belmont dealer.

| Part name | Standard service life | Part name | Standard service life |
|-------------------------|-----------------------------|-----------|-----------------------------|
| Movable part | 7 years | Switch | 5 years |
| Wire for a movable part | 5 years | PC board | 5 years |

List of service parts required for the periodic inspection

List of consumables required for periodic inspection

| Part name |
|-----------|
| O-ring |
| Packing |
| Diaphragm |



Always entrust periodic inspection to your local authorized Belmont dealer.

If you do not carry out periodic inspection, use of the product may cause injury or damage to nearby devices.

7-7 Detachable parts

| Part name | Part name | |
|---|------------------------|--|
| Handpiece | Vacuum tip | |
| Handpiece hose | Saliva ejector nozzle | |
| Holder sleeve | Solid collector cap | |
| Cuspidor bowl | Solid collector filter | |
| Cover for the drain outlet of the cuspidor bowl | Water service coupler | |
| Cuspidor bowl strainer | Air service coupler | |
| Vacuum handpiece | Syringe nozzle | |
| Saliva ejector handpiece | Syringe body | |
| Oil mist separator | Syringe hose | |

7-8 Storage method

If the product is not used for an extended period of time after consulting hours or during holidays, make sure you observe the precautions below:

1. Always turn off the main switch after consulting hours. (This is to stop supply of air, water, and electric power.) Make it a habit to do this to prevent water leak and electrical accidents.

2. After consulting hours, turn the water main valve knob counterclockwise to the vertical position to close the water main valve. Make it a habit to do this to prevent accidents by water leaks.

3. Turn off the breaker for the compressor, and discharge air. (Also make sure you have turned off the power.)

4. Turn off the breaker for the vacuum pump. (Also make sure you have turned off the power.)

5. Turn off the breaker for the device in the clinic. (Also make sure you have turned off the power.)

6. Set the chair to the lowest position and the backrest to the most reclined position.

8-1 After-sales service

When you request for repair

Refer to 'Troubleshooting' [from Page 217 on] before you check the device. If the problem persists, turn off the main switch, and contact your local authorized Belmont dealer to request a repair.

8-2 Service life

The service life of this product is 10 years on condition that maintenance and inspection are properly conducted [according to our self-certification (our data)].

However, the standard service lives of service parts that require periodic inspection vary according to the part. [Page 214]

8-3 Period of Parts Retention

We hold service parts such as consumables for products for 10 years from the time of launch.

* Service parts are parts required for repair to return the product to the original state and functions or to maintain its functions.

9-1 Troubleshooting

If you encounter any of the problems listed below, take the countermeasures described below before requesting a repair. If the problem persists even after troubleshooting, stop using the product immediately, turn off the main switch, and contact your local authorized Belmont dealer.

| Phenomenon | Please check | Remedy |
|---|---|--|
| The product does not work at all. | Is the main switch turned on? | Turn on the main switch. [Page 61] |
| | Is the power to the compressor turned on? | Turn on the power. |
| | Is the breaker for the device on the switch- board of the dentist's office turned on? | Turn on the breaker for the device. |
| The chair does not move. | Is the chair lock function activated? | Unlock the chair. [Pages 103 to 106] |
| | Have you been operating the product continuously for a long time? | The overheating prevention device for the hydraulic motor has been triggered. Wait for approx. 10 minutes, and move the chair. |
| The product cannot be operated with the wireless foot controller. | Are the correct batteries inserted? | Insert the batteries with proper polarities. [Page 131] |
| | Are the batteries flat? | Replace the batteries with new (or charged) ones with proper polarities. [Page 131] |
| | Is the wireless foot controller too far away from the product? | Keep the distance between the foot controller and product to approx. 1 meter or less. |
| | If you are using more than one products, are you mixing it up with another product's foot controller? | Check the pairing identification code, and use the combination with the same codes. [Page 130] |
| No air is being supplied. | Is the power to the compressor turned on? | Turn on the power. |
| | Is the knob that controls the air supply to the syringe or other parts closed? | Open the air supply control knob. [Pages 187, 188] |
| No water is being supplied. | Is the water main valve closed? | Open the water main valve. [Page 60] |
| | Is the water supply control knob to the handpiece or syringe closed? | Open the water supply control knob. [Pages 187, 188] |
| The vacuum handpiece or saliva ejector handpiece does not | Is the power to the vacuum pump turned on? | Turn on the power. |
| activate the suction function. | Is the solid collector filter dirty? | Clean the filter. [Page 208] |
| | Is the solid collector filter properly attached? | Attach the solid collector properly. [Page 208] |
| Is there a message displayed on the touch panel? | Read the message. | Take the corrective measures indicated in the message. [Pages 218 to 220] |
| Water keeps running from the cupfiller and doesn't stop. | Is there any dirt or droplet on the surface of cupfiller sensor? | Turn off the main switch and clean the surface of the cupfiller sensor. |
| Water doesn't come out of the cupfiller. | Is there any dirt or droplet on the surface of cupfiller sensor? | Turn off the main switch and clean the surface of the cupfiller sensor. |
| | Isn't the sensor disable mode activated? | Cancel the sensor disable mode. [Page 136] |

9-2 Message on the touch panel and measures to be taken

| Message on the touch panel | Details | Remedy | |
|--|--|---|--|
| E01: Communication w/cuspidor section | Internal communication error for the device | Turn off the main switch, and contact your local authorized Belmont dealer. | |
| E02: Communication w/FC | device | iocal authorized Belmont dealer. | |
| E03: Communication w/water heater (HP) | | | |
| E04: Communication w/micromotor | | | |
| E05: Communication w/scalar | | | |
| E06: Communication w/chair | | | |
| E07: Communication w/cupfiller | | | |
| E08: Communication w/Washing the vacuum line | | | |
| E12: Control panel (Dr.) | The membrane switch is pressed when the power is turned on. | Keep your hand away from the membrane switch. If the error code does not go off when your hand is removed, contact your local authorized Belmont dealer. | |
| E13: Control panel (ASST) | The switch on the assistant's control panel is pressed when the power is turned on. | Keep your hand away from the assistant's control panel. If the error code does not go off when your hand is removed, contact your local authorized Belmont dealer. | |
| E14: Air pressure too low | The master pressure in the pneumatic system decreases. | Turn off the main switch, and contact your local authorized Belmont dealer. | |
| E20: HP picked up while FC is on | The pedal of the foot controller is stepped on when the handpiece is picked up. | Release the pedal of the foot controller. If the error code does not go off when you take your foot off, contact your local authorized Belmont dealer. | |
| E22: Foot controller (Slide) | A wire in the foot controller has become disconnected. | Turn off the main switch, and contact your local authorized Belmont dealer. | |
| E24: Communication line with FC is busy. | Communications from the foot controller are jammed. (Wireless foot controller) | Use the cable contained in the package. | |
| E25: Foot controller EEPROM | EEPROM (memory contents) read error for the foot controller | Turn off the main switch, and contact your local authorized Belmont dealer. | |
| E30: Chair EEPROM | EEPROM (memory contents) read error for the chair | | |
| E31: Potentiometer (Seat) | Chair position detection error | | |
| E32: Potentiometer (Backrest) | | | |
| E33: Potentiometer (Legrest) | | | |
| E34: Thermistor (Solenoid valve) | Chair oil temperature detection error | | |

| Message on the touch panel | Details | Remedy |
|--|--|--|
| E35: Stick switch | The stick switch is pressed when the power is turned on. | Check that there is not a leg or an object in contact with the stick switch on the chair. If nothing is in contact with it but the error code does not go off, contact your local authorized Belmont dealer. |
| E37: Potentiometer (Roll-up legrest) | Chair position detection error | Turn off the main switch, and contact your local authorized Belmont dealer. |
| E38: Potentiometer (Headrest) | * | |
| E39: Timeout (Chair) | In automatic operation, the chair cannot complete the movement to the target position within the specified time. | |
| E42: Timeout (Cupfiller) | Cup-filling was not completed within the specified time. | If the water supply is insufficient, adjust the supply with the cupfiller control knob. If the error code does not go off after adjustment, contact your local authorized Belmont dealer. |
| E43: Cupfiller EEPROM | EEPROM (memory contents) read error in cupfiller | Turn off the main switch, and contact your local authorized Belmont dealer. |
| E44: No water in the bottle | The water level detection inside the bottle or the solenoid valve is behaving abnormally. | Press the OK switch. Return the vacuum and saliva ejector handpieces to the instrument holder, and then contact your local authorized Belmont dealer. |
| E45: Water in the bottle cannot be discharged. | The vacuum and saliva ejector handpiece are not connected to the insertion point. | Press the OK switch and try again. |
| | The water level detection inside the bottle or the water suctioning function is behaving abnormally. | Press the OK switch. Return the vacuum and saliva ejector handpieces to the instrument holder, and then contact your local authorized Belmont dealer. |
| E46: Cleaner pump is not functioning. | The pump is overloaded. | iogai autrionzed deimont dealer. |

| Message on the touch panel | Details | Remedy |
|--|--|---|
| C01: Cancelled. Dr table is in the way. | The doctor table has been moved behind the backrest while the chair is moving, and the chair stops in the middle. | Move the doctor table next to the armrest, and move the chair. |
| C02: Cancelled. Dr table is in contact. | An object is interfering with the doctor table while the chair is moving, and the chair stops in the middle. | Remove the cause of the emergency stop, and move the chair. When the chair moves properly, the error code will go off. |
| C03: Cancelled. Asst arm is in contact. | An object is interfering with the assistant arm while the chair is moving, and the chair stops in the middle. | |
| C04: Cancelled. Cuspidor bowl is in the way. | The cuspidor bowl is removed while the chair is moving, and the chair stops in the middle. | Return the cuspidor bowl, and move the chair. |
| C05: Cancelled. Footrest is in contact. | An object is interfering with the footrest while the chair is moving, and the chair stops in the middle. | Remove the cause of the emergency stop, and move the chair. When the chair moves properly, the error code will go off. |
| C06: Cancelled. Backrest is in contact. | An object is interfering with the backrest while the chair is moving, and the chair stops in the middle. | |
| C07: Cancelled. Rear link cover is in contact. | An object is interfering with the rear link cover while the chair is moving, and the chair stops in the middle. | |
| C08: Cancelled. Utility box is in contact. | An object is interfering with the top surface of the junction box and the junction box lid has come off while the chair is moving, and the chair stops in the middle. | Remove the cause of the emergency stop, fix the lid of the junction box back in place, and move the chair. |
| C09: Cancelled. FC is in contact. | The pedal is stepped on while the chair is moving, and the chair stops in the middle. | Release the pedal of the foot controller, and move the chair. When the chair moves properly, the error code will go off. |
| C10: Wait. Chair is warming up. | When the temperature is low, the chair warming-up function is activated. | Wait until the error code goes off (or warming up is completed) |
| C12: First priority (Handpiece) | The priority function is activated. | Return all the handpieces to the instrument holder. If the error code does not go off, contact your local authorized Belmont dealer. |
| C13: Return handpiece to holder. | A handpiece is not placed in the instrument holder when the power is turned on. | Check that all the handpieces are placed in the instrument holder. If the error code does not go off with all the handpieces in place, contact your local authorized Belmont dealer. |
| C14: LP position | When the chair is to be moved from the last position to the gargle position, the chair is already at the gargle position | Move the chair using a switch other than the last position switch. |

10-1 Accessories

Instructions for Use (this document) Installation Instructions Declaration of Conformity Cable (only for the wireless foot controller)

10-2 Consumables

Consumables are parts that will normally wear or deteriorate, change their appearance, or become damaged after use. Please note that repair or replacement of consumables are not covered by the warranty and will be charged for.

(* Degree of wear, deterioration or damage and timing for replacement depends on the use environment and conditions at the customer's premises.)

Consumables (Parts listed below are out of the guarantee coverage and charged parts.)

[Reference] List of service parts required for the periodic inspection [Page 214]

EC REP

TAKARA COMPANY EUROPE GmbH

Berner Strasse 18, 60437 Frankfurt am Main, Germany TEL : +49-69-506878-0 FAX : +49-69-506878-20



[]Belmont



